Etac Easy / Smart / Edge / Swift / Swift kommod

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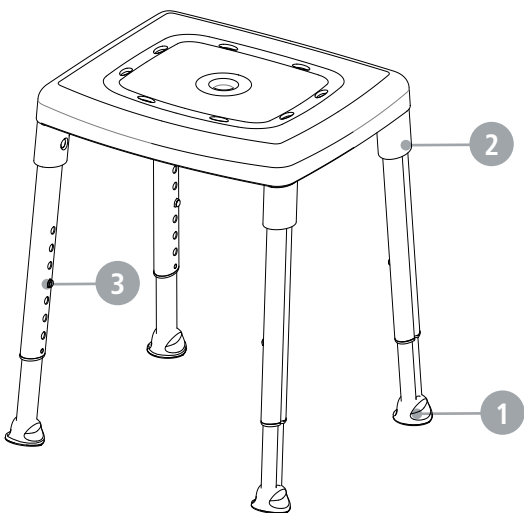
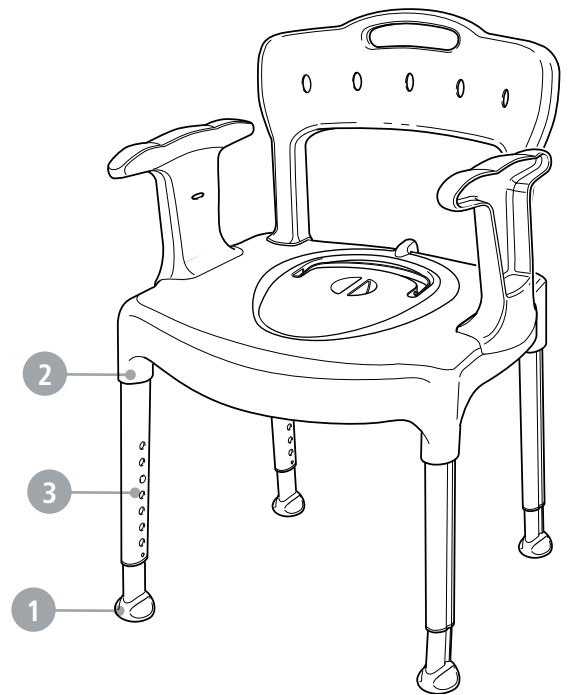
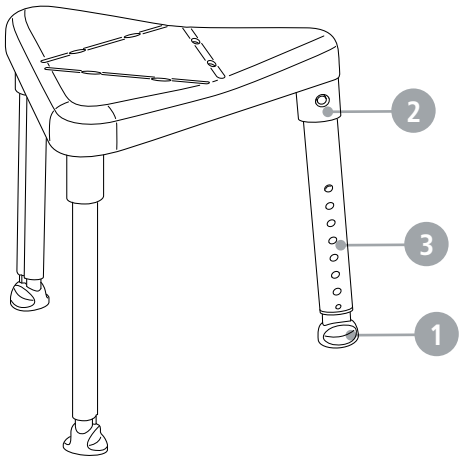
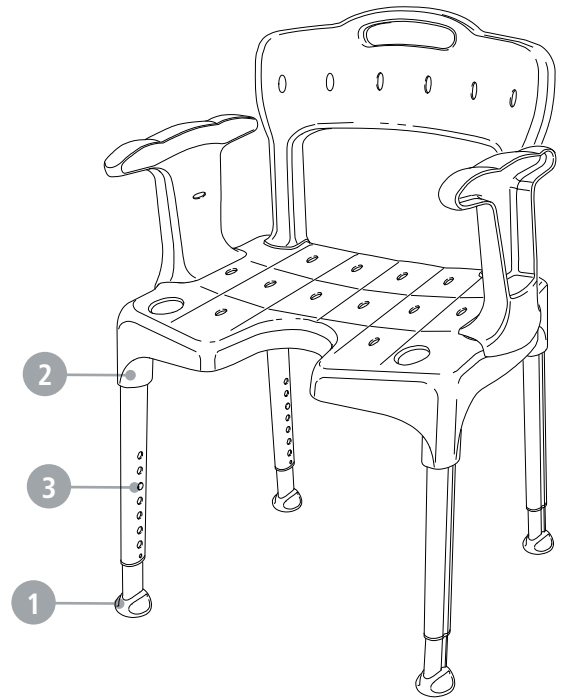
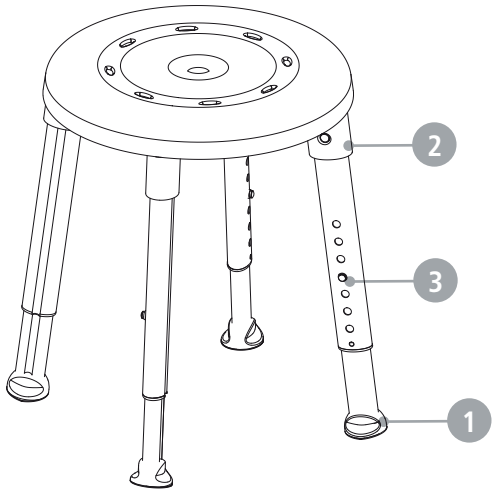
General

- The products may be further used.
- Original spare parts from Etac should always be used.
- Remember that the service life of the product is limited

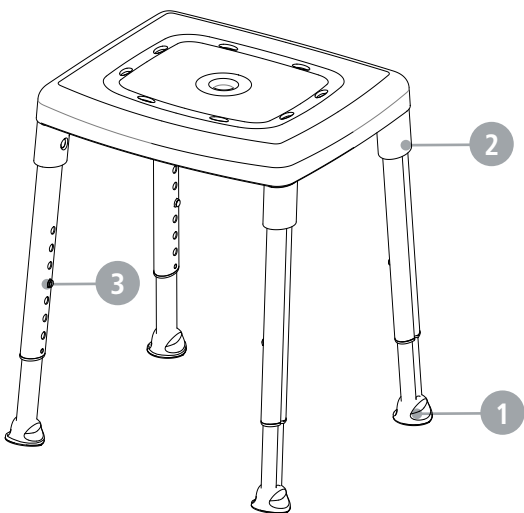
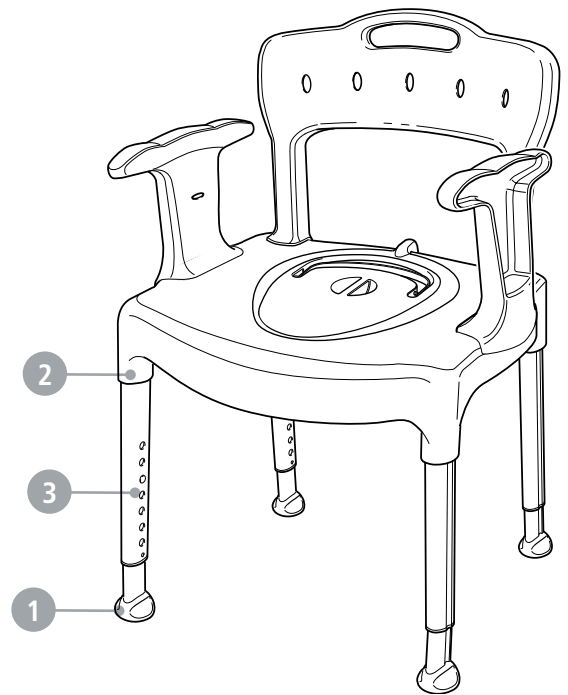
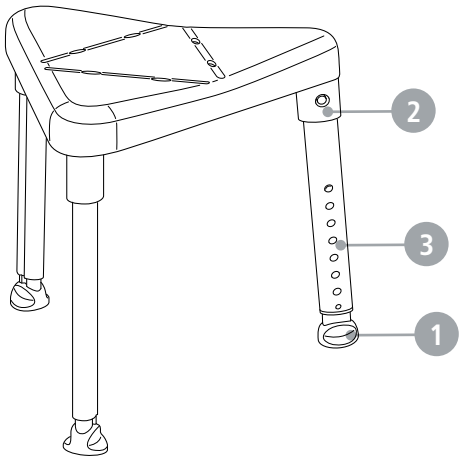
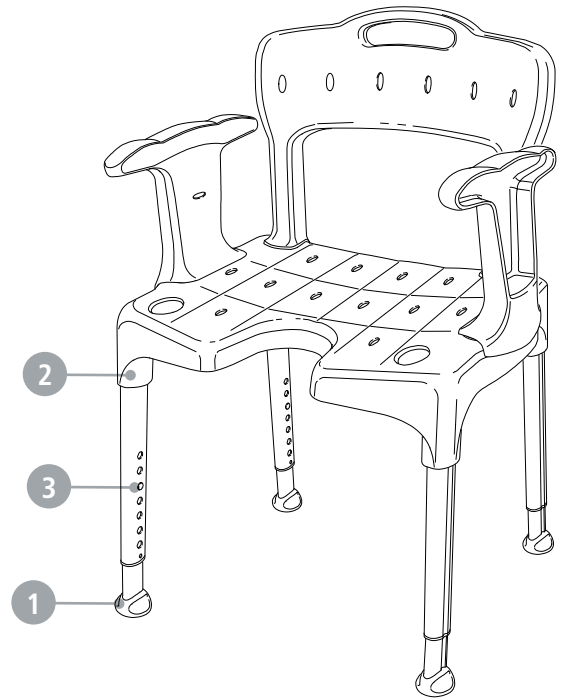
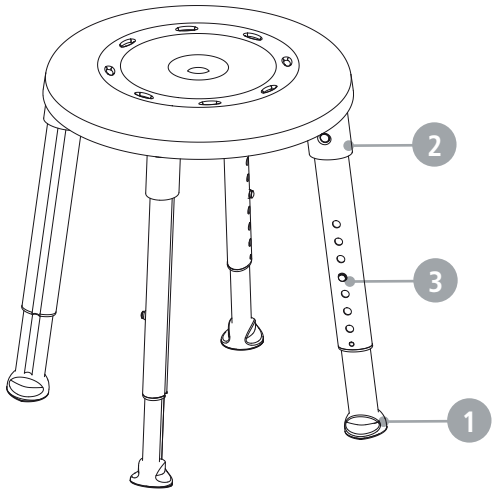
(see manual). This is particularly important when prescribing the product to a user who is expected to use it for an extended period of time.

Washing

- The products must be cleaned using conventional non-abrasive cleaning agents (pH value of between 5-9). They must be cleaned regularly. The products and their accessories can be autoclaved at 85° for a maximum of 3 minutes and also dried in a drying cabinet at 70° for 10 minutes (unless otherwise stated).

Disinfect the product with an alcohol-based disinfectant, 70%.

1	Ferrules
Check that the leg ferrules are soft, intact and functional. Replace parts if in doubt. Check that there are no cracks or damage.	
2	Legs attachment
Check that the seat's leg attachments have not become warped or been tampered with. Check that the legs stand symmetrically on the floor and have not become warped.	
Check that there are no cracks or damage.	
3	Legs height adjustment
Check that there are height adjustment buttons on the legs and that they function satisfactorily. Perform a functionality test of leg height adjustment.	
Check that there are no cracks or damage.	
4	Plastic components
Make sure that plastic parts have no cracks. Check that all plastic parts are intact.	

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