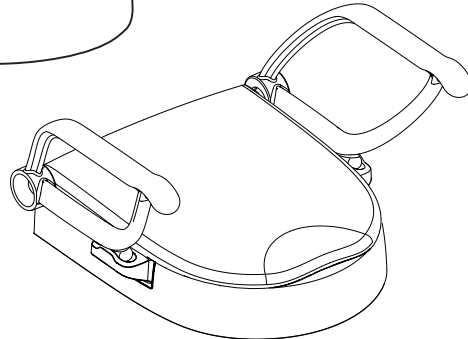
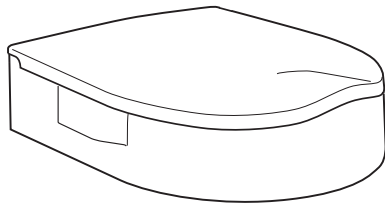
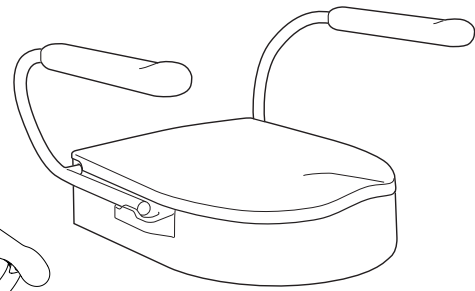
Cloo

from. Serie nr. 28810

tom. Serie nr. 28809

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
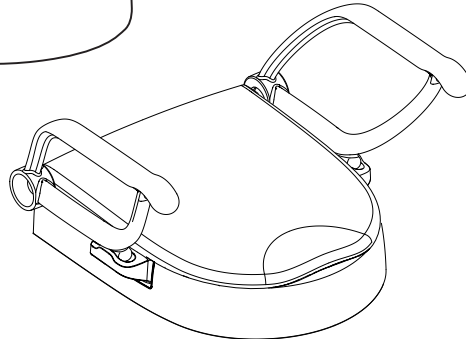
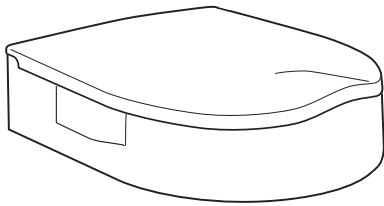
from. serial no. 28810

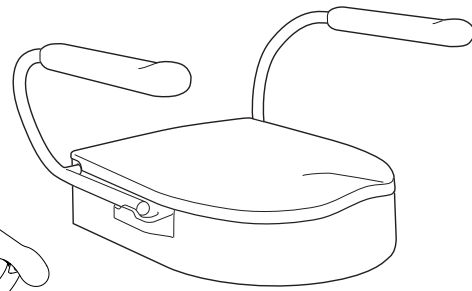up to serial no. 28809

Spare parts can be ordered as per the separate spare parts list for Cloo. Remember that the service life of the product is limited (see manual). This is particularly important when prescribing the product to a user who is expected to use it for an extended period of time. Washing and cleaning instructions for the product are found on the product or in the manual.

The products must be cleaned using conventional non-abrasive cleaning agents. They must be cleaned on a regular basis. The products can be cleaned in a machine at 85° for a maximum of 3 minutes.

When reconditioning it is recommended to inspect the following:

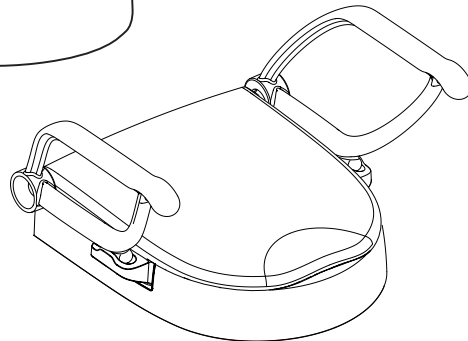
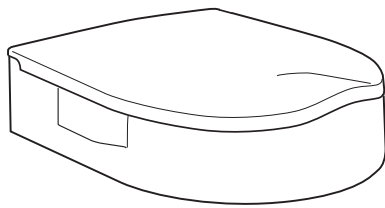
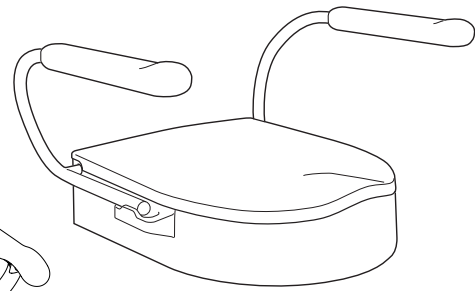
- 1) Check that the quick-release nuts, top cones and adapters are in good condition. As these components are exposed to major strain and an aggressive environment, a detailed inspection is required. The components can be seen as safety components and for this reason must be replaced if there is any doubt. Also check that the quick-release nuts are properly tightened.
- 2) Check that height spacers, brackets and anti-skid feet are properly tightened. As these are important for the stability and safety of the product, it is important that they are fitted properly.
- 3) Check that the armrest support is securely fastened. This is important as it supports the armrest when it is put under pressure.
- 4) Check that the armrest washers are securely fastened. Check that the screws are properly tightened and that the armrest washer is not defective. (Applies up to and including Serial no. 28809).
- 5) Check that the locking components are in working order (the plastic components that lock the seat unit in place to the attachment). Check that the holder for the seat unit is intact and that the red marking is facing forward.
- 6) Check that the different functions in the product are in working order. Check that the seat unit can be fitted/removed from the attachment. Check that the product can be folded up and down. Check that the armrests can be folded up and down if they are not fitted in place.
- 7) That detachable items are in good condition and are Etac's original products. This is done to take full responsibility for the product once it has been reconditioned.
- 8) Repair lacquer damages with Etac's touch-up paint RAL 9003 (gloss 90)

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