max 400 mm (15½")

max 75 kg (165 lb)


Manatee User Guide






EN 12182
ISO 17966

2018-08 rev.05

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support.R82.org 


	3	ENGLISH DEUTSCH	19 - 22 23 - 26
	4	NEDERLANDS DANSK	27 - 30 31 - 34
	5	NORSK SVENSKA	35 - 38 39 - 42
	6	SUOMI ESPAÑOL	43 - 46 47 - 50
	7	FRANÇAIS ITALIANO	51 - 54 55 - 58
	8	PORTUGUÊS PORTUGUÊS DO BRASIL	59 - 62 63 - 66
	9	РОССИЯ POLSKI	67 - 70 71 - 74
	10	ČESKY 中國	75 - 78 79 - 82
	11	日本 ΕΛΛΗΝΙΚΗ	83 - 86 87 - 90
	11		
	12		
	13 - 15		
mm (inch)	16 - 17		
	18		

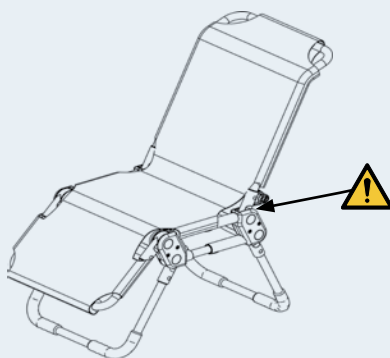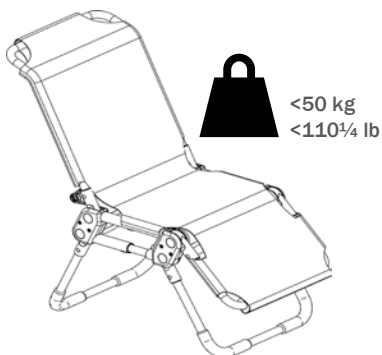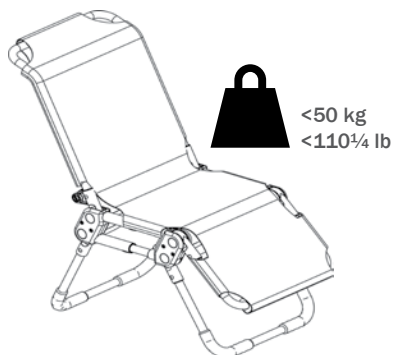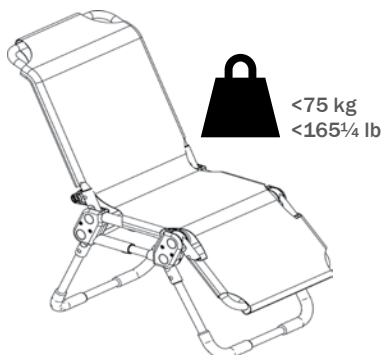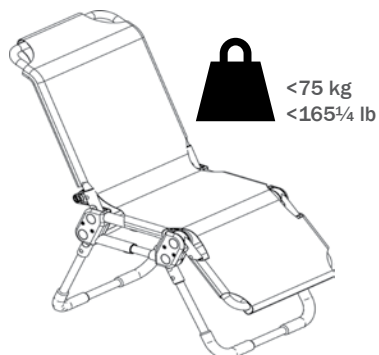**MANATEE 0****MANATEE1****MANATEE 2****MANATEE 3**

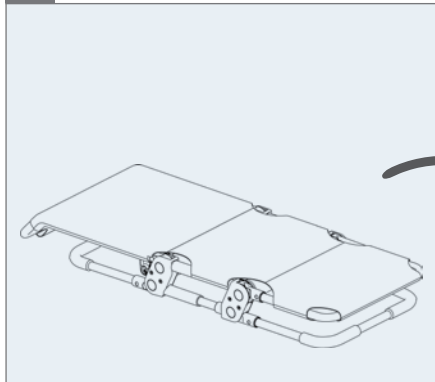
1.


2.

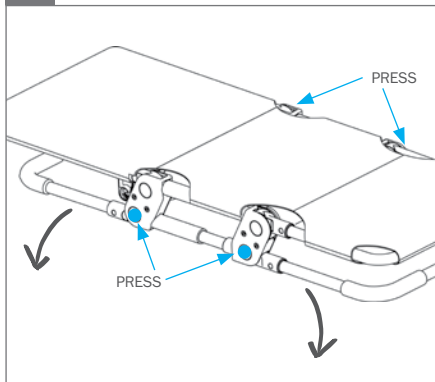
3.

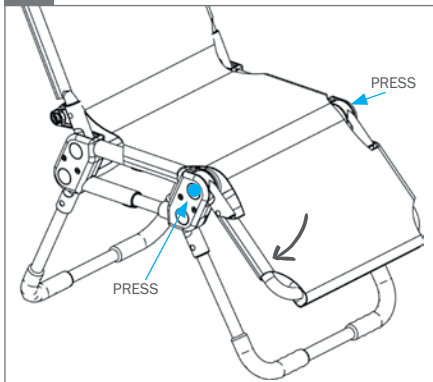
4.

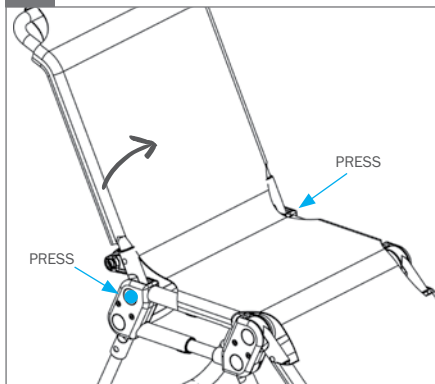5.

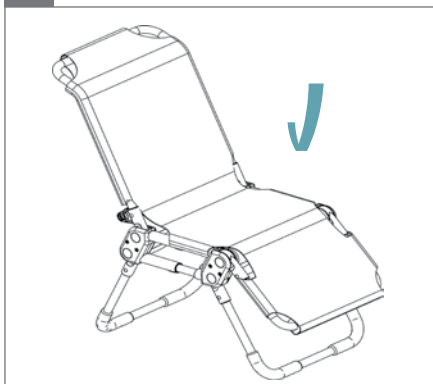
6.


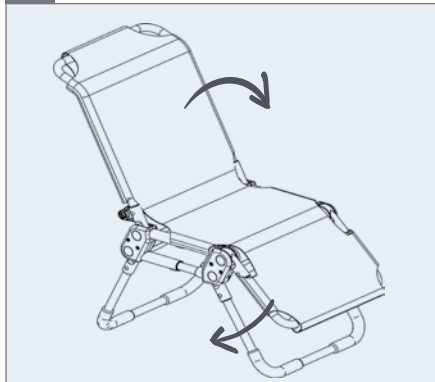
1.

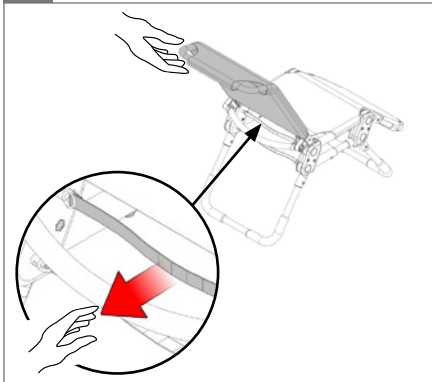
2.

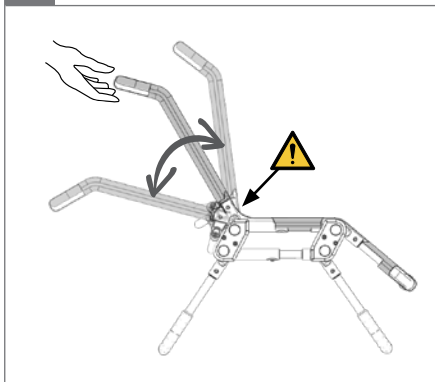
3.

 05

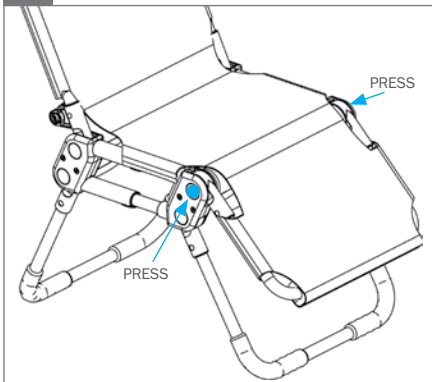
4.

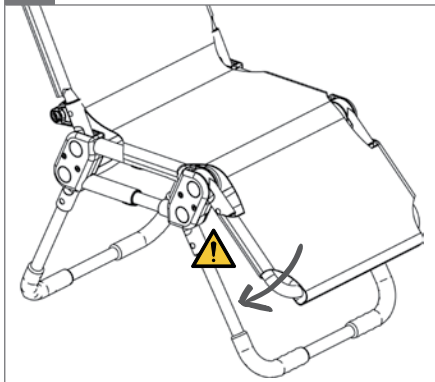5.


 05

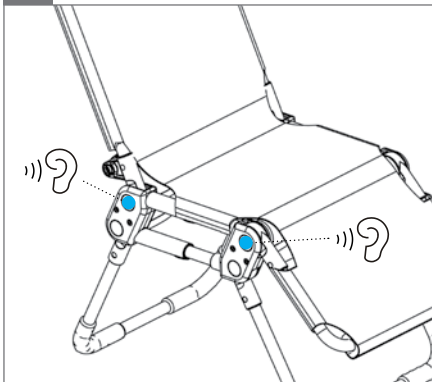6.

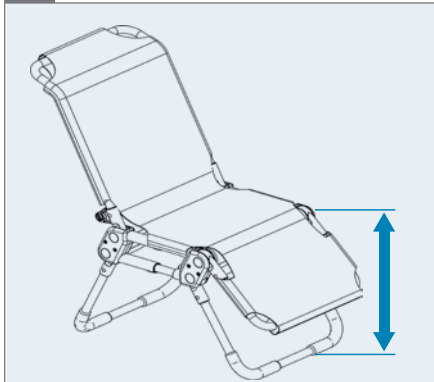
1.

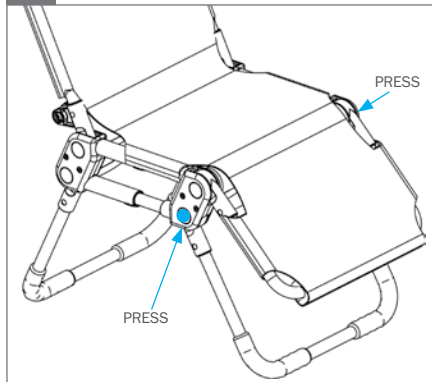
2.


3.

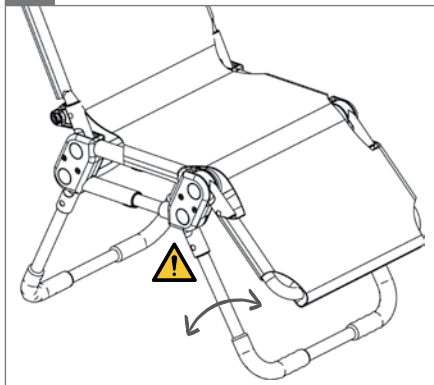05

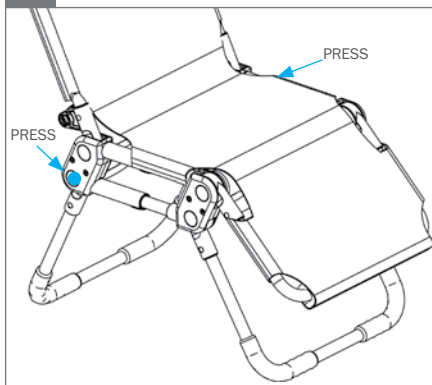
4.


5.

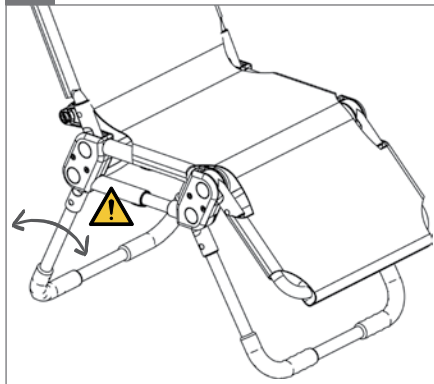05

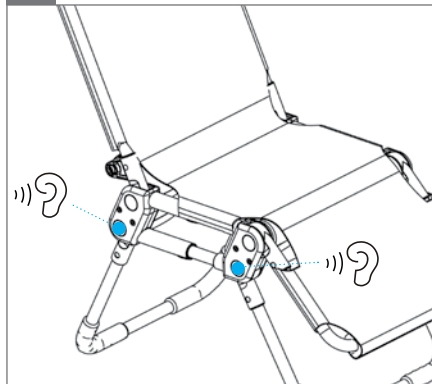
6.

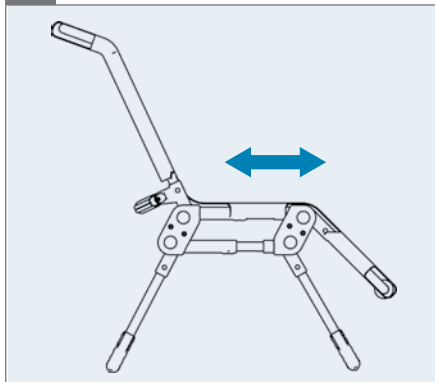
1.

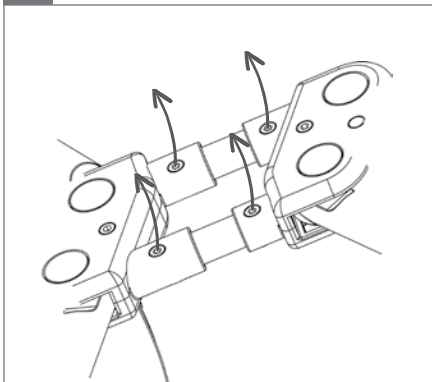
2.


Mantee 0 - $\varnothing$ 4 mm

3.

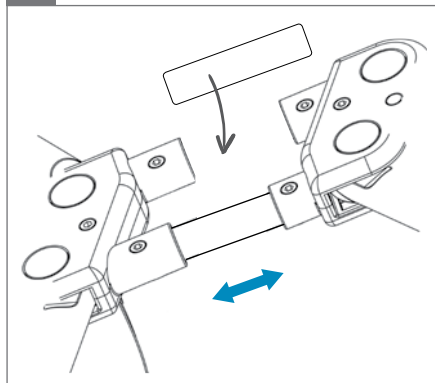
Mantee 0

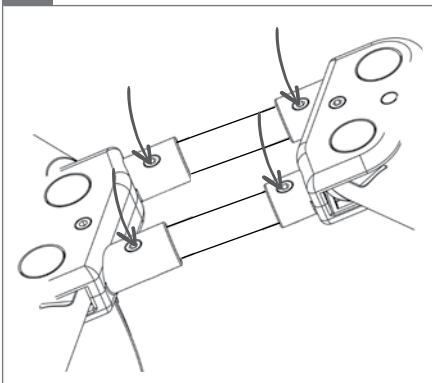
4.

Mantee 0 - $\varnothing$ 4 mm, 2,5 Nm

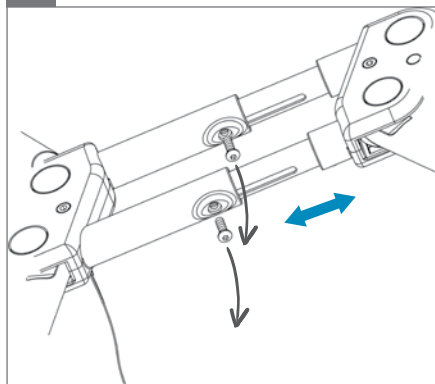5.


Manatee 1+2 - $\varnothing$ 4 mm

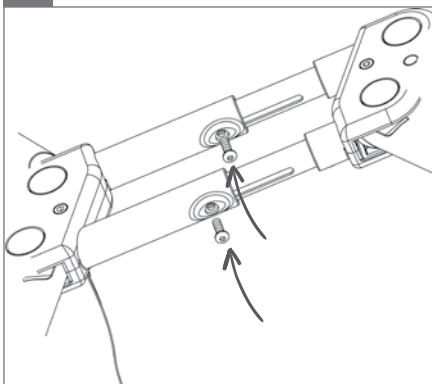6.


Manatee 1+2 - $\varnothing$ 4 mm, 2,5 Nm

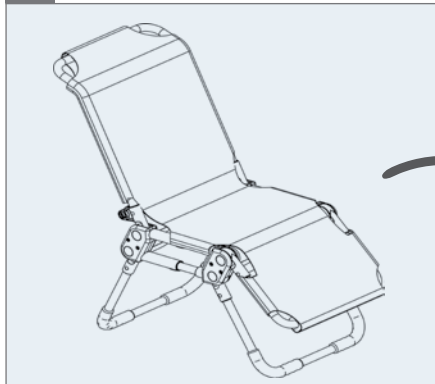
1.

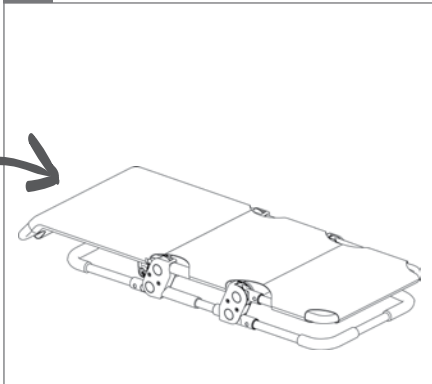
2.

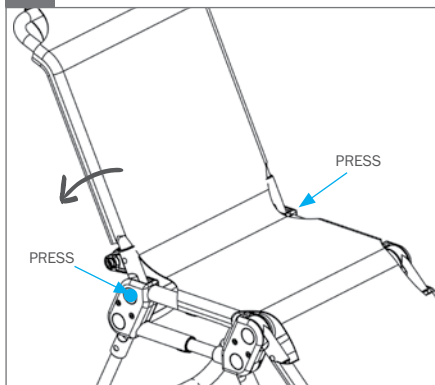
3.

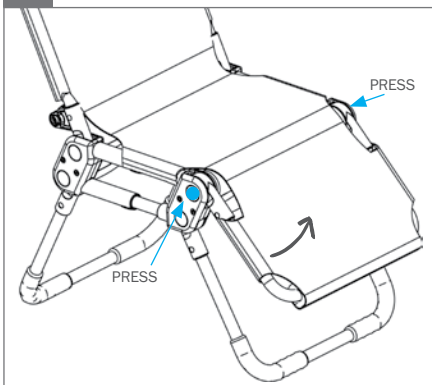
4.

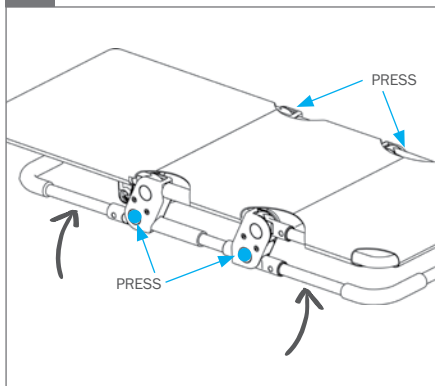5.

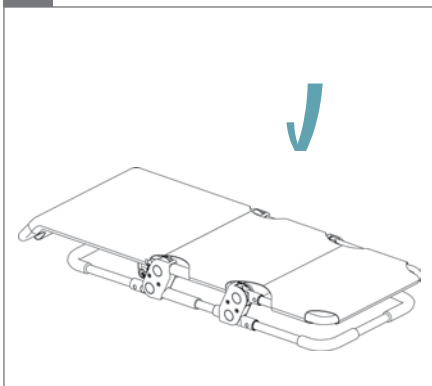
6.


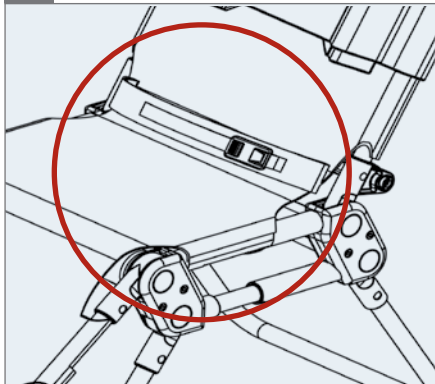
1.

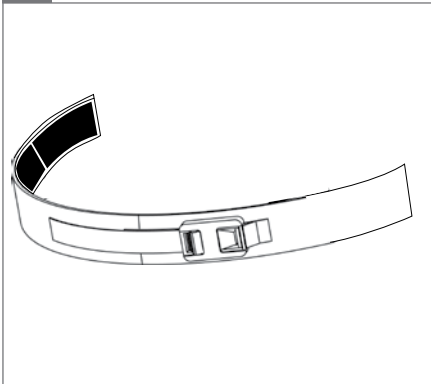
2.

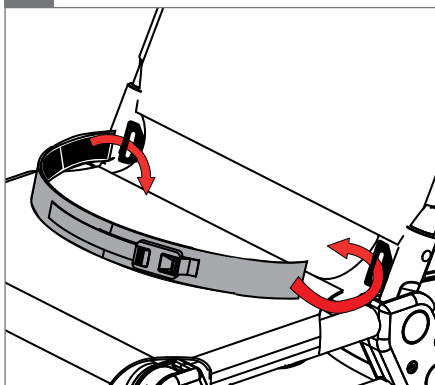3.

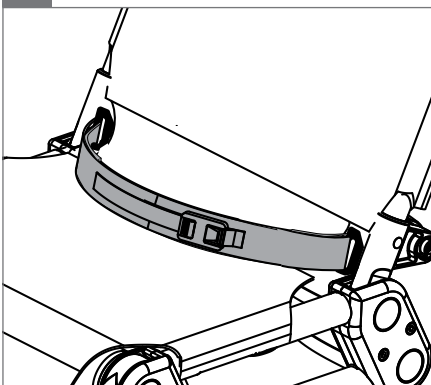
4.


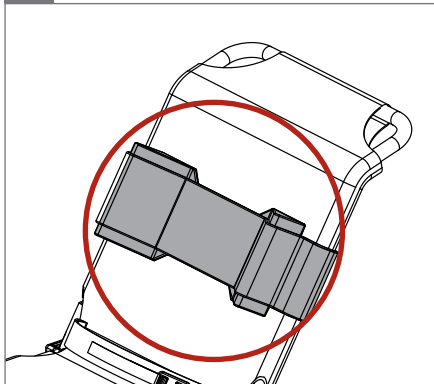
1.

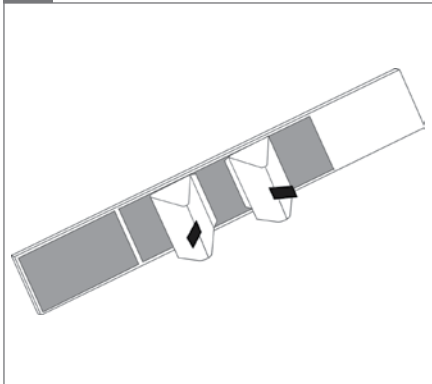
2.

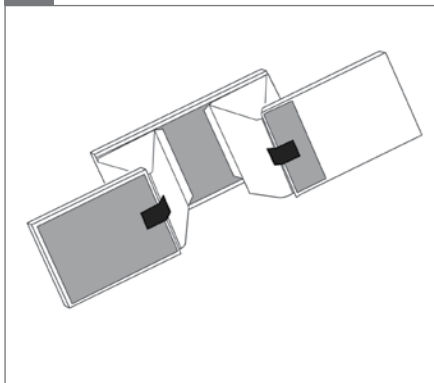
3.


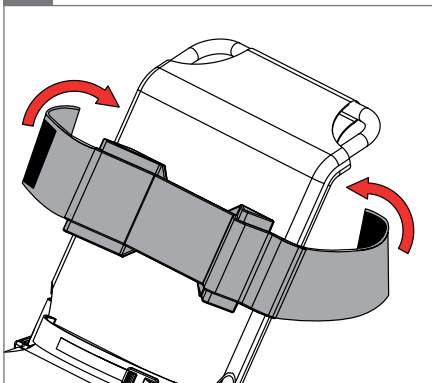
4.

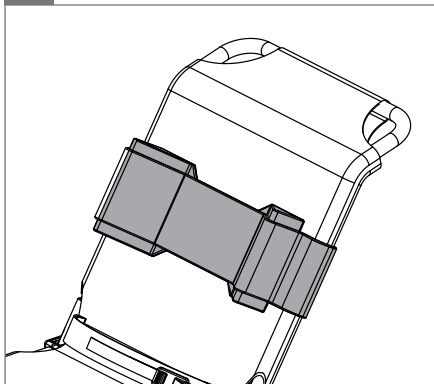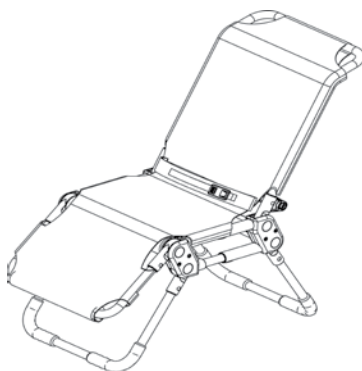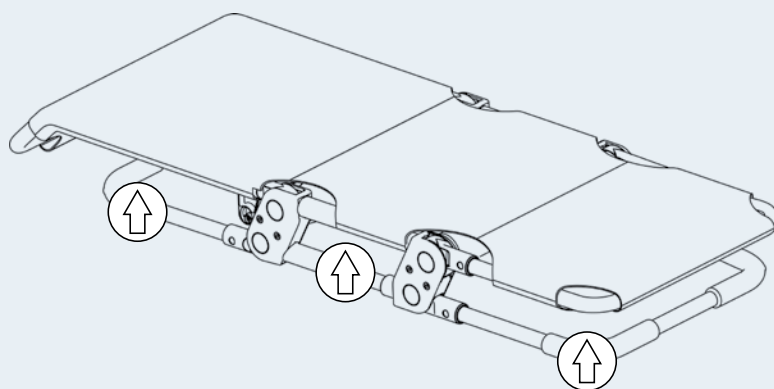
5.




Size 0

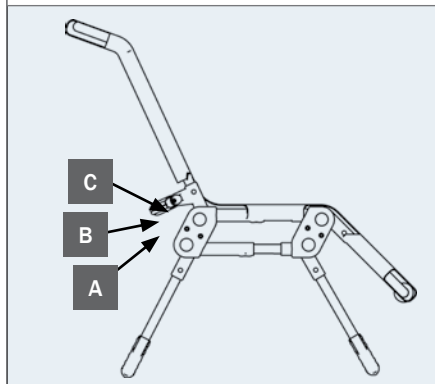Size 1 - 3

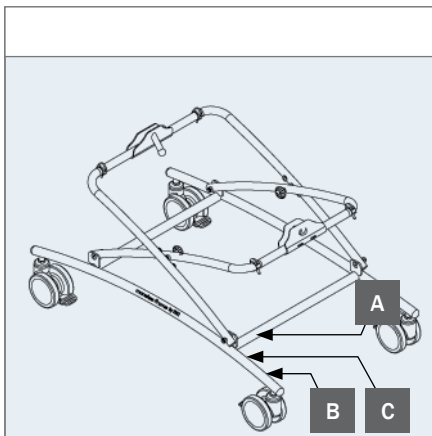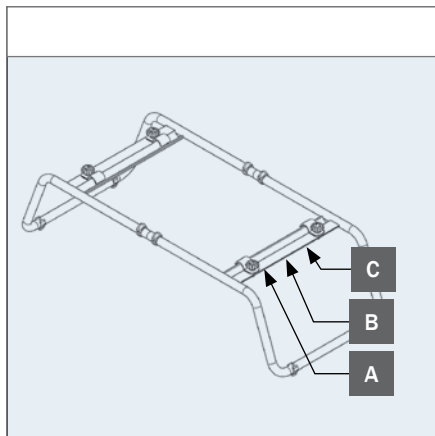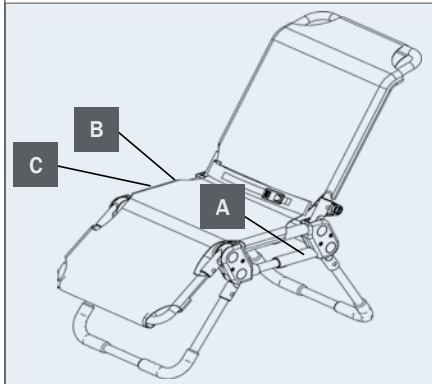A

B

C

**EN:** For indoor use**NO:** For innendørs bruk**FR:** Utilisation intérieur**RU:** Для использования в помещении**JP:** 屋内使用**DE:** Für den Innengebrauch**SV:** För inomhusbruk**IT:** Per uso interno**PL:** Do użytku wewnątrz**GR:** Κατάλληλο για εσωτερική χρήση**NL:** Voor binnenhuis gebruik**FIN:** Sisäkäyttöön**PT:** Para utilização em interiores**CZ:** Pro vnitřní používání**DK:** Til indendørs brug**ES:** Para uso en interiores**BR:** Para uso interno**SN:** 适用于室内使用**EN:** Consult instructions. The latest version of the instructions is always available on the R82 website and can be printed in larger sizes**NO:** Sjekk bruksanvisning på nett. Nyeste versjon av bruksanvisningen er alltid tilgjengelig, og kan skrives ut fra R82 nettsted**FR:** Se reporter aux instructions / consulter les instructions. La version la plus récente est toujours disponible et peut être imprimée dans un grand format à partir du site internet de R82**RU:** Прочитайте инструкции. Новейшая версия инструкций всегда доступна на веб-сайте компании R82 и может быть распечатана в увеличенном размере**JP:** 説明をお読みください。この説明の最新バージョンは、R82 の Web サイトからいつでもご利用いただけます。大きなサイズで印刷することができます**DE:** Bedienungsanleitung. Die neueste Version der User Guide ist auf der R82 Internetseite verfügbar und steht zum Download bzw. Ausdruck zur Verfügung**SV:** Se anvisningarna. Den senaste versionen av anvisningarna finns alltid tillgänglig och kan skrivas ut i större storlek från R82:s webbplats**IT:** Leggere le istruzioni. Le ultime versioni dei manuali di istruzione sono sempre disponibili sul sito web R82 e possono essere stampate in dimensioni maggiori**PL:** Zapoznanie się z instrukcją. Najnowsze wersje instrukcji obsługi są dostępne na stronie internetowej R82 i można je wydrukować w większych formatach**GR:** Διαβάστε τις οδηγίες. Οι τελευταίες εκδόσεις των οδηγιών είναι πάντοτε διαθέσιμες στην ιστοσελίδα της R82 και μπορούν να εκτυπωθούν σε μεγαλύτερα μεγέθη**NL:** Raadpleeg de instructies. De meest recente versie van alle instructies zijn altijd beschikbaar en kunnen worden afgedrukt in de groter formaat vanaf de R82 website**FIN:** Lisäohjeet. Viimeisin versio kaikista ohjeista on aina saatavilla ja tulostettavissa R82 nettisivustolta**PT:** Leia as instruções. As últimas versões das instruções estão sempre disponíveis na página da R82 na internet e podem ser impressas em tamanhos maiores**CZ:** Poradenství a instrukce. Nejaktuálnější verze všech instrukcí je vždy dostupná na webu R82, odkud se dá stáhnout a i vytisknout**DK:** Se vejledning. Den seneste version af denne vejledning er altid tilgængelig og kan udskrives i større størrelse fra R82 hjemmeside**ES:** Consulte las instrucciones. La última versión de todas las instrucciones está siempre disponible y se puede imprimir en tamaños superiores desde la página web de R82**BR:** Leia as instruções. As versões mais recentes das instruções sempre encontram-se disponíveis no site da R82 e podem ser impressas em tamanhos maiores**SN:** 参考说明。所有说明的最新版本都能够在R82网络上下载或打印**EN:** Correct lifting point**NO:** Riktig løftepunkt**FR:** Point de levée approprié**RU:** Надлежащая точка подъема**JP:** リフティング・ポイントの修正**DE:** Korrekter Hebe punkt**SV:** Korrekt lyftpunkt**IT:** Punto di sollevamento corretto**PL:** Prawidłowy punkt podnoszenia**GR:** Σωστό σημείο λαβής**NL:** Correcte techniek van optillen**FIN:** Oikea nostopiste**PT:** Corrija o ponto de elevação**CZ:** Správný zvedací bod**DK:** Korrekt løftepunkt**ES:** Punto de elevación correcto**BR:** Ponto de elevação correto**SN:** 请调整吊点

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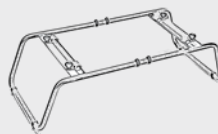
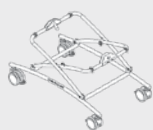
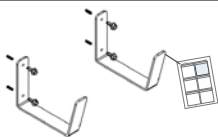
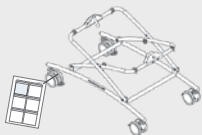
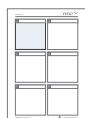
9996097456

ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The Manatee is a bathing chair suitable for disabled children. The chair is designed to place the user in exactly the right sitting position. The depth of the seat can be adjusted, allowing the chair to grow with the child. The product is for indoor use in a bathroom environment.

The target group for this device is disabled children with cerebral palsy (CP). Also, children with muscular dystrophy, spina bifida, spinal cord injuries and neurological impairments can be subject to the device. The product is suitable for children with GMFCS level 3-5.

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- EN 12182
- ISO 17966

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

Presales information

- support.r82.org

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Caregiver

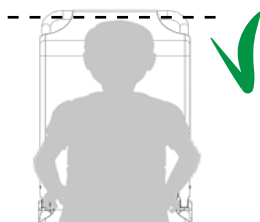
- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41°C then allow the product to cool before use

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered
- The head of a user must always be within the top of the frame

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Ensure the stability of the product is maintained before placing the user in it
- Please check on a regular basis that the zippers are safely closed.
- Check that all release buttons mesh into place after adjustment
- Ensure the Anti-slip rubber covers are functional
- Check on the cover, belts and accessories and make sure that the Velcro fastens securely
- The enclosed belts and accessories cannot be used as long-term supports but may only be used as an extra help when positioning the user.

Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

If the product is subject to tap water containing chlorine, then discoloration of plastics and fabric can be expected.

1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference

After every use

- Rinse the product with water to remove leftovers from soap and organic material

Daily

- Clean the anti-slip rubber covers, using a cloth with warm water and mild detergent and wipe them dry
- Visual inspection. Repair or replace any damaged or worn-out parts
- Check all fittings and buckles for signs of worn out parts
- Ensure that all fasteners are in place and fitted correctly
- Be aware of any MAX mark indications before adjustment

Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

Every 6 months

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Before disinfection

Remove the following items and clean them separately: Cover, Cushioning & Accessories

Washing

The cover can be washed at max. 50 °C using a mild detergent in a washing machine.

Washing machine

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

This product can be washed at 85° using a mild detergent for 3 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

Before washing

Remove the following items and clean them separately: Cover, Cushioning & Accessories

Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Backrest and leg support is adjustable and can be locked in position
- Zipper(s) & Velcro
- Chair is foldable/unfoldable
- Nuts and bolts on the product are tightened
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

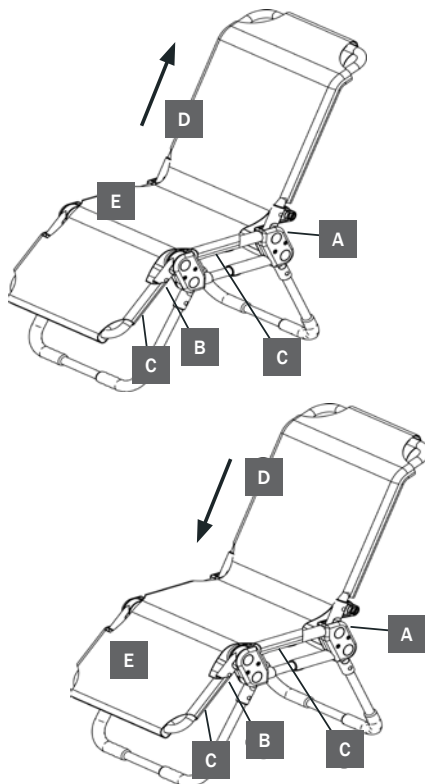
Dismounting and Mounting the Cover

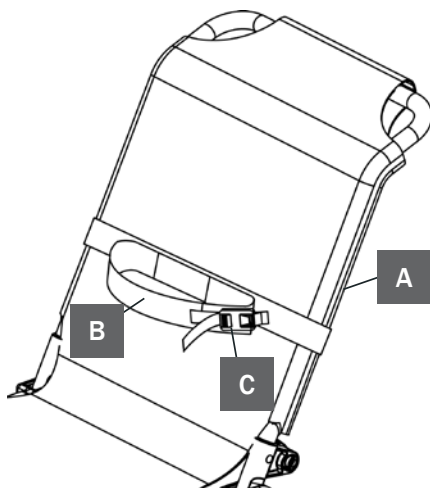
Follow the description below when the cover needs to be dismantled and washed:

1. Unzip the cover on the back (A) and underneath the seat (B)
2. Open the velcro underneath the chair (C) and the cover can be pulled up (D) and off the chair

Follow the description below to remount the cover on the chair:

1. Put the cover over the back and pull down (D). Make sure that the foam cushions in the cover are placed around the frame
2. Zip the cover on the back of the chair (A)
3. Mount the small cover piece with the zipper underneath the chair (B). Make sure that the velcro is pointing downwards
4. Tip up the leg support (E) and the velcro underneath the seat (C) and underneath the leg support (C) can be closed. Make sure that the foam cushions in the cover are placed around the frame
5. Tip down the leg support (E) and make sure the velcro is fastened correctly

Accessories

Body support

Please follow the description below to mount the body support the correct way:

1. Fasten the strap with velcro around the back (A)
2. Place the user in the chair. Mount the strap (B) around the body
3. Tighten the body support with velcro and the buckle (C)

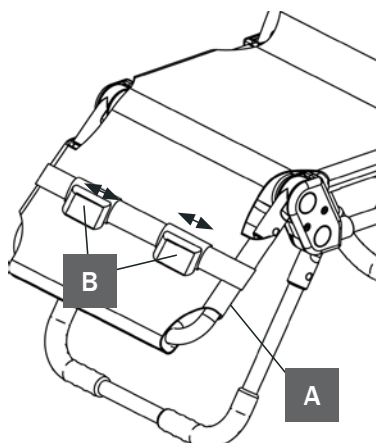
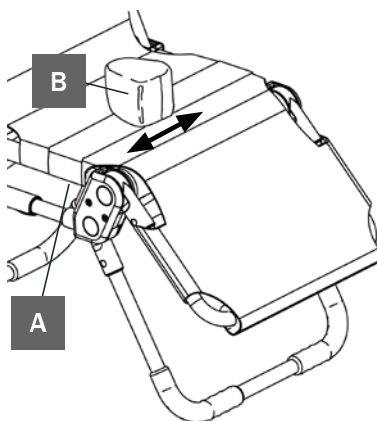
The body support may not be used as a long-term support. This support is only designed to make it easier to place the user in the chair.

Pommel with strap

Please follow the instructions below to mount the pommel and the strap the correct way:

1. Fasten the strap with velcro underneath the seat (A)
2. Adjust the pommel (B) sideways to gain the optimal position

The pommel with strap may not be used as a long-term support. This support is only designed to make it easier to place the user in the chair.

Straps for leg control

Please follow the description below to mount the straps the correct way:

1. Fasten the strap with velcro underneath the leg support (A)
2. Adjust the straps (B) sideways to gain the optimal position. Fasten with velcro

The straps may not be used as a long-term support. This support is only designed to make it easier to place the user in the chair.

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© 2010 R82 A/S All rights reserved The R82 logo and the icons are registered trademarks of R82 A/S
R82 A/S