Molift Raiser

molift[®]
by Etac

75135G 28-03-2017

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- Kan maskintvättas i 60-80° C temperatur.

Molift Raiser


Reconditioning instruction / Re-use
English

Up to serial number 16090201

Molift Raiser can be reconditioned.

This reconditioning instruction is meant for the technician in charge of the reconditioning.

molift[®]
by Etac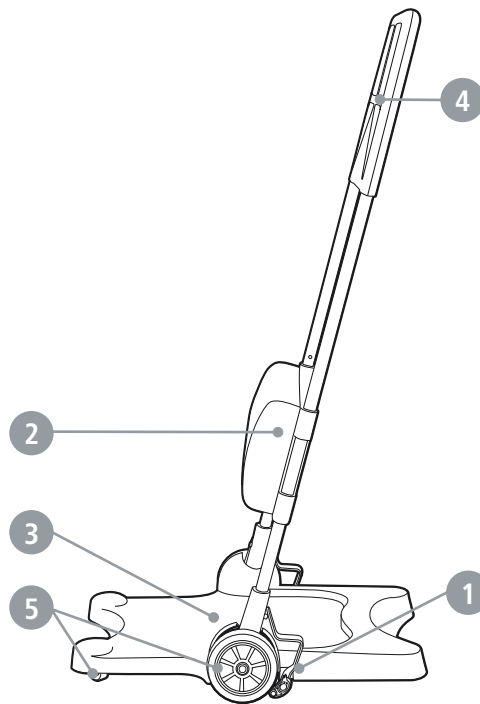

Washing

- Use a solvent-free detergent with a pH level between 5 and 9.
- Can be washed at 70° C for 10 minutes. (not with leg support).
- Use a wet wipe for the leg support.
- Do not rinse directly on bearings and leg support with a pressure washer.
- Lay the Raiser on the floor in an angled position after washing, so the water runs out.
- You may also use compressed air to blow dry.

1 Check brake functionality

- Parking brake
- The wheels must lock in brake position.
- The parking brake must stay in locked position.
- Lubricate the brake mechanism.

2 Leg Support

- Check the height controls and that the snap function can lock in all positions
- Check that the pad does not have any damage.

3 Base plate

- Check that the base plate is secured and that it does not have any damage.

4 Handle

- Ensure that all ten screw mounts have tightened screws.
- Ensure that the handle is not damaged and is properly secured.

5 Castors / Wheels / Wheel axles

- Check that the tyres, bearings and wheel axles are not worn, and that the wheels turn easily.

6 Textile accessories

- Can be machine washed at 60-80 ° C temperature.

Molift Raiser


Reconditioning instruction / Re-use
English

From serial number 16090202

Molift Raiser can be reconditioned.

This reconditioning instruction is meant for the technician in charge of the reconditioning.

molift[®]
by Etac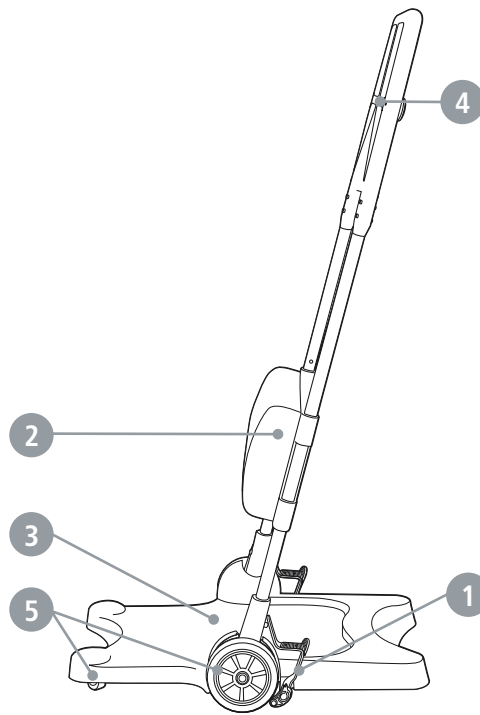

Washing

- Use a solvent-free detergent with a pH level between 5 and 9.
- Can be washed at 70° C for 10 minutes. (not with leg support).
- Use a wet wipe for the leg support.
- Do not rinse directly on bearings and leg support with a pressure washer.
- Lay the Raiser on the floor in an angled position after washing, so the water runs out.
- You may also use compressed air to blow dry.

1 Check brake functionality

- Parking brake
- The wheels must lock in brake position.
- The parking brake must stay in locked position.
- Lubricate the brake mechanism.

2 Leg Support

- Check the height controls and that the snap function can lock in all positions
- Check that the pad does not have any damage.

3 Base plate

- Check that the base plate is secured and that it does not have any damage.

4 Handle

- Ensure that all four mounts have tightened screws.
- Ensure that the attachments for the Raiser Strap have tightened screws and fit firmly.
- Ensure that the handle is not damaged and is properly secured.

5 Castors / Wheels / Wheel axles

- Check that the tyres, bearings and wheel axles are not worn, and that the wheels turn easily.

6 Textile accessories

- Can be machine washed at 60-80 ° C temperature.