

BINGO Evolution Spare parts catalogue



# HOGGÏ

Medema-group Ver. 2021-02-04

en

#### Index

| 1 | Modelidentifikation and basic equipment                    |          |  |  |  |  |  |
|---|--|----------|--|--|--|--|--|
|   | 1.1 Modelidentifikation                                    | 4        |  |  |  |  |  |
|   | 1.2 Basic equipment  | 4        |  |  |  |  |  |
|   |  |          |  |  |  |  |  |
| 2 | Spare parts  |          |  |  |  |  |  |
|   | 2.1 Mobility base, with swivelling front wheels            | 5        |  |  |  |  |  |
|   | 2.2 Mobility base, with rigid front wheels                 |          |  |  |  |  |  |
|   | 2.3 Mobility base for additional equipment                 | 7        |  |  |  |  |  |
|   | 2.4 Mobility base for trike                                | 8        |  |  |  |  |  |
|   | 2.5 Mobility base Mini (invert),                           |          |  |  |  |  |  |
|   | 2.6 Seat unit  | 10       |  |  |  |  |  |
|   | 2.7 Seat unit Mini   | 11       |  |  |  |  |  |
|   | 2.8 Frame, for swivel front wheels                         | 12       |  |  |  |  |  |
|   | 2.9 Frame, for rigid front wheels                          | 13       |  |  |  |  |  |
|   | 2.10 Frame, for additional equipment                       | 14       |  |  |  |  |  |
|   | 2.11 Frame, for trike                                      | 15       |  |  |  |  |  |
|   | 2.12 Frame, for Mini with swivel front wheels              | 16<br>17 |  |  |  |  |  |
|   | 2.13 Front bar   | 1/       |  |  |  |  |  |
|   | 2.14 Frame suspension system                               | 18<br>19 |  |  |  |  |  |
|   | 2.15 Spring selection                                      | 19       |  |  |  |  |  |
|   | 2.16 Rear frame  | 20<br>21 |  |  |  |  |  |
|   | 2.17 Break   | 21       |  |  |  |  |  |
|   | 2.18 Push bar (parte list)                                 | 22       |  |  |  |  |  |
|   | 2.19 Push bar (parts list)                                 | 2/       |  |  |  |  |  |
|   | 2.21 Seat Bearing, Mini                                    |          |  |  |  |  |  |
|   | 2.21 Seat Bearing, Willing                                 | 25<br>26 |  |  |  |  |  |
|   | 2.22 Seat Bearing, size                                    | 27       |  |  |  |  |  |
|   | 2.24 Tie down kit Mini                                     | 28       |  |  |  |  |  |
|   | 2.25 Castor fork   |          |  |  |  |  |  |
|   | 2.26 Wheel options   |          |  |  |  |  |  |
|   | 2.27 Trike front wheel                                     | 31       |  |  |  |  |  |
|   | 2.28 Drum break's  | 32       |  |  |  |  |  |
|   | 2.29 Seat base   |          |  |  |  |  |  |
|   | 2.30 Releaser, tiltingmechanism                            |          |  |  |  |  |  |
|   | 2.31 Releaser, tiltingmechanism Mini                       |          |  |  |  |  |  |
|   | 2.32 Seat surface  | 36       |  |  |  |  |  |
|   | 2.33 Seat surface Mini                                     | 37       |  |  |  |  |  |
|   | 2.34 Hip support   | 38       |  |  |  |  |  |
|   | 2.35 Pelotte thigh   |          |  |  |  |  |  |
|   | 2.36 Footresthanger 90°                                    | 4.0      |  |  |  |  |  |
|   | 2.37 Foot rest hanger angle adjustable 90-160°             | 41       |  |  |  |  |  |
|   | 2.38 Foot rest hanger angle adjustable 90-160°(parts list) | 42       |  |  |  |  |  |
|   | 2.39 Foot rest hanger Mini angle adjustable                | 43       |  |  |  |  |  |
|   | 2.40 Foot rest hanger, angle adjust. 85-175° (new version) | 44       |  |  |  |  |  |
|   | 2.41 Footresthanger 90° (new version)                      | 45       |  |  |  |  |  |
|   | 2.42 Foot board assembly angle adjustable and foldable     | 46       |  |  |  |  |  |
|   | 2.43 Footrest flip up lock                                 | 47       |  |  |  |  |  |
|   | 2.44 Foot plate  | 48       |  |  |  |  |  |
|   | 2.45 Back base   | 49       |  |  |  |  |  |
|   | 2.46 Back hase OT  | 50       |  |  |  |  |  |


#### Index

#### 2 Spare parts 2.47 OT Back base plus \_\_\_\_\_\_ 51 2.48 Lateral support522.49 Lateral support aluminum53 2.50 Back insert 54 2.51 Backinsert OT 55 2.52 Back base Mini \_\_\_\_\_\_ 56 2.53 Back Mini\_\_\_\_\_\_ 57 2.54 Upholstery\_\_\_\_\_\_ 58 2.55 Trunk support592.56 Thoraxpelotte Mini60 2.57 Abduction block swing&detachable \_\_\_\_\_\_\_61 2.60 Retainer for accessories grab rail or therapy tray Mini \_\_\_\_\_\_ 64 2.61 Grab rail \_\_\_\_\_\_ \_\_\_\_\_ 65 2.62 Therapy tray,662.63 Therapy tray, height adjustable67 2.62 Therapy tray, 2.64 Swivel lock \_\_\_\_\_\_\_ 68 2.65 Attendant brake\_\_\_\_\_\_\_\_69 2.66 Canopy \_\_\_\_\_\_\_ 70 2.67 Headrest 71

The lable with the serial number for the mobility base, is placed on the left side of the side bar.

The lable with the serial number for the seat unit, is placed in the middle at the bottom of the seat base.


Please give us the Serial number with every spare part order, to ensure a correct delivery.

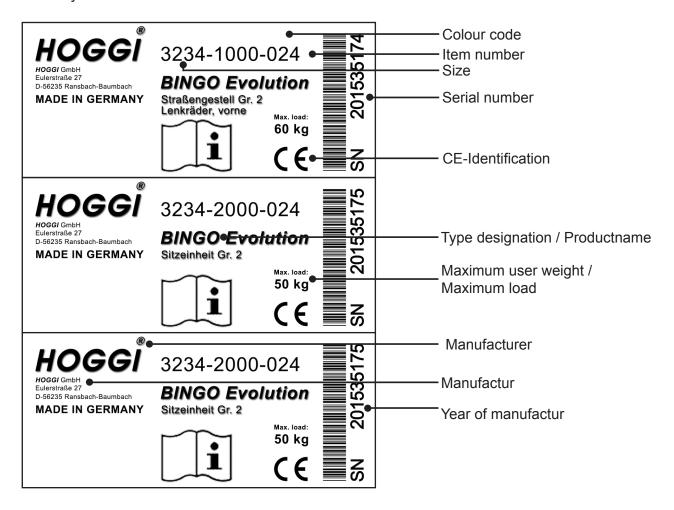

#### 1.2 Basic configuration


- Bingo EVO Reha-Buggy (Mobility base and seat unit)
- Front- and rear wheels with Pneumatic- oder PU-Wheels
- Upholstery (IPS)
- Pelotte thigh support
- Foot rest hanger, angle adjustable
- Foot rest bracket and foot rest

## Mobility base swivelling front wheels wheels

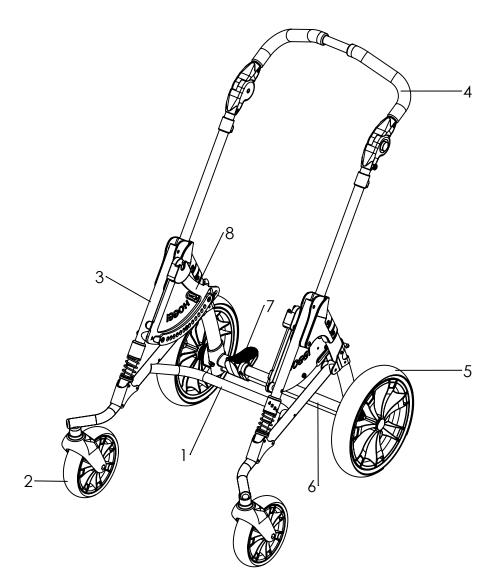


| Pos | Medema<br>number | Article number | Articel description   | Page |
|-----|------------------|----------------|---|------|
| 1-8 | 1802-1201        | 3232-1000-xxx  | BINGO-Evolution mobility base size 1, swivelling front wheels |      |
|     | 1802-1199        | 3234-1000-xxx  | BINGO-Evolution mobility base size 2, swivelling front wheels |      |
| 1   | 1802-1215        | 3232-1100E-xxx | Frame size 1 (swivelling front wheels)                        | 12   |
|     | 1802-1217        | 3234-1100E-xxx | Frame size 2 (swivelling front wheels)                        |      |
| 2   | Side 30          | 12xx-          | Front wheel   | 30   |
| 3   | 1802-1241        | 3231-1300E-xxx | Front bar   | 17   |
|     | 1802-1242        | 3231-7400E-xxx | Frame suspension system                                       | 18   |
| 4   | 1806-1298        | 3231-1400E-xxx | Push handle size 1  | 22   |
|     | 1806-1299        | 3233-1400E-xxx | Push handle size 2  |      |
| 5   | Side 30          | 12xx-          | Rear wheel  | 30   |
| 6   | 1802-1220        | 3231-1200E-xxx | Rear frame size 1   | 20   |
|     | 1802-1221        | 3233-1200E-xxx | Rear frame size 2   |      |
| 7   | 1803-1270        | 3201-1301E-xxx | Brake size 1  | 21   |
|     | 1803-1271        | 3203-1301E-xxx | Brake size 2  |      |
| 8   | 1831-1187        | 3231-1510E-xxx | Bearing (seat), left  | 24   |
|     | 1831-1188        | 3231-1520E-xxx | Bearing (seat), right   |      |

# Mobility base for rigid front wheels

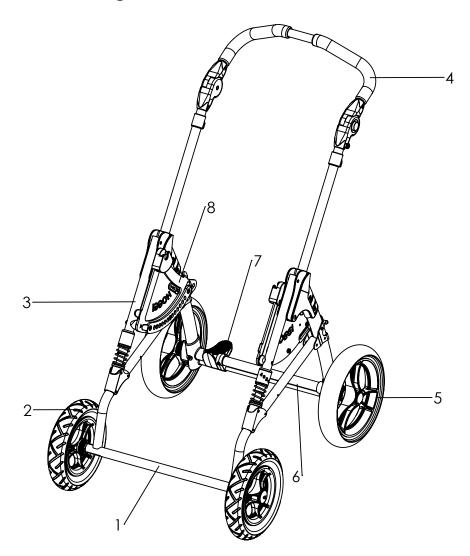


| Pos | Medema<br>number | Article number | Articel description                                     | Page |
|-----|------------------|----------------|---|------|
| 1-8 | 1802-1249        | 3231-1011-032  | BINGO Evo mobility base, with rigid front wheels Mini   |      |
|     | 1802-1264        | 3231-1000-xxx  | BINGO Evo mobility base, with rigid front wheels size 1 |      |
|     | 1802-1265        | 3233-1000-xxx  | BINGO Evo mobility base, with rigid front wheels size 2 |      |
| 1   | 1802-1214        | 3231-1100E-xxx | Frame size 1, rigid front wheels                        | 13   |
|     | 1802-1216        | 3233-1100E-xxx | Frame size 2, rigid front wheels                        |      |
| 2   | Side 30          | 12xx-          | Front wheel   | 30   |
| 3   | 1802-1241        | 3231-1300E-xxx | Front bar   | 17   |
|     | 1802-1242        | 3231-7400E-xxx | Frame suspension system                                 | 18   |
| 4   | 1806-1298        | 3231-1400E-xxx | Push handle size 1                                      | 22   |
|     | 1806-1299        | 3233-1400E-xxx | Push handle size 2                                      |      |
| 5   | Side 30          | 12xx-          | Rear wheel  | 30   |
| 6   | 1802-1220        | 3231-1200E-xxx | Rear frame size 1                                       | 20   |
|     | 1802-1221        | 3233-1200E-xxx | Rear frame size 2                                       |      |
| 7   | 1803-1270        | 3201-1301E-xxx | Brake size 1  | 21   |
|     | 1803-1271        | 3203-1301E-xxx | Brake size 2  |      |
| 8   | 1831-1187        | 3231-1510E-xxx | Bearing (seat), left                                    | 24   |
|     | 1831-1188        | 3231-1520E-xxx | Bearing (seat), right                                   |      |
|     | 1831-1189        | 3230-1510E     | Bearing (seat), Mini left                               | 26   |
|     | 1831-1190        | 3230-1520E     | Bearing (seat), Mini right                              |      |

## Mobility base for additional equipment

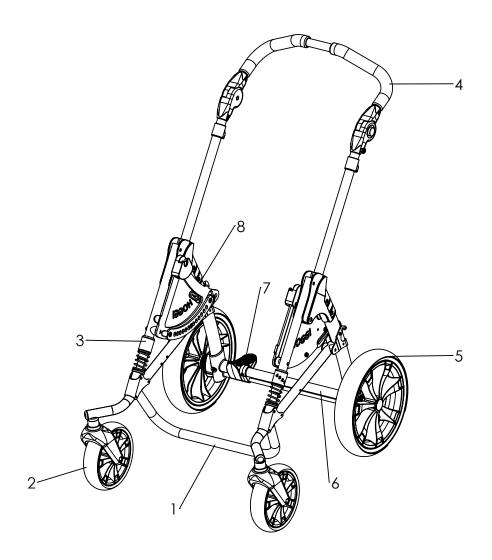


| Pos | Medema<br>number | Article number | Articel description                                     | Page |
|-----|------------------|----------------|---|------|
| 1-8 | 1802-1250        | 3238-1011-032  | BINGO Evo mobility base for additional equipment Mini   |      |
|     | 1802-1266        | 3238-1000-xxx  | BINGO Evo mobility base for additional equipment size 1 |      |
|     | 1802-1267        | 3239-1000-xxx  | BINGO Evo mobility base for additional equipment size 2 |      |
| 1   | 1802-1243        | 3238-1100E-xxx | Frame size 1 ( for additional equipmen)                 | 14   |
|     | 1802-1244        | 3239-1100E-xxx | Frame size 2 ( for additional equipmen)                 |      |
| 2   | Side 30          | 12xx-          | Front wheel   | 30   |
| 3   | 1802-1241        | 3231-1300E-xxx | Front bar   | 17   |
|     | 1802-1242        | 3231-7400E-xxx | Frame suspension system                                 | 18   |
| 4   | 1806-1298        | 3231-1400E-xxx | Push handle size 1                                      | 22   |
|     | 1806-1299        | 3233-1400E-xxx | Push handle size 2                                      |      |
| 5   | Side 30          | 12xx-          | Rear wheel  | 30   |
| 6   | 1802-1220        | 3231-1200E-xxx | Rear frame size 1                                       | 20   |
|     | 1802-1221        | 3233-1200E-xxx | Rear frame size 2                                       |      |
| 7   | 1803-1270        | 3201-1301E-xxx | Brake size 1  | 21   |
|     | 1803-1271        | 3203-1301E-xxx | Brake size 2  |      |
| 8   | 1831-1187        | 3231-1510E-xxx | Bearing (seat), left                                    | 24   |
|     | 1831-1188        | 3231-1520E-xxx | Bearing (seat), right                                   |      |
|     | 1831-1189        | 3230-1510E-xxx | Bearing (seat), Mini left                               | 26   |
|     | 1831-1190        | 3230-1520E-xxx | Bearing (seat), Mini right                              |      |

## Mobility base trike

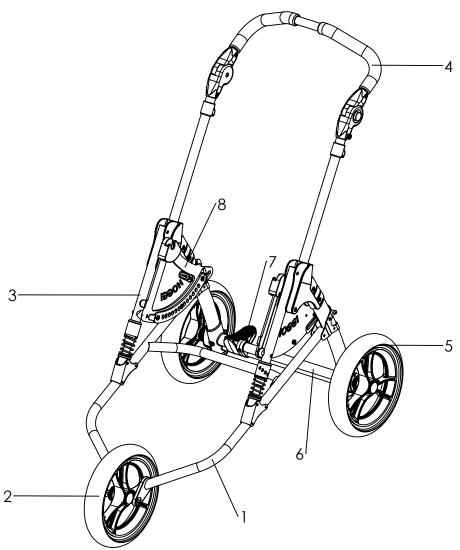


| Pos | Medema<br>number | Article number | Articel description                         | Page |
|-----|------------------|----------------|---|------|
| 1-8 | 1802-1270        | 3236-1011-032  | BINGO-Evolution mobility base Mini, trike   |      |
|     | 1802-1268        | 3236-1000-xxx  | BINGO-Evolution mobility base size 1, trike |      |
|     | 1802-1269        | 3237-1000-xxx  | BINGO-Evolution mobility base size 2, trike |      |
| 1-3 | 1802-1218        | 3236-1100E-xxx | Frame size 1 (swivelling front wheels)      | 15   |
|     | 1802-1219        | 3237-1100E-xxx | Frame size 2 (swivelling front wheels)      |      |
| 2   | Side 31          | 12xx-          | Front wheel                                 | 31   |
| 3   | 1802-1241        | 3231-1300E-xxx | Front bar                                   | 17   |
|     | 1802-1242        | 3231-7400E-xxx | Frame suspension system                     | 18   |
| 4   | 1806-1298        | 3231-1400E-xxx | Push handle size 1                          | 22   |
|     | 1806-1299        | 3233-1400E-xxx | Push handle size 2                          |      |
| 5   | Side 31          | 12xx-          | Rear wheel                                  | 30   |
| 6   | 1802-1220        | 3231-1200E-xxx | Rear frame size 1                           | 20   |
|     | 1802-1221        | 3233-1200E-xxx | Rear frame size 2                           |      |
| 7   | 1803-1270        | 3201-1301E-xxx | Brake size 1                                | 21   |
|     | 1803-1271        | 3203-1301E-xxx | Brake size 2                                |      |
| 8   | 1831-1187        | 3231-1510E-xxx | Bearing (seat), left                        | 24   |
|     | 1831-1188        | 3231-1520E-xxx | Bearing (seat), right                       |      |
|     | 1831-1189        | 3230-1510E-xxx | Bearing (seat), Mini left                   | 26   |
|     | 1831-1190        | 3230-1520E-xxx | Bearing (seat), Mini right                  |      |


## Mobility base Mini (invert)

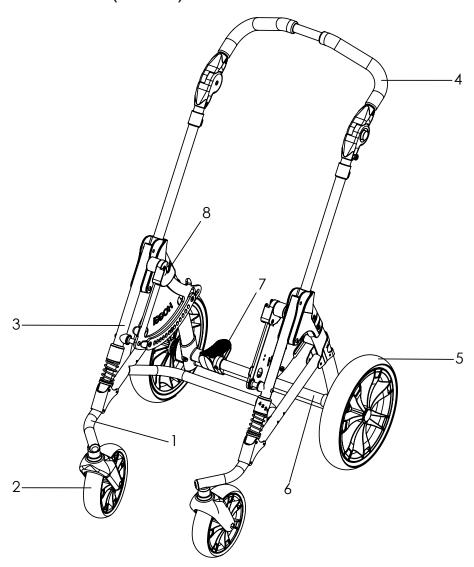


| Pos | Medema<br>number | Article number | Articel description                         | Page |
|-----|------------------|----------------|---|------|
| 1   | 1802-1257        | 3230-1100E-032 | Frame Mini invert (swivelling front wheels) |      |
| 2   | side 30          | 12xx-          | Front wheel                                 | 16   |
| 3   | 1802-1273        | 3231-7400-032  | Front bar                                   | 30   |
|     | 1802-1242        | 3231-7400E-032 | Frame suspension system                     | 17   |
| 4   | 1806-1298        | 3231-1400-032  | Push handle size 1                          | 18   |
| 5   | Side 30          | 12xx-          | Rear wheel                                  | 22   |
| 6   | Side 20          | 3230-1200-032  | Rear frame size 1                           | 30   |
| 7   | Side 21          | 3201-1301-032  | Rear frame size 1                           | 20   |
| 8   | 1831-1189        | 3230-1510-xxx  | Bearing (seat), Mini left                   | 21   |
|     | 1831-1190        | 3230-1520-xxx  | Bearing (seat), Mini right                  | 26   |

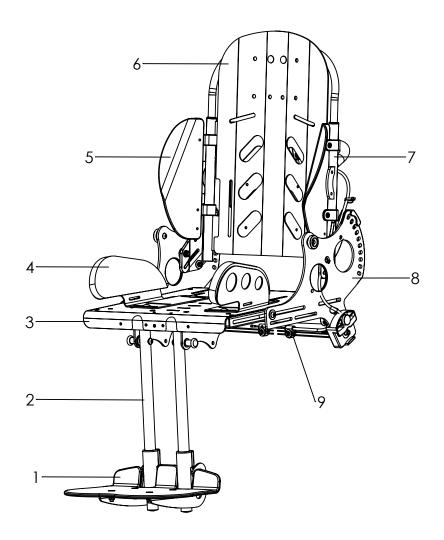

| Pos | Medema<br>number | Article number | Articel description | Page |
|-----|------------------|----------------|---------------------|------|
| 1-9 | 1807-1303        | 3232-2000-xxx  | Seat unit size 1    |      |
|     | 1807-1304        | 3234-2000-xxx  | Seat unit size 2    |      |
|     | 1807-1324        | 3235-2000-xxx  | Seat unit size 2XL  |      |
| 1   | Side 44          | 323X-2XXXE     | Foot board          | 44   |
| 2   | Side 40          | 323X-2X00E-xxx | Footrest hanger     | 40   |
| 3   | Side 36          | 323X-2200-xxx  | Seat surface        | 36   |
| 4   | Side 39          | 323X-2900E-xxx | Thigh support       | 39   |
|     | Side 38          | 323X-2600E-xxx | Hip support         | 38   |
| 5   | Side 48          | 323X-27XX      | Lateral support     | 48   |
| 6   | Side49           | 323X-2800E-xxx | Back insert         | 49   |
| 7   | Side 47          | 323X-2300E-xxx | Back base           | 47   |
| 8   | Side 33          | 323X-2100E-xxx | Seat base           | 33   |
| 9   | 1802-1213        | 3231-1900E     | Tilt mechanism      | 34   |

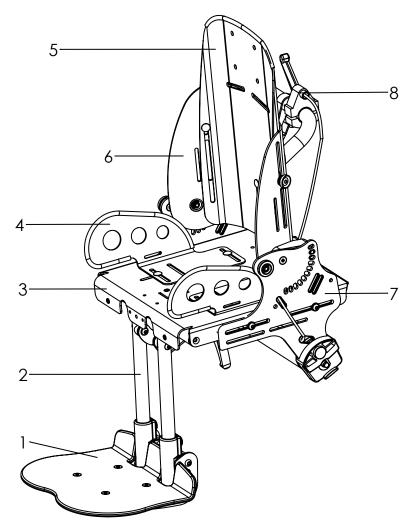


| Pos | Medema<br>number | Article number | Articel description | Page |
|-----|------------------|----------------|---------------------|------|
| 1-6 | 1807-1325        | 3230-2000-032  | Seat unit Mini      |      |
| 1   | Side 44          | 323X-2XXXE     | Foot board          | 44   |
| 2   | Side 43          | 3230-2400-xxx  | Footrest hanger     | 43   |
| 3   | Side 37          | 3230-2200E-032 | Seat surface        | 37   |
| 4   | Side 39          | 3230-290X-032  | Thigh support       | 39   |
|     | Side 38          | 3230-2600E-032 | Hip support         | 38   |
| 5   | Side 51          | 3230-2302-xxx  | Back insert         | 51   |
| 6   | Side 50          | 3230-2301-xxx  | Back base           | 50   |
| 7   | Side 33          | 3230-2100-xxx  | Seat base           | 33   |
| 8   | Side 35          | 3230-1900      | Tilt mechanism      | 35   |

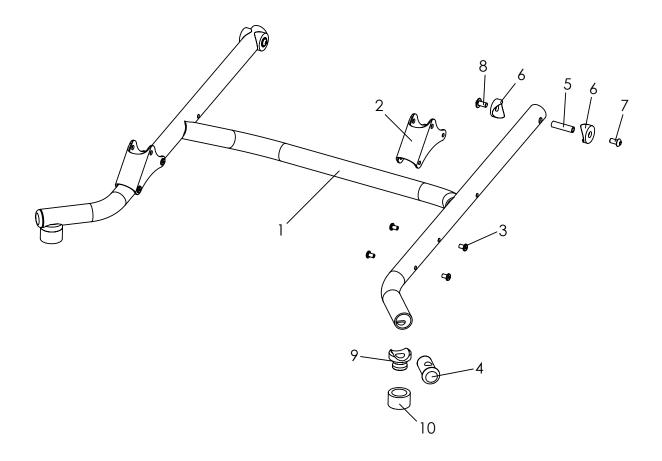


| Pos  | Medema<br>number | Article number | Articel description  | Quantity | Unit |
|------|------------------|----------------|--|----------|------|
| 1-10 | 1802-1215        | 3232-1100E-xxx | Frame size 1 (swivel front wheels)                                     | 1        | Pc.  |
|      | 1802-1217        | 3234-1100E-xxx | Frame size 2 (swivel front wheels)                                     | 1        | Pc.  |
| 1-4  | 1802-1245        | 3232-1110E     | Frame size 1   | 1        | Pc.  |
|      | 1802-1246        | 3234-1110E     | Frame size 2   | 1        | Pc.  |
| 2    | Included         | 3231-1120      | Front strut bearing  | 2        | Pc.  |
| 3    | Included         | 1069-0004      | Blind rivet F5x10 black  | 8        | Pc.  |
| 4    | Included         | 3202-1130      | Block  | 2        | Pc.  |
| 5    | Included         | 1114-0027      | Threaded bushing 8xM6x32   | 2        | Pc.  |
| 6    | Included         | 1111-0012      | Saddle washer PA black 24x8.4x3  | 4        | Pc.  |
| 7*   | Included         | 1029-6001      | Button head socket screw, Tufflock M6x12, with frame colour, 024       | 2        | Pc.  |
|      | Included         | 1029-6001-033  | Button head socket screw black, Tufflock M6x12, with frame colour, 032 | 2        | Pc.  |
| 8    | Included         | 1030-0612      | Button head socket screw with flange M6x12                             | 2        | Pc.  |
| 9    | Included         | 3202-1140      | Saddle bushing for front caster  | 2        | Pc.  |
| 10   | Included         | 3202-1151      | Cover ring   | 2        | Pc.  |

<sup>\*</sup> Screw`s change with frame colour!

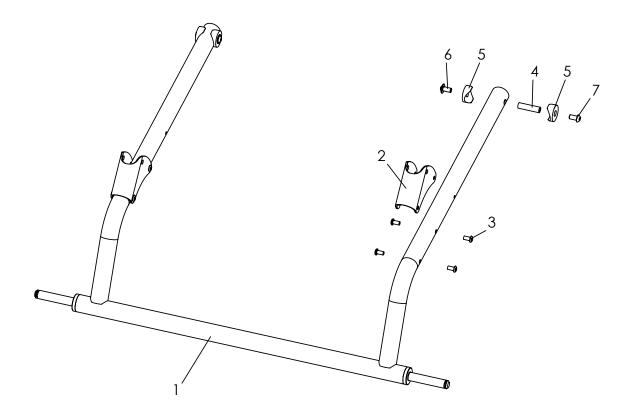


| Pos  | Medema<br>number | Article number | Articel description   | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-10 | 1802-1214        | 3231-1100E-xxx | Frame size 1 (rigid front wheels)                                   | 1        | Pc.  |
|      | 1802-1216        | 3233-1100E-xxx | Frame size 2 (rigid front wheels)                                   | 1        | Pc.  |
| 1-3  | 1802-1247        | 3231-1110E-xxx | Frame size 1 (rigid front wheels)                                   | 1        | Pc.  |
|      | 1802-1248        | 3233-1110E-xxx | Frame size 2 (rigid front wheels)                                   | 1        | Pc.  |
| 2    | Included         | 3231-1120-xxx  | Front strut bearing   | 2        | Pc.  |
| 3    | Included         | 1069-0004      | Blind rivet F5x10 black   | 8        | Pc.  |
| 4    | Included         | 1114-0027      | Threaded bushing 8xM6x32  | 2        | Pc.  |
| 5    | Included         | 1111-0012      | Saddle washer PA black 24x8.4x3                                     | 4        | Pc.  |
| 6    | Included         | 1030-0612      | Button head socket screw with flange M6x12                          | 2        | Pc.  |
| 7*   | Included         | 1029-6001      | Button head socket screw, Tufflock M6x12, with frame colour, 024    | 2        | Pc.  |
|      | Included         | 1029-6001-033  | Buttonheadsocketscrew black, Tufflock M6x12, with frame colour, 032 | 2        | Pc.  |

<sup>\*</sup> Screw`s change with frame colour!

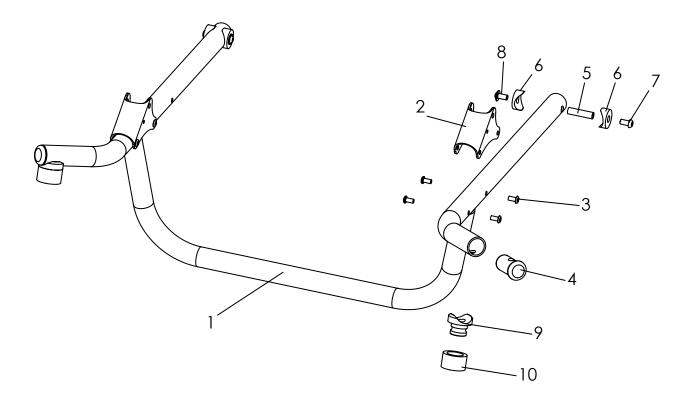


| Pos  | Medema<br>number | Article number | Articel description   | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-10 | 1802-1251        | 3238-1100E-xxx | Front frame for additional equipment size 1, complete               | 1        | Pc.  |
|      | 1802-1252        | 3239-1100E-xxx | Front frame for additional equipment size 2, complete               | 1        | Pc.  |
| 1-4  | 1802-1253        | 3238-1110E-xxx | Front frame for additional equipment size 1                         | 1        | Pc.  |
|      | 1802-1254        | 3239-1110E-xxx | Front frame for additional equipment size 2                         | 1        | Pc.  |
| 2    | Included         | 3231-1120      | Front strut bearing   | 2        | Pc.  |
| 3    | Included         | 1069-0004      | Blind rivet F5x10 black   | 8        | Pc.  |
|      | Included         | 3202-1130      | Block   | 2        | Pc.  |
| 5    | Included         | 1114-0027      | Threaded bushing 8xM6x32  | 2        | Pc.  |
| 6    | Included         | 1111-0012      | Saddle washer PA black 24x8.4x3                                     | 4        | Pc.  |
| 7*   | Included         | 1029-6001      | Button head socket screw, Tufflock M6x12, with frame colour, 024    | 2        | Pc.  |
|      | Included         | 1029-6001-033  | Buttonheadsocketscrew black, Tufflock M6x12, with frame colour, 032 | 2        | Pc.  |
| 8    | Included         | 1030-0612      | Button head socket screw with flange M6x12                          | 2        | Pc.  |
| 9    | Included         | 3202-1140      | Saddle bushing for front caster                                     | 2        | Pc.  |
| 10   | Included         | 3202-1151      | Cover ring  | 2        | Pc.  |

<sup>\*</sup> Screw`s change with frame colour!

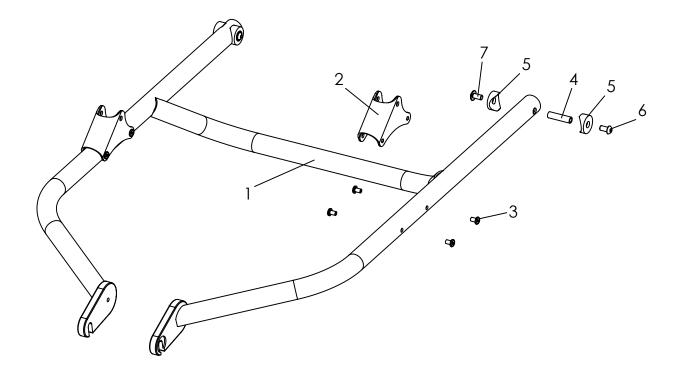


| Pos | Medema<br>number | Article number | Articel description   | Quantity | Unit |
|-----|------------------|----------------|---|----------|------|
| 1-7 | 1802-1218        | 3236-1100E-xxx | Trike-Frame size 1, complete  | 1        | St   |
|     | 1802-1219        | 3237-1100E-xxx | Trike-Frame size 2, complete  | 1        | St   |
| 1-3 | 1802-1255        | 3236-1110E-xxx | Trike-Frame size 1  | 1        | St   |
|     | 1802-1256        | 3237-1110E-xxx | Trike-Frame size 2  | 1        | St   |
| 2   | Included         | 3231-1120      | Front strut bearing   | 2        | St   |
| 3   | Included         | 1069-0004      | Blind rivet F5x10 black   | 8        | St   |
| 4   | Included         | 1114-0027      | Threaded bushing 8xM6x32  | 2        | St   |
| 5   | Included         | 1111-0012      | Saddle washer PA black 24x8.4x3                                     | 4        | St   |
| 6*  | Included         | 1029-6001      | Button head socket screw, Tufflock M6x12, with frame                | 2        | St   |
|     | Included         | 1029-6001-033  | Buttonheadsocketscrew black, Tufflock M6x12, with frame colour, 032 | 2        | St   |
| 7   | Included         | 1030-0612      | Button head socket screw with flange M6x12                          | 2        | St   |

<sup>\*</sup> Screw`s change with frame colour!

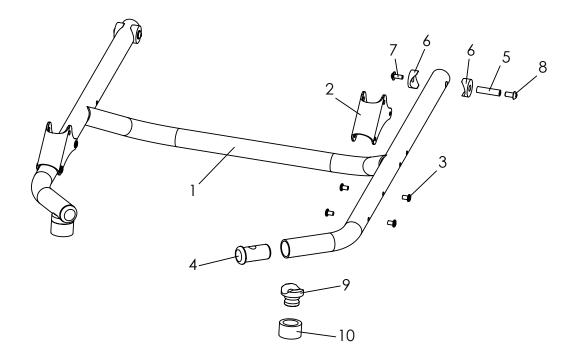


| Pos  | Medema<br>number | Article number | Articel description   | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-10 | 1802-1257        | 3230-1100E-032 | Frame Mini (swivel front wheels)                                    | 1        | Pc.  |
| 1-4  | 1802-1258        | 3230-1110E-032 | Frame Mini (swivel front wheels)                                    | 1        | Pc.  |
| 2    | Included         | 3231-1120      | Front strut bearing   | 2        | Pc.  |
| 3    | Included         | 1069-0004      | Blind rivet F5x10 black   | 8        | Pc.  |
| 4    | Included         | 3202-1130      | Block   | 2        | Pc.  |
| 5    | Included         | 1114-0027      | Threaded bushing 8xM6x32  | 2        | Pc.  |
| 6    | Included         | 1111-0012      | Saddle washer PA black 24x8.4x3                                     | 4        | Pc.  |
| 7    | Included         | 1029-0612      | Button head socket screw with flange M6x12                          | 2        | Pc.  |
| 8*   | Included         | 1029-6001      | Button head socket screw, Tufflock M6x12, with frame colour, 024    | 2        | Pc.  |
|      | Included         | 1029-6001-033  | Buttonheadsocketscrew black, Tufflock M6x12, with frame colour, 032 | 2        | Pc.  |
| 9    | Included         | 3202-1140      | Saddle bushing for front caster                                     | 2        | Pc.  |
| 10   | Included         | 3202-1151      | Cover ring  | 2        | Pc.  |

<sup>\*</sup> Screw`s change with frame colour!

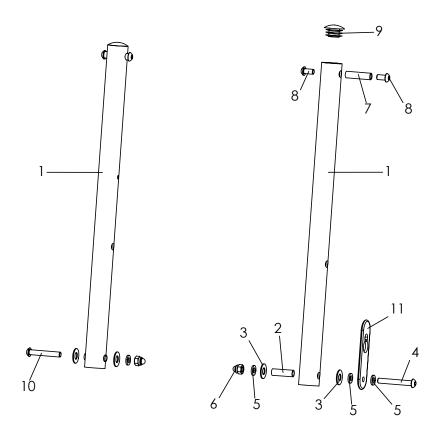


| Pos  | Medema<br>number | Article number | Articel description   | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-10 | 1802-1259        | 3231-1300E-xxx | Front bar, complete   | 1        | Pair |
| 1    | Included         | 3231-1310-xxx  | Front bar   | 2        | Pc.  |
| 2    | Included         | 1114-0025      | Bushing zp 8x0,8x27   | 2        | Pc.  |
| 3    | Included         | 1113-0003      | Washer PA 18x8,4x1 black  | 4        | Pc.  |
| 4    | Included         | 1029-0645      | Button head socket screw M6x45                                      | 2        | Pc.  |
| 5    | Included         | 1051-0006      | Washer A6,4   | 4        | Pc.  |
| 6    | Included         | 1044-0006      | Hexagonal domed cap nut self-locking M6                             | 2        | Pc.  |
| 7    | Included         | 1114-0027      | Threaded bushing 8xM6x32  | 2        | Pc.  |
| 8*   | Included         | 1029-0612      | Button head socket screw, Tufflock M6x12, with frame colour, 024    | 4        | Pc.  |
|      | Included         | 1029-0612-033  | Buttonheadsocketscrew black, Tufflock M6x12, with frame colour, 032 | 4        | Pc.  |
| 9    | Included         | 1141-0009      | Ribbed insert SFL 25x1-2,5  | 2        | Pc.  |
| 10*  | Included         | 1029-0640      | Button head socket screw, Tufflock M6x40, with frame colour, 024    | 1        | Pc.  |
|      | Included         | 1029-0640-033  | Buttonheadsocketscrew black, Tufflock M6x40, with frame colour, 032 | 1        | Pc.  |
| 11   | 1802-1260        | 3231-1431      | Catch strap, left bar.  | 1        | Pc.  |
|      | 1802-1261        | 3235-1431      | With size 2XL use catch strap long.                                 | 1        | Pc.  |

<sup>\*</sup> Screw`s change with frame colour!

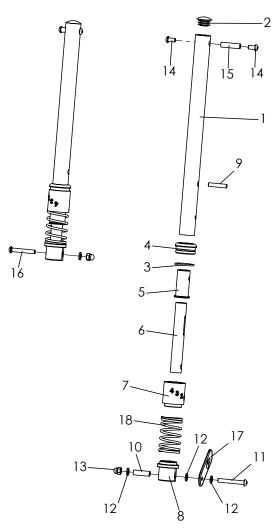

| Pos  | Medema<br>number | Article number | Articel description   | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-16 | 1802-1242        | 3231-7400E-xxx | Frame suspension system, fourfold adjustable, complete              | 1        | Set  |
| 1    | Included         | 3231-7431-xxx  | Front suspension bar  | 2        | Pc.  |
| 2    | Included         | 1141-0009      | Ribbed insert SFL 25x1-2,5  | 2        | Pc.  |
| 3    | Included         | 1150-0012      | O-Ring NBR 70 DIN/ISO 3771 28x2                                     | 2        | Pc.  |
| 4    | Included         | 3231-7436      | End bushing   | 2        | Pc.  |
| 5    | Included         | 3231-7435      | Guiding bushing, suspension   | 2        | Pc.  |
| 6    | Included         | 3231-7433-006  | Guide tube suspension anodised                                      | 2        | Pc.  |
| 7    | Included         | 3231-7434      | Spring hanger (at the top)  | 2        | Pc.  |
| 8    | Included         | 3231-7432      | Spring hanger (at the bottom)                                       | 2        | Pc.  |
| 9    | Included         | 1071-0630      | Parallel pin DIN/ISO 7 6 m6x30                                      | 2        | Pc.  |
| 10   | Included         | 1114-0025      | Bushing zp 8x0,8x27   | 2        | Pc.  |
| 11   | Included         | 1029-0645      | Button head socket screw M6x45                                      | 1        | Pc.  |
| 12   | Included         | 1051-0006      | Washer A6,4   | 4        | Pc.  |
| 13   | Included         | 1044-0006      | Hexagonal domed cap nut self-locking M6                             | 2        | Pc.  |
| 14*  | Included         | 1029-0612      | Button head socket screw, Tufflock M6x12, with frame colour, 024    | 4        | Pc.  |
|      | Included         | 1029-0612-033  | Buttonheadsocketscrew black, Tufflock M6x12, with frame colour, 032 | 4        | Pc.  |
| 15   | Included         | 1114-0027      | Threaded bushing 8xM6x32  | 2        | Pc.  |
| 16*  | Included         | 1029-0640      | Button head socket screw, Tufflock M6x40, with frame colour, 024    | 1        | Pc.  |
|      | Included         | 1029-0640-033  | Buttonheadsocketscrew black, Tufflock M6x40, with frame colour, 032 | 1        | Pc.  |
| 17   | 1802-1260        | 3231-1431      | Catch strap   | 1        | Pc.  |
|      | 1802-1261        | 3235-1431      | Catch strap 2XL   | 1        | Pc.  |
| 18   | Side 19          | 1331-901X      | Coil spring   | 1        | Pc.  |
|      |                  |                | For the corect spring please see next page!                         |          |      |

<sup>\*</sup> Screw`s change with frame colour!

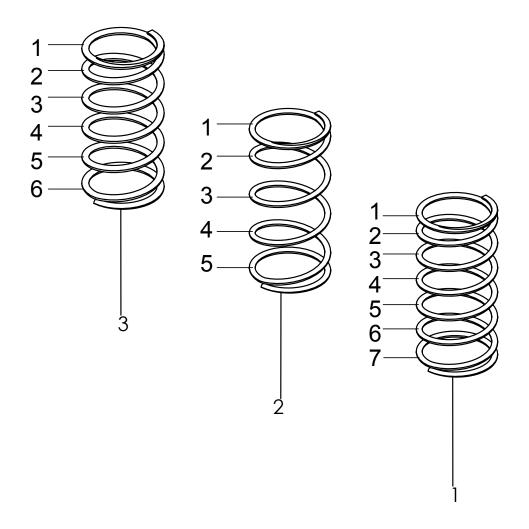


| Pos | Medema<br>number | Article number | Articel description  | Quantity | Unit |
|-----|------------------|----------------|--|----------|------|
| 1   | 1802-1238        | 1131-9017E-022 | Spring set for load up to 15kg, standard color, signal white | 1        | Set  |
| 2   | 1802-1240        | 1131-9018E-022 | Spring set for load up to 25kg, standard color, signal white | 1        | Set  |
| 3   | 1802-1239        | 1131-9019E-022 | Spring set for load from 25kg, standard color, signal white  | 1        | Set  |


#### With order replace **xxx** with colour code!

| 338             | 339                  | 340             | 344                      | 345            |
|-----------------|----------------------|-----------------|--------------------------|----------------|
| sand / irongrey | turquoise / irongrey | pink / irongrey | black diamond / irongrey | red / irongrey |


|  | 1802-1223 | 1131-9018E-yellow | Spring set for load up to 25kg, yellow | 1 | Set |
|--|-----------|-------------------|--|---|-----|
|  | 1802-1224 | 1131-9019E-red    | Spring set for load from 25kg, red     | 1 | Set |

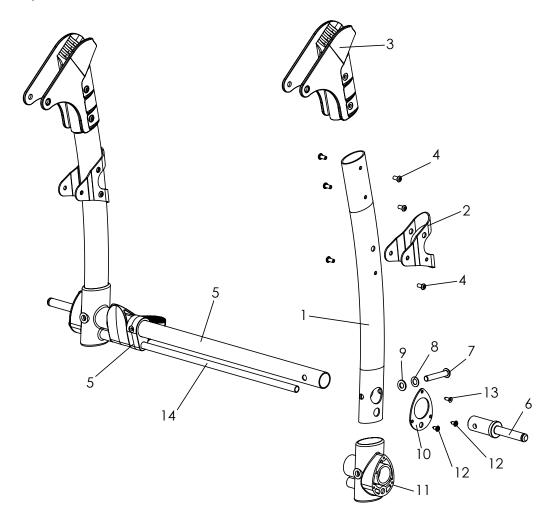

| Pos   | Medema<br>number | Article number | Articel description                                      | Quantity | Unit |
|-------|------------------|----------------|--|----------|------|
| 1-13  | 1802-1220        | 3231-1200E-xxx | Rear frame, Gr. 1, komplett                              | 1        | Pc.  |
|       | 1802-1221        | 3233-1200E-xxx | Rear frame, Gr. 2, komplett                              | 1        | Pc.  |
| 1-4   | 1802-1262        | 3231-1211E-xxx | "Side bar rear, incl. joint bandage                      | 2        | Pc.  |
| 1     | Included         | 3231-1211-xxx  | "Side bar rear, incl. joint bandage                      | 2        | Pc.  |
| 2     | Included         | 3231-1212-xxx  | Joint bandage  | 2        | Pc.  |
| 3     | Included         | 3231-1213      | Rear frame hinge   | 2        | Pc.  |
| 4     | Included         | 1069-0004      | Blind rivet F4,8x10 black                                | 12       | Pc.  |
| 5     | Included         | 3201-1220E-xxx | Axle tube size 1, with mounted brake rod retainer        | 1        | Pc.  |
|       | Included         | 3203-1220E-xxx | Axle tube size 2, with mounted brake rod retainer        | 1        | Pc.  |
| *6    | Included         | 3201-1240      | Axle stump   | 2        | Pc.  |
| 7     | Included         | 1029-0845      | Button head socket screw M8x45                           | 2        | Pc.  |
| 8     | Included         | 1052-0008      | Spring washer B8   | 2        | Pc.  |
| 9     | Included         | 1051-0008      | Washer A8,4  | 2        | Pc.  |
| 10-12 | 1831-1186        | 3201-1280E     | Connector with fasteners                                 | 2        | Pc.  |
| 10    | Included         | 3201-1281      | Stabilizer   | 2        | Pc.  |
| 11    | Included         | 3201-1280      | Connector  | 2        | Pc.  |
| 12    | Included         | 1039-0001      | Screw for plastic, cross rcessed pan head w. flange 4x10 | 2        | Pc.  |
| 13    | Included         | 1039-0006      | Tapping screw with countersunk head                      | 1        | Pc.  |
| 14    | 1803-1270        | 3201-1301-xxx  | Break, see next page!                                    | 1        | Pc.  |
| *6-9  | 1803-1275        | 3201-1240E     | Axle stump as spare part                                 | 1        | Рс   |

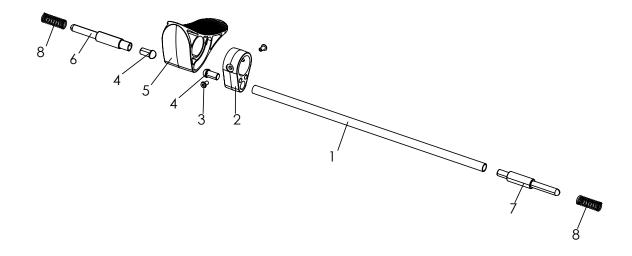


| Pos | Medema<br>number | Article number | Articel description       | Quantity | Unit |
|-----|------------------|----------------|---------------------------|----------|------|
| 1-8 | 1803-1270        | 3201-1300E-xxx | Break size 1, complete    | 1        | Pc.  |
|     | 1803-1271        | 3203-1300E-xxx | Break size 2, complete    | 1        | Pc.  |
| 1   | Included         | 3201-1311-xxx  | Break rod DURO size 1     | 1        | Pc.  |
|     | Included         | 3203-1311-xxx  | Break rod DURO size 2     | 1        | Pc.  |
| 2   | Included         | 3201-1320      | Wheel lock rod retainer   | 1        | Pc.  |
| 3   | Included         | 1062-0408      | Blind rivet F4x8          | 2        | Pc.  |
| 4   | Included         | 3201-1340      | Slide insert              | 2        | Pc.  |
| 5   | Included         | 3201-1310      | Wheel lock lever          | 1        | Pc.  |
| 6   | Included         | 3201-1360      | Lock pin, right           | 1        | Pc.  |
| 7   | Included         | 3201-1350      | Lock pin left             | 1        | Pc.  |
| 8   | Included         | 1131-0034      | Compression spring D-180Q | 2        | Pc.  |

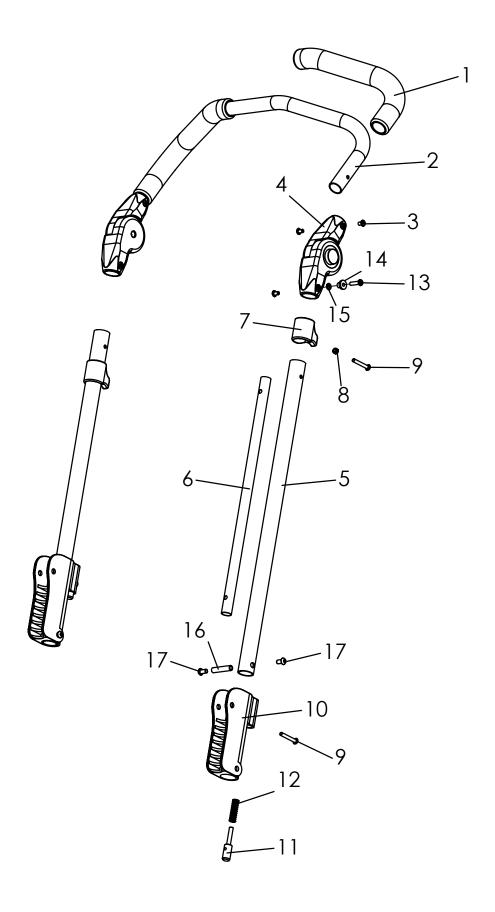


| •    | •                |                |   |          |      |
|------|------------------|----------------|---|----------|------|
| Pos  | Medema<br>number | Article number | Articel description   | Quantity | Unit |
| 1-17 | 1806-1298        | 3231-1400E-xxx | Push bar size 1   | 1        | Pc.  |
|      | 1806-1299        | 3233-1400E-xxx | Push bar size 2   | 1        | Pc.  |
|      | 1806-1321        | 3235-1400E-xxx | Push bar size 2XL   | 1        | Pc.  |
| 1    | 1431-1004        | 1143-0001      | Handle grip, black  | 2        | Pc.  |
| 2-3  | 1806-1322        | 3201-1420E-xxx | Upper part push handle size 1, with fastemers                       | 1        | Pc.  |
|      | 1806-1323        | 3203-1420E-xxx | Upper part push handle size 2, with fastemers                       | 1        | Pc.  |
|      | 1806-1324        | 3233-1420Exxx  | Upper part push handle size 2XL, with fastemers                     | 1        | Pc.  |
| 2    | 1806-1325        | 3201-1420-xxx  | Push bar size 1   | 1        | Pc.  |
|      | 1806-1326        | 3203-1420-xxx  | Push bar size 2   | 1        | Pc.  |
|      | 1806-1327        | 3233-1421-xxx  | Push bar size 2 XL  | 1        | Pc.  |
| 3*   | Included         | 1069-0003      | Blind rivet F5x8 black, with frame colour 032                       | 7        | Pc.  |
|      | Included         | 1062-0508      | Blind rivet F5x8 silver, with frame colour 024                      | 7        | Pc.  |
| 4    | Included         | 1129-0009      | Ratchet joint 25x22   | 1        | Pair |
| 5    | 1806-1328        | 3231-1412-xxx  | Push bar bottom part size 1 and 2                                   | 2        | Pc.  |
|      | 1806-1329        | 3235-1414-xxx  | Push bar bottom part size 2 XL                                      | 2        | Pc.  |
| 6    | 1806-1330        | 3231-1415      | Guide rail size 1 and 2   | 2        | Pc.  |
|      | 1806-1331        | 3235-1415      | Guide rail size 3   | 2        | Pc.  |
| 7    | 1806-1332        | 3231-1414      | Activator   | 2        | Pc.  |
| 8    | Included         | 1042-0005      | Hexagon nut self-locking zp DIN/ISO 985 M5                          | 2        | Pc.  |
| 9    | Included         | 1034-0535      | Torxscrew M5x35   | 4        | Pc.  |
| 10   | Included         | 3231-1411      | Push handle joint   | 2        | Pc.  |
| 11   | Included         | 3231-1413      | Folding-security bolt   | 2        | Pc.  |
| 12   | Included         | 1131-0043      | Compression spring D-143F   | 2        | Pc.  |
| 13   | Included         | 1062-0520      | Blind rivet F5x20   | 1        | Pc.  |
| 14   | Included         | 3231-1432      | Shoulder bushing black  | 1        | Pc.  |
| 15   | Included         | 1051-0005      | Washer A5,3   | 1        | Pc.  |
| 16   | Included         | 1114-0028      | Threaded bushing 8xM6x38  | 2        | Pc.  |
| 17*  | Included         | 1029-0612      | Button head socket screw, Tufflock M6x12, with frame colour, 024    | 4        | Pc.  |
|      |                  | 1029-0612-033  | Buttonheadsocketscrew black, Tufflock M6x12, with frame colour, 032 | 4        | Pc.  |
|      |                  |                |   |          |      |

<sup>\*</sup> Screw`s and rivet`s change with frame colour!

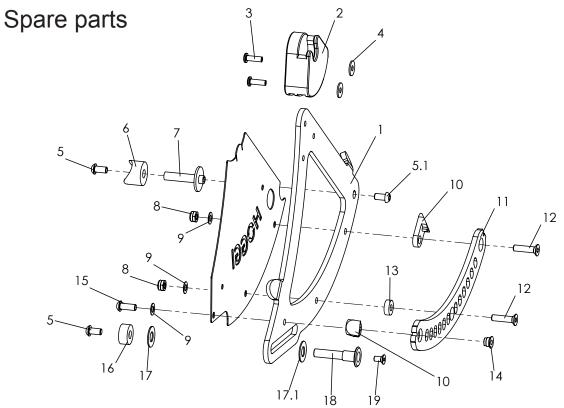


| Pos  | Medema<br>number | Article number | Articel description  | Quantity | Unit |
|------|------------------|----------------|--|----------|------|
| 1-17 | 1831-1187        | 3231-1510E-xxx | Seat Bearing left size 1 and 2, complete, displayed                    | 1        | Pc.  |
|      | 1831-1188        | 3231-1520E-xxx | Seat Bearing right size 1 and 2, complete                              | 1        | Pc.  |
| 1-4  | 1807-1324        | 3231-1511E-xxx | Seat Bearing left, with fasteners                                      | 1        | Pc.  |
|      | 1807-1325        | 3231-1512E-xxx | Seat Bearing right, with fasteners                                     | 1        | Pc.  |
| 1    | Included         | 3231-1511      | Seat Bearing left  | 1        | Pc.  |
|      | Included         | 3231-1512      | Seat Bearing right   | 1        | Pc.  |
| 2    | Included         | 3231-1516      | Scanteling bearing   | 1        | Pc.  |
| 3    | Included         | 1062-0516      | Blind rivet F5x16  | 2        | Pc.  |
| 4    | Included         | 1055-0005      | Washer A5,3  | 2        | Pc.  |
| 5*   | Included         | 1029-0612      | Button head socket screw silver M6x12 for frame colour 024             | 2        | Pc.  |
|      | Included         | 1029-0612-033  | Button head socket screw black M6x12, for frame colour 032             | 2        | Pc.  |
| 5.1* | Included         | 1029-6001      | Button head socket screw silver with lock, M6x12, for frame colour 024 | 1        | Pc.  |
|      | Included         | 1029-6001-033  | Button head socket screw black with lock, M6x12, for frame colour 032  | 1        | Pc.  |
| 6    | Included         | 1111-0013      | Saddle washer PA black 24x8.4x10                                       | 1        | Pc.  |
| 7    | Included         | 3231-1515      | Bearing shaft  | 1        | Pc.  |
| 8    | Included         | 1042-0006-033  | Hexagon nut self-locking, black M6                                     | 2        | Pc.  |
| 9    | Included         | 1052-0006-033  | Spring washer black B6   | 3        | Pc.  |
| 10   | Included         | 3231-1514      | Latch ramp   | 2        | Pc.  |
| 11   | Included         | 3231-1525-006  | Scantling rail left  | 1        | Pc.  |
|      | Included         | 3231-1526-006  | Scantling rail right   | 1        | Pc.  |
| 12   | Included         | 1024-0625-033  | Hexagon socket countersunk head screw black M6x25                      | 2        | Pc.  |
| 13   | Included         | 1113-0051      | Spacer 15x6,4x5  | 1        | Pc.  |
| 14   | Included         | 3231-1527      | Threded insert   | 1        | Pc.  |
| 15   | Included         | 1029-0616-033  | Button head socket screw black M6x16                                   | 1        | Pc.  |
| 16   | Included         | 3231-1518      | Distance bearing   | 1        | Pc.  |
| 17   | Included         | 1113-0056      | Washer PA natural 10,5x21,0x0,8  | 1        | Pc.  |
| 17.1 | Included         | 1113-0068      | Washer PA natural 10,0x18,0x0,8  | 1        | Pc.  |
| 18   | Included         | 3231-1517      | Gudie shaft seat bearing   | 1        | Pc.  |
| 19   | Included         | 1024-0510      | Hexagon socket countersunk head screw M5x10                            | 1        |      |

<sup>\*</sup> Screw`s change with frame colour!

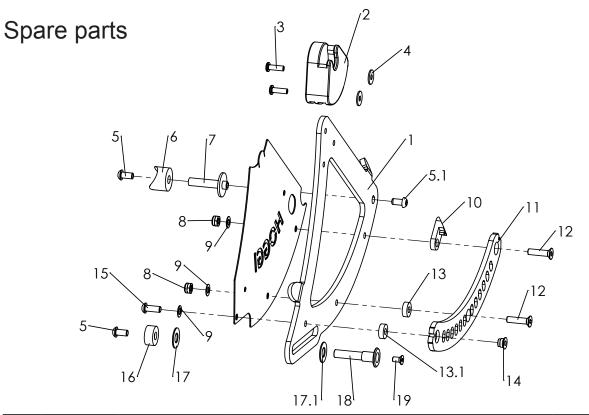


| Pos  | Medema<br>number | Article number | Articel description  | Quantity | Unit |
|------|------------------|----------------|--|----------|------|
| 1-17 | 1807-1326        | 3235-1510E-xxx | Seat Bearing left, complete, displayed only for size 2XL               | 1        |      |
|      | 1807-1327        | 3235-1520E-xxx | Seat Bearing right, complete, only for size 2XL                        | 1        |      |
| 1-4  | 1807-1324        | 3231-1511E-xxx | Seat Bearing left, with fasteners                                      | 1        |      |
|      | 1807-1325        | 3231-1512E-xxx | Seat Bearing right, with fasteners                                     | 1        |      |
| 1    | Included         | 3231-1511-xxx  | Seat Bearing left  | 1        |      |
|      | Included         | 3231-1512-xxx  | Seat Bearing right   | 1        |      |
| 2    | Included         | 3231-1516      | Scanteling bearing   | 1        |      |
| 3    | Included         | 1062-0516      | Blind rivet F5x16  | 2        |      |
| 4    | Included         | 1055-0005      | Washer A5,3  | 2        |      |
| 5*   | Included         | 1029-0612      | Button head socket screw silver M6x12 for frame colour 024             | 2        |      |
|      | Included         | 1029-0612-033  | Button head socket screw black M6x12, for frame colour 032             | 2        |      |
| 5.1* | Included         | 1029-6001      | Button head socket screw silver with lock, M6x12, for frame colour 024 | 1        |      |
|      | Included         | 1029-6001-033  | Button head socket screw black with lock, M6x12, for frame colour 032  | 1        |      |
| 6    | Included         | 1111-0013      | Saddle washer PA black 24x8.4x10                                       | 1        |      |
| 7    | Included         | 3231-1515      | Bearing shaft  | 1        |      |
| 8    | Included         | 1042-0006-033  | Hexagon nut self-locking, black M6                                     | 2        |      |
| 9    | Included         | 1052-0006-033  | Spring washer black B6   | 3        |      |
| 10   | Included         | 3231-1514      | Latch ramp   | 2        |      |
| 11   | Included         | 3231-1525-006  | Scantling rail left  | 1        |      |
|      | Included         | 3231-1526-006  | Scantling rail right   | 1        |      |
| 12   | Included         | 1024-0625-033  | Hexagon socket countersunk head screw black M6x25                      | 2        |      |
| 13   | Included         | 1113-0051      | Spacer 15x6,4x5  | 1        |      |
| 13.1 | Included         | 1113-0031      | Spacer 8,4x15x5  | 1        |      |
| 14   | Included         | 3231-1527      | Threded insert   | 1        |      |
| 15   | Included         | 1029-0616-033  | Button head socket screw black M6x16                                   | 1        |      |
| 16   | Included         | 3231-1518      | Distance bearing   | 1        |      |
| 17   | Included         | 1113-0056      | Washer PA natural 10,5x21,0x0,8  | 1        |      |
| 17.1 | Included         | 1113-0068      | Washer PA natural 10,0x18,0x0,8  | 1        |      |
| 18   | Included         | 3231-1517      | Gudie shaft seat bearing   | 1        |      |
| 19   | Included         | 1024-0510      | Hexagon socket countersunk head screw M5x10                            | 1        |      |


<sup>\*</sup> Screw`s change with frame colour!

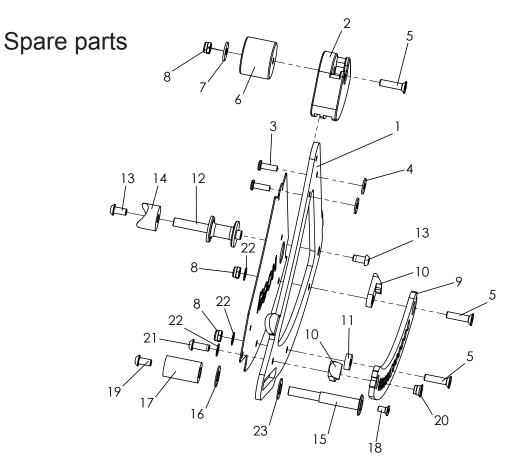


| Pos  | Medema<br>number | Article number | Articel description                                  | Quantity | Unit |
|------|------------------|----------------|--|----------|------|
| 1-23 | 1807-1328        | 3230-1510E-032 | Seat Bearing left, complete, displayed only for Mini | 1        | Pc.  |
|      | 1807-1329        | 3230-1520E-032 | Seat Bearing right, complete, only for Mini          | 2        | Pc.  |
| 1-4  | 1807-1330        | 3230-1511E-032 | Seat Bearing Mini left, with fasteners               | 1        | Pc.  |
|      | 1807-1331        | 3230-1512E-032 | Seat Bearing Mini right, with fasteners              | 1        | Pc.  |
| 1    | Included         | 3230-1511-032  | Seat bearing, links                                  | 1        | Pc.  |
|      | Included         | 3230-1512-032  | Seat bearing, rechts                                 | 1        | Pc.  |
| 2    | Included         | 3230-1516      | Scantlings bearing Mini                              | 1        | Pc.  |
| 3    | Included         | 1062-0516      | Blind rivet F5x16                                    | 2        | Pc.  |
| 4    | Included         | 1055-0005      | Washer A5,3  | 2        | Pc.  |
| 5    | Included         | 1024-0625-033  | Hexagon socket countersunk head screw black M6x25    | 3        | Pc.  |
| 6    | Included         | 3230-1519      | Distance piece for Mini                              | 1        | Pc.  |
| 7    | Included         | 1055-0006      | Washer A6,4  | 1        | Pc.  |
| 8    | Included         | 1042-0006-033  | Hexagon nut self-locking black M6                    | 3        | Pc.  |
| 9    | Included         | 3231-1525-006  | Scantling rail left                                  | 1        | Pc.  |
|      | Included         | 3231-1526-006  | Scantling rail right                                 | 1        | Pc.  |
| 10   | Included         | 3231-1514      | Latch ramp   | 2        | Pc.  |
| 11   | Included         | 1113-0051      | Spacer PA 6,4x15x5                                   | 1        | Pc.  |
| 12   | Included         | 3230-1515-033  | Bearing shaft mini black                             | 1        | Pc.  |
| 13   | Included         | 1029-6001      | Button head socket screw, Tufflock M6x12             | 2        | Pc.  |
| 14   | Included         | 1111-0013      | Saddle washer PA black 24x8.4x10                     | 1        | Pc.  |
| 15   | Included         | 3230-1517-008  | Guided shaft seat bearing for Mini                   | 1        | Pc.  |
| 16   | Included         | 1113-0056      | Washer PA natural 10,5x21,0x0,8 mm                   | 1        | Pc.  |
| 17   | Included         | 3230-1518-007  | Distance piece, Guided shaft for Mini                | 1        | Pc.  |
| 18   | Included         | 1024-0510      | Hexagon socket countersunk head screw M5x10          | 1        | Pc.  |
| 19   | Included         | 1029-0612      | Button head socket screw M6x12                       | 1        | Pc.  |
| 20   | Included         | 3231-1527      | Threaded insert                                      | 1        | Pc.  |
| 21   | Included         | 1029-0616-033  | Button head socket screw black M6x16                 | 1        | Pc.  |
| 22   | Included         | 1052-0006-033  | Spring washer black B 6                              | 3        | Pc.  |
| 23   | Included         | 1113-0068      | Washer PA natural 10,0x18,0x0,8                      | 1        | Pc.  |

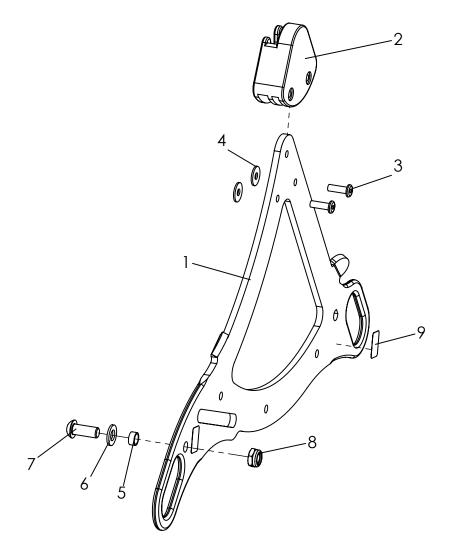


| Pos | Medema<br>number | Article number | Articel description                         | Quantity | Unit |
|-----|------------------|----------------|---|----------|------|
| 1-9 | 1817-1115        | 3231-7300E-xxx | Tie down kit                                | 1        | Set  |
| 1-4 | Included         | 3231-7311E-xxx | Seat bearing plate with lashing eyes, right | 1        | Pc.  |
|     | Included         | 3231-7312E-xxx | Seat bearing plate with lashing eyes, left  | 1        | Pc.  |
| 2   | Included         | 3231-1516      | Scantlings bearing Mini                     | 2        | Pc.  |
| 3   | Included         | 1062-0516      | Blind rivet F5x16                           | 4        | Pc.  |
| 4   | Included         | 1055-0005      | Washer A5,3                                 | 4        | Pc.  |
| 5   | Included         | 3201-7350      | Slide bushing                               | 2        | Pc.  |
| 6   | Included         | 1051-0008      | Washer A8,4                                 | 2        | Pc.  |
| 7   | Included         | 1029-0820      | Button head socket screw M8x20              | 2        | Pc.  |
| 8   | Included         | 1042-0008      | Hexagon nut self-locking M8                 | 2        | Pc.  |
| 9   | Included         | 1914-0007      | Adhesive lable with hook symbol (4 pcs.)    | 1        | Pc.  |

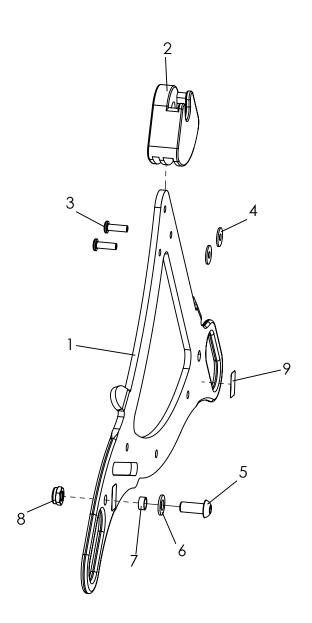

| Pos  | Medema<br>number | Article number | Articel description                         | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-13 | 1817-1117        | 3230-7300E     | Tie down kit Mini                           | 1        | Pc.  |
| 1    | Included         | 3231-7311E-xxx | Seat bearing plate with lashing eyes, left  | 1        | Pc.  |
|      | Included         | 3231-7312E-xxx | Seat bearing plate with lashing eyes, right | 1        | Pc.  |
| 2    | Included         | 3230-1516      | Scantlings bearing Mini                     | 2        | Pc.  |
| 3    | Included         | 1062-0516      | Blind rivet F5x16                           | 4        | Pc.  |
| 4    | Included         | 1055-0005      | Washer A5,3                                 | 4        | Pc.  |
| 5    | Included         | 1029-0820      | Slide bushing                               | 2        | Pc.  |
| 6    | Included         | 1051-0008      | Washer A8,4                                 | 2        | Pc.  |
| 7    | Included         | 3201-7350      | Button head socket screw M8x20              | 2        | Pc.  |
| 8    | Included         | 1042-0008      | Hexagon nut self-locking M8                 | 2        | Pc.  |
| 9    | Included         | 1914-0007      | Adhesive lable with hook symbol (4 pcs.)    | 1        | Pc.  |

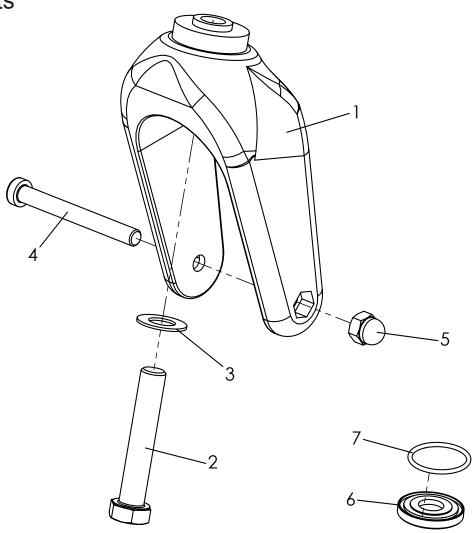

| Pos | Medema<br>number | Article number | Articel description                             | Quantity | Unit |
|-----|------------------|----------------|---|----------|------|
| 1-5 | 1803-1204        | 1281-0007E     | Castor fork with fasteners                      | 1        | Pc.  |
| 1   | Included         | 1281-0007      | Castor fork LG-K-200-12 schw 200x60             | 1        | Pc.  |
| 2   | Included         | 1012-1265      | Hexagon head screw M12x65                       | 1        | Pc.  |
| 3   | Included         | 1052-0012      | Spring washer B12                               | 1        | Pc.  |
| 4   | Included         | 1022-0875      | Hexagon socket head cap screw M8x75             | 1        | Pc.  |
| 5   | Included         | 1044-0008      | Hexagonal domed cap nut self-locking M8         | 1        | Pc.  |
| 6-7 | 1803-1332        | 3202-7120E     | Sealing for steering fork bearing for two forks | 1        | Set  |
| 6   | Included         | 3224-1174      | Sealing disc caster fork                        | 2        | Pc.  |
| 7   | Included         | 1150-0018      | O-ring 26x2                                     | 2        | Pc.  |

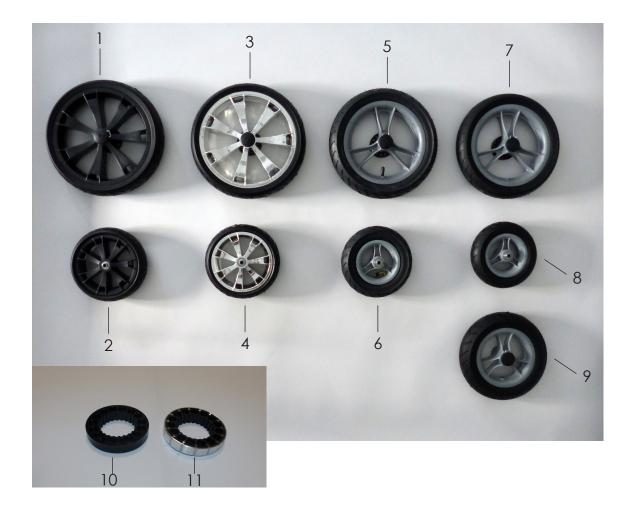


| Pos | Medema<br>number | Article number | Articel description                                       | Quantity  | Unit |
|-----|------------------|----------------|---|---|------|
| 1   | 1803-1286        | 1221-3063E     | PU-Wheel R300/60 black rim, with break disc, plastic      | 2   | Pc.  |
|     | 1803-1333        | 1221-3064      | PU-Rad R300/60 black rim, break disk with alloy ring      | 2   | Pc.  |
| 2   | 1803-1287        | 1221-2049      | PU-Rad 200x48, black rim                                  | 2   | Pc.  |
| 3   | 1803-1282        | 1221-3061E     | PU-Rad 300x60, chromed rim with break disk, plastic       | preak disk with alloy ring  2 Po 2 Po 2 Po 3 Po 4 Po 5 Po 5 Po 6 Po 6 Po 6 Po 7 Po 7 Po 7 Po 8 Po 8 Po 8 Po 8 Po 8 Po 9 Po 1 | Pc.  |
|     | 1803-1334        | 1221-3062      | PU-Rad 300x60, chromed rim and break disk with alloy ring | 2   | Pc.  |
| 4   | 1803-1281        | 1221-2048      | PU-Rad 200x48, rim chromed                                | 2   | Pc.  |
| 5   | 1803-1335        | 1211-2962      | Luftrad 290x60, rim grey and break disc with alloy ring   | 2   | Pc.  |
| 6   | 1803-1336        | 1211-1950      | Luftrad 190x50, rim grey                                  | 2   | Pc.  |
| 7   | 1803-1214        | 1221-2961      | PU-Rad 290x60, rim grey with break disc, plastic          | 2   | Pc.  |
|     | 1803-1337        | 1221-2962      | PU-Rad 290x60, rim grey and break disk with alloy ring    | 2   | Pc.  |
| 8   | 1803-1210        | 1221-1950      | PU-Rad 190x50, rim grey                                   | 2   | Pc.  |
| 9   | 1403-1215        | 1211-0019      | Pneumatic 250x250, rim grey (rigid frame, front wheels)   | 2   | Pc.  |
|     | 1403-1216        | 1221-0019      | PU-Rad 250x250, rim grey (rigid frame, front wheels)      | 2   | Pc.  |
| 10  | 1803-1338        | 1204-0002      | Brake disc, plastic                                       | 2   | Pc.  |
| 11  | 1803-1339        | 1204-0005      | Brake disc with alloy-ring                                | 2   | Pc.  |
|     | 1803-1301        | 1200-0001      | Tire for wheel 290x60                                     | 1   | Pc.  |
|     | 1803-1302        | 1200-0002      | Tire for wheel 190x50                                     | 1   | Pc.  |
|     | 1803-1304        | 1201-0001      | Tube for wheel 290 x 60                                   | 1   | Pc.  |
|     | 1803-1305        | 1201-0002      | Tube for wheel 190x50                                     | 1   | Pc.  |




| Pos | Medema<br>number | Article number | Articel description                                  | Quantity | Unit |
|-----|------------------|----------------|--|----------|------|
| 1-3 | 1803-1208        | 1211-2961      | Pneumatic wheel 290x60, quick release                | 1        | Pc.  |
|     | 1803-1214        | 1221-2961      | PU-Wheel 290x60, quick release                       | 1        | Pc.  |
| 2-4 | Included         | 1209-0011      | Distance ring, plastic                               | 1        | Pc.  |
| 3   | Included         | 1090-0002      | Grooved ball bearing 6201                            | 1        | Pc.  |
| 4   | Included         | 1211-0024      | Pneumatic 290x60, 5 spoke wheel black, quick release | 1        | Pc.  |
|     | Included         | 1221-0035      | PU-Wheel 290x60, 5 Spoke wheel black, quick release  | 1        | Pc.  |
| 5   | Included         | 1205-0004      | Quick release 128mm for hollow axle                  | 1        | Pc.  |
| 6   | Included         | 3206-1160      | Pressure bushing L 23                                | 1        | Pc.  |
| 7   | Included         | 3206-1150      | Pressure bushing L 19                                | 1        | Pc.  |
| 8   | Included         | 3206-1140      | Axel   | 1        | Pc.  |




| Pos   | Medema<br>number | Article number | Articel description   | Quantity | Unit |
|-------|------------------|----------------|---|----------|------|
| 1-24  | 1803-1340        | 1219-0029E     | Drum brake wheels, pneumatic tyres,black 5-spoke rim, rigid front wheels      | 1        | Set  |
|       | 1803-1272        | 1229-0029E     | Drum brake wheels, PU tyres,black 5-spoke rim, rigid front wheels             | 1        | Set  |
|       | 1803-1341        | 1219-0030E     | Drum brake wheels, pneumatic tyres,black 5-spoke rim, swivelling front wheels | 1        | Set  |
|       | 1805-1252        | 1229-0030E     | Drum brake wheels, PU tyres,black 5-spoke rim, swivelling front wheels        | 1        | Set  |
|       | 1803-1342        | 1219-0031E     | Drum brake wheels, pneumatic tyres, black 5-spoke rims, TRIKE                 | 1        | Set  |
|       | 1803-1343        | 1229-0031E     | drum brake wheels, PU tyres,black 5-spoke rim, TRIKE                          | 1        | Set  |
| 1     | Included         | 3201-1271      | Bracket   | 2        | Pc.  |
| 2     | Included         | 1205-0041      | Brake plate steel left with 12 mm hole  | 1        | Pc.  |
|       | Included         | 1205-0042      | Brake plate steel right with 12 mm hole                                       | 1        | Pc.  |
| 3     | Included         | 1051-0012      | Washer A 13   | 2        | Pc.  |
| 4     | Included         | 1043-0012      | Hexagon nut M12   | 2        | Pc.  |
| 5     | Included         | 3201-1242      | Axle bolt   | 2        | Pc.  |
| 6     | Included         | 1029-0635      | Button head socket screw M6x35  | 2        | Pc.  |
| 7     | Included         | 1114-0052      | Threaded bushing 8xM6x23  | 2        | Pc.  |
| 8     | Included         | 1030-0614      | Button head socket screw with flange M6x14                                    | 2        | Pc.  |
| 9     | Included         | 1219-0032      | Pneumatic wheel 310x55, black 5 spoke plastic rim, drum brake left side       | 1        | Set  |
|       | Included         | 1219-0033      | Pneumatic wheel 310x55, 5 spoke plastic rim, drum brake right side            | 1        | Set  |
|       | Included         | 1229-0025      | PU-wheel 290x60, 5 spoke plastic rim, drum brake left side                    | 1        | Set  |
|       | Included         | 1229-0026      | PU-wheel 290x60, 5 spoke plastic rim, drum brake right side                   | 1        | Set  |
| 10    | Included         | 1049-0065      | BILOC-nut M12   | 2        | Pc.  |
| 11    | Included         | 1209-0004      | Hub cap KN-S-32/12  | 2        | Pc.  |
| 12    | Included         | 1211-0017      | Pneumatic wheel 190x50 with black 3 spoke plastic rim                         | 2        | Pc.  |
|       | Included         | 1221-0017      | PU-wheel 190x50 with black 3 spoke plastic rim                                | 2        | Pc.  |
|       | Included         | 1211-0025      | Pneumatic wheel 250x50, black 5 spoke plastic rim with quick release          | 2        | Pc.  |
|       | Included         | 1221-0036      | PU-wheel 250x50, black 5 spoke plastic rim with quick release                 | 2        | Pc.  |
|       | Included         | 1211-0024      | Pneumatic 5 spoke wheel black, 290x60, quick release                          | 1        | Pc.  |
| 13    | Included         | 1209-0030      | Brake lever 36 ° manually detent 22 mm  | 2        | Pc.  |
| 14-16 | 1803-1278        | 1209-0001      | Bowden wire set 1,5m cable  | 2        | Set  |
| 17    | Included         | 1089-0004      | Cable tie, black, 5x180   | 2        | Pc.  |
| 18    | Included         | 1149-0001      | Cable clip D 25   | 2        | Pc.  |
| 19    | Included         | 1050-0006      | Washer A6,4   | 2        | Pc.  |
| 20    | Included         | 1149-0003      | Cable clip d 6,4 / 8 mm, black  | 4        | Pc.  |
| 21    | Included         | 1062-0516      | Blind rivet F5x16   | 2        | Pc.  |
| 22    | Included         | 1051-0005      | Washer A5,3   | 6        | Pc.  |
| 23    | Included         | 1029-0616      | Button head socket screw M6x16  | 2        | Pc.  |
| 24    | Included         | 1051-0006      | Washer A6,4   | 2        | Pc.  |

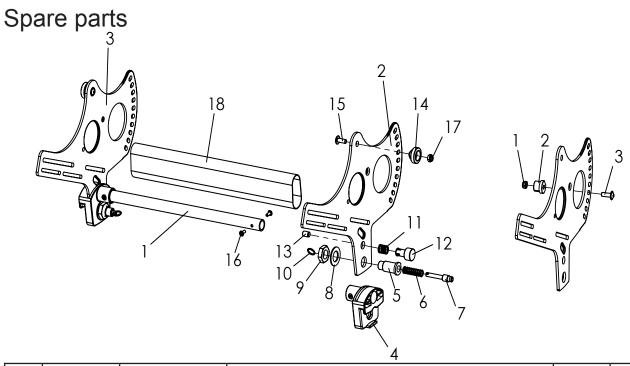


| Pos  | Medema<br>number | Article number          | Articel description                                  | Quantity | Unit |
|------|------------------|-------------------------|--|----------|------|
| 1-18 | 1807-1332        | 3230-2100E-xxx          | Seat base Mini, complete                             | 1        | Pc.  |
|      | 1807-1333        | 3232-2100E-xxx          | Seat base size 1, complete                           | 1        | Pc.  |
|      | 1807-1334        | 3234-2100E-xxx          | Seat base size 2, complete                           | 1        | Pc.  |
|      | 1807-1335        | 3235-2100E-xxx          | Seat base size 2XL, complete                         | 1        | Pc.  |
|      | Included         | 3230-2119-xxx           | Cross strut, seat Mini                               | 1        | Pc.  |
| 1    | Included         | 3231-2119-xxx           | Cross strut, seat size 1                             | 1        | Pc.  |
|      | Included         | 3233-2119-xxx           | Cross strut, seat size 2                             | 1        | Pc.  |
| 2    | Included         | 3230-2121-007           | Side plate seat base Mini, left                      | 1        | Pc.  |
|      | Included         | 3230-2122-007           | Side plate seat base Mini, right                     | 1        | Pc.  |
| 3    | Included         | 3231-2121-007           | Side plate, seat base size 1 and 2, left             | 1        | Pc.  |
|      | Included         | 3231-2122-007           | Side plate, seat base size 1 and 2, right            | 1        | Pc.  |
|      | Included         | 3235-2121-007           | Side plate, seat base size 2XL, left                 | 1        | Pc.  |
|      | Included         | 3235-2122-007           | Side plate, seat base size 2XL, right                | 1        | Pc.  |
| 4    | Included         | 3231-2114               | Scantlings guide                                     | 2        | Pc.  |
| 5    | Included         | 3231-2115               | Locking bolt housing                                 | 2        | Pc.  |
| 6    | Included         | 1131-0070               | Compression D-162                                    | 2        | Pc.  |
| 7    | Included         | 3240-2422               | Locking bolt   | 2        | Pc.  |
| 8    | Included         | 1052-0016               | Spring washer B16                                    | 2        | Pc.  |
| 9    | Included         | 1040-0016               | Hexagon nut 16x1,5                                   | 2        | Pc.  |
| 10   | Included         | 1129-0036               | Key ring 16x1,6                                      | 2        | Pc.  |
| 11   | Included         | 1131-0065               | Compression spring D-117J                            | 1        | Pc.  |
| 12   | Included         | 3231-2124               | Lock button  | 1        | Pc.  |
| 13   | Included         | 1031-0820               | Hexagon socket set screw M8x20                       | 1        | Pc.  |
| 14   | Included         | 3231-2113               | Scantlings shaft                                     | 2        | Pc.  |
| 15   | Included         | 1030-0616               | Button head socket screw with flange M6x16           | 2        | Pc.  |
| 16   | Included         | 1062-0410               | Blind rivet F4x10                                    | 4        | Pc.  |
| 17   | Included         | 1042-0006               | Hexagon nut self-locking M6                          | 2        | Pc.  |
| 18   | Included         | 3231-2140               | cover for tilt mechanism size 1                      | 1        | Pc.  |
| 18.1 | Included         | 3233-2140               | cover for tilt mechanism size 2                      | 1        | Pc.  |
| 1-3  |                  | Limitation of back tilt | Size 2XL, to be mounted fourth hole from the bottom! | 1        | Set  |
| 1    | Included         | 1042-0006               | Hexagon nut self-locking M6                          | 1        | Pc.  |
| 2    | Included         | 3240-1180               | Bearing plug   | 1        | Pc.  |
| 3    | Included         | 1030-0618               | Button head socket screw with flange M6x18           | 1        | Pc.  |

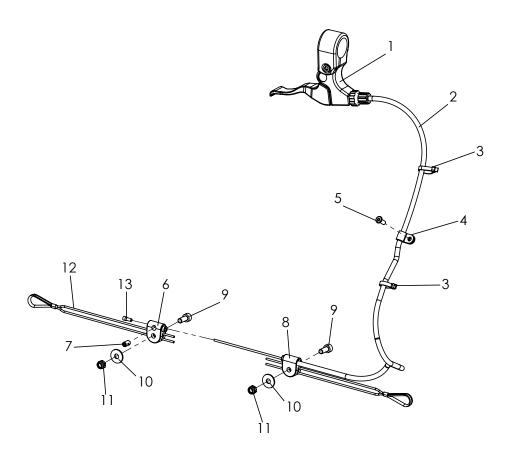


| Pos  | Medema<br>number | Article number | Articel description   | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 2-13 | 1802-1213        | 3231-1900E     | Releaser, tiltingmechanism                                      | 1        | Pc.  |
| 1    | 1803-1344        | 1209-0017      | Brake lever with splittet bracket, d 22 mm black                | 1        | Pc.  |
| 2    | Included         | 1209-0029      | Cable BZ-670x780  | 1        | Pc.  |
| 3    | Included         | 1089-0007      | Cable tie 3,3 x 145   | 2        | Pc.  |
| 4    | Included         | 1119-0003      | Wire clamp  | 1        | Pc.  |
| 5    | Included         | 1062-0410      | Blind rivit F4x10   | 1        | Pc.  |
| 6    | Included         | 3243-1932      | Nipple bearing  | 1        | Pc.  |
| 7    | Included         | 1099-0004      | Screw nipple 1100, D5x8   | 1        | Pc.  |
| 8    | Included         | 3243-1931      | Bowden bearing  | 1        | Pc.  |
| 9    | Included         | 1021-0512      | Cylinder head screw, M5x12                                      | 2        | Pc.  |
| 10   | Included         | 1055-0005      | Washer A5,3   | 2        | Pc.  |
| 11   | Included         | 1042-0005      | Hexagon nut self-locking M5                                     | 2        | Pc.  |
| 12   | Included         | 1129-1001      | Universal wire-rope with rope thimble for seat angle adjustment | 2        | Pc.  |
| 13   | Included         | 1209-0007      | End cap aluminium U-BA-S for Bowden cable wire                  | 1        | Pc.  |

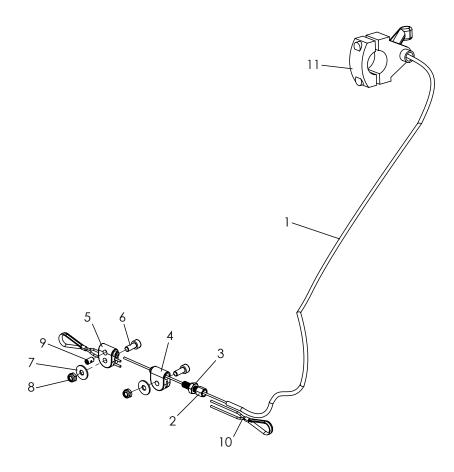

| Pos  | Medema<br>number | Article number | Articel description  | Quantity | Unit |
|------|------------------|----------------|--|----------|------|
| 1-10 | 1802-1263        | 3230-1900E     | Releaser, tiltingmechanism, Mini                                       | 1        | Pc.  |
| 1    | Included         | 1209-0029      | Cable BZ-670x780   | 1        | Pc.  |
| 2    | Included         | 1099-0003      | Adjusting screw  | 1        | Pc.  |
| 3    | Included         | 1042-0006      | Hexagon nut self-locking M6  | 1        | Pc.  |
| 4    | Included         | 3243-1931      | Wire clamp   | 1        | Pc.  |
| 5    | Included         | 3243-1932      | Nipple bearing   | 1        | Pc.  |
| 6    | Included         | 1021-0512      | Cylinder head screw M5x12  | 2        | Pc.  |
| 7    | Included         | 1055-0005      | Washer A5,3  | 2        | Pc.  |
| 8    | Included         | 1042-0005      | Hexagon nut self-locking M5  | 2        | Pc.  |
| 9    | Included         | 1099-0004      | Screw nipple 1100, D5x8  | 1        | Pc.  |
| 10   | Included         | 3230-1129      | Universal wire-rope with rope thimble, for seat angle adjustment, Mini | 2        | Pc.  |
| 11   | Included         | 1209-0036      | Release lever D22  | 1        | Pc.  |

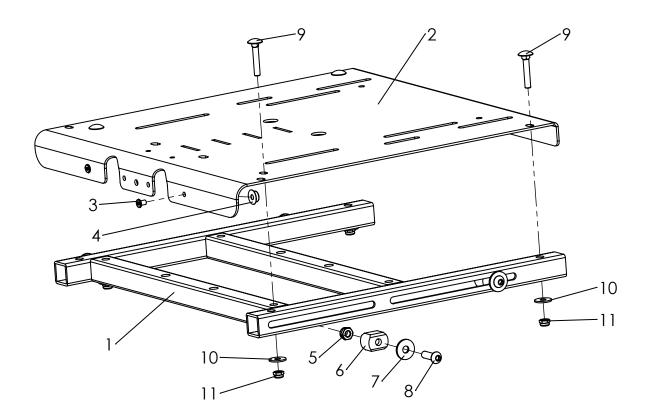


| Pos  | Medema<br>number | Article number | Articel description                   | Quantity | Unit |
|------|------------------|----------------|---------------------------------------|----------|------|
| 1-11 | 1807-1336        | 3232-2200E-xxx | Seat surface size 1, with conecters   | 1        | Pc.  |
|      | 1807-1337        | 3234-2200E-xxx | Seat surface size 2, with conecters   | 1        | Pc.  |
|      | 1807-1338        | 3235-2200E-xxx | Seat surface size 2XL, with conecters | 1        | Pc.  |
| 1    | 1807-1339        | 3231-2250-xxx  | Adjustable seat frame size 1          | 1        | Pc.  |
|      | 1807-1340        | 3233-2250-xxx  | Adjustable seat frame size 2          | 1        | Pc.  |
|      | 1807-1341        | 3235-2250-xxx  | Adjustable seat frame size 2 XL       | 1        | Pc.  |
| 2    | 1807-1342        | 3232-2221-xxx  | Seat surface size 1                   | 1        | Pc.  |
|      | 1807-1343        | 3234-2221-xxx  | Seat surface size 2                   | 1        | Pc.  |
|      | 1807-1344        | 3235-2221-xxx  | Seat surface size 2XL                 | 1        | Pc.  |
| 3    | Included         | 1061-0406      | Blind rivet F4x6                      | 4        | Pc.  |
| 4    | Included         | 1533-0005      | Ring-ball part DRK 95B                | 4        | Pc.  |
| 5    | Included         | 1042-0006      | Hexagon nut self-locking M6           | 4        | Pc.  |
| 6    | Included         | 3231-2262      | Clamping block                        | 4        | Pc.  |
| 7    | Included         | 1055-0006      | Washer A6,4                           | 4        | Pc.  |
|      | Included         | 1055-0006-033  | Washer A6,4 black                     | 4        | Pc.  |
| 8    | Included         | 1029-0620      | Button head socket screw M6x20        | 4        | Pc.  |
|      | Included         | 1029-0620-033  | Button head socket screw black M6x20  | 4        | Pc.  |
| 9    | Included         | 1026-0530      | Mushroom head square neck bolt M5x30  | 4        | Pc.  |
| 10   | Included         | 1055-0005      | Washer A5,3                           | 4        | Pc.  |
| 11   | Included         | 1042-0005      | Hexagon nut self-locking M5           | 4        | Pc.  |

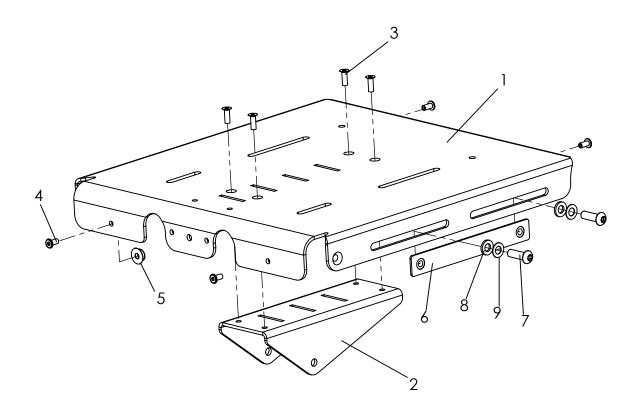


| Pos | Medema<br>number | Article number | Articel description                  | Quantity | Unit |
|-----|------------------|----------------|--------------------------------------|----------|------|
| 1-9 | 1807-1345        | 3230-2200-032  | Seat surface Mini, complete          | 1        | Pc.  |
| 1-3 | 1807-1346        | 3230-2211-032  | Seat surface Mini                    | 1        | Pc.  |
| 2   | Included         | 3230-2212-032  | Retainer for Footrest hanger         | 1        | Pc.  |
| 3   | Included         | 1063-0412      | Blind rivet S4x12                    | 4        | Pc.  |
| 4   | Included         | 1061-0406      | Blind rivet F4x6                     | 4        | Pc.  |
| 5   | Included         | 1533-0005      | Ring-ball part                       | 4        | Pc.  |
| 6   | Included         | 3230-2231      | Threadplate M6x110                   | 2        | Pc.  |
| 7   | Included         | 1029-0620-033  | Button head socket screw black M6x20 | 4        | Pc.  |
| 8   | Included         | 1055-0006-033  | Washer A6,4                          | 4        | Pc.  |
| 9   | Included         | 1052-0006-033  | Spring washer B6                     | 4        | Pc.  |


| Pos | Medema<br>number | Article number | Articel description                                      | Quantity | Unit |
|-----|------------------|----------------|--|----------|------|
| 1-9 | 1813-1494        | 3230-2602E     | Hip support right Mini                                   | 1        | Pc.  |
|     | 1813-1495        | 3230-2601E     | Hip support left Mini                                    | 1        | Pc.  |
|     | 1813-1463        | 3231-2602E     | Hip support right size 1                                 | 1        | Pc.  |
|     | 1813-1464        | 3231-2601E     | Hip support left size 1                                  | 1        | Pc.  |
|     | 1813-1465        | 3233-2602E     | Hip support right size 2                                 | 1        | Pc.  |
|     | 1813-1466        | 3233-2601E     | Hip support left size 2                                  | 1        | Pc.  |
|     | 1813-1476        | 3235-2602E     | Hüftpelotte right Gr. 2XL                                | 1        | Pc.  |
|     | 1813-1475        | 3235-2601E     | Hüftpelotte left Gr. 2XL                                 | 1        | Pc.  |
| 1   | 1813-1498        | 3230-2612-032  | Bracket for hip supports, right, Mini, black dull matt   | 1        | Pc.  |
|     | 1813-1499        | 3230-2611-032  | Bracket for hip supports, left, Mini, black dull matt    | 1        | Pc.  |
|     | 1813-1500        | 3231-2612-032  | Bracket for hip supports, right, size 1, black dull matt | 1        | Pc.  |
|     | 1813-1501        | 3231-2611-032  | Bracket for hip supports, left, size 1, black dull matt  | 1        | Pc.  |
|     | 1813-1502        | 3233-2612-032  | Bracket for hip supports, right, size 2, black dull matt | 1        | Pc.  |
|     | 1813-1503        | 3233-2611-032  | Bracket for hip supports, leftt, size 2, black dull matt | 1        | Pc.  |
| 2-3 | 1813-1504        | 3230-2623E     | Hip support formed part, right, Mini with velcro         | 1        | Pc.  |
|     | 1813-1505        | 3230-2621E     | Hip support formed part, left, Mini with velcro          | 1        | Pc.  |
|     | 1813-1506        | 3231-2623E     | Hip support formed part, right, size 1 with velcro       | 1        | Pc.  |
|     | 1813-1507        | 3231-2621E     | Hip support formed part, left, size 1 with velcro        | 1        | Pc.  |
|     | 1813-1508        | 3233-2623E     | Hip support formed part, right, size 2 with velcro       | 1        | Pc.  |
|     | 1813-1509        | 3233-2621E     | Hip support formed part, left, size 2 with velcro        | 1        | Pc.  |
|     | 1813-1510        | 3235-2625-032  | Hip support formed part, right, size 2XL with velcro     | 1        | Pc.  |
|     | 1813-1511        | 3235-2624-032  | Hip support formed part, left, size 2XL with velcro      | 1        | Pc.  |
| 3   | Included         | 1501-0003      | Hook strap black 25mm                                    | 0,2      | m    |
| 4   | Included         | 3231-2622      | Locking hinge  | 2        | Pc.  |
| 5   | Included         | 1030-0512      | Button head socket screw with flange M5x12               | 4        | Pc.  |
| 6   | Included         | 1042-0005      | Hexagon nut self-locking M5                              | 4        | Pc.  |
| 7   | Included         | 1026-0525      | Mushroom head square neck bolt M5x25                     | 4        | Pc.  |
| 8   | Included         | 1123-0500      | Wing nut PA black M5                                     | 4        | Pc.  |
| 9   | Included         | 1049-0066      | BILOC-nut M5   | 4        | Pc.  |

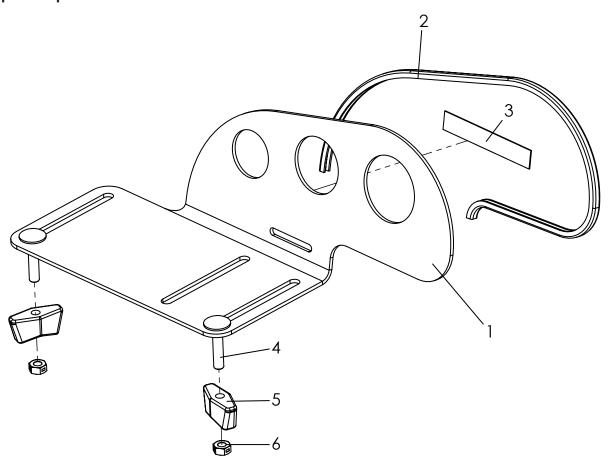


| Pos | Medema<br>number | Article number | Articel description                         | Quantity | Unit |
|-----|------------------|----------------|---|----------|------|
| 1-6 | 1813-1512        | 3230-2901      | Pelotte thigh support Mini, left, picture   | 1        | Pc.  |
|     | 1813-1513        | 3230-2902      | Pelotte thigh support Mini, right           | 1        | Pc.  |
|     | 1813-1514        | 3231-2901E     | Pelotte thigh support size 1, left          | 1        | Pc.  |
|     | 1813-1515        | 3231-2902E     | Pelotte thigh support size 1, right         | 1        | Pc.  |
|     | 1813-1516        | 3233-2901E     | Pelotte thigh support size 2, left, picture | 1        | Pc.  |
|     | 1813-1517        | 3233-2902E     | Pelotte thigh support size 2, right         | 1        | Pc.  |
|     | 1813-1518        | 3235-2901E     | Pelotte thigh support size 2XL, left        | 1        | Pc.  |
|     | 1813-1519        | 3235-2902E     | Pelotte thigh support size 2XL, right       | 1        | Pc.  |
| 1-3 | 1813-1520        | 3230-2911E     | Pelotte thigh support Mini, left, picture   | 1        | Pc.  |
|     | 1813-1521        | 3230-2912E     | Pelotte thigh support Mini, right           | 1        | Pc.  |
|     | 1813-1522        | 3231-2911E     | Pelotte thigh support size 1, left          | 1        | Pc.  |
|     | 1813-1523        | 3231-2912E     | Pelotte thigh support size 1, right         | 1        | Pc.  |
|     | 1813-1524        | 3233-2911E     | Pelotte thigh support size 2, left, picture | 1        | Pc.  |
|     | 1813-1525        | 3233-2912E     | Pelotte thigh support size 2, right         | 1        | Pc.  |
|     | 1813-1526        | 3235-2911-XXX  | Pelotte thigh support size 2XL, left        | 1        | Pc.  |
|     | 1813-1527        | 3235-2912-XXX  | Pelotte thigh support size 2XL, right       | 1        | Pc.  |
| 2   | Included         | 1144-0001      | KS-Profil PVC                               | 1        | Pc.  |
| 3   | Included         | 1501-0003      | Hook strap black 25mm                       | 0,4      | m    |
| 4   | Included         | 1026-0525      | Mushroom head square neck bolt M5x25        | 2        | Pc.  |
| 5   | Included         | 1123-0500      | Wing nut PA black M5                        | 2        | Pc.  |
| 6   | Included         | 1049-0066      | BILOC-nut M5                                | 2        | Pc.  |

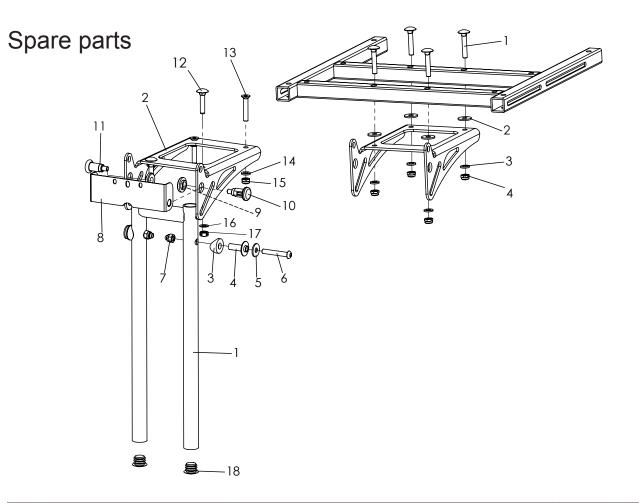


| Pos  | Medema<br>number | Article number     | Articel description                                  | Quantity | Unit |
|------|------------------|--------------------|--|----------|------|
| 1-18 | 1808-1339        | 3231-2501E-xxx     | Footresthanger 90° fix size 1                        | 1        | Pc.  |
|      | 1808-1340        | 3233-2501E-xxx     | Footresthanger 90° fix size 2                        | 1        | Pc.  |
| 1    | 1808-1341        | 3231-2520-xxx      | Bracket for footrest hanger size 1                   | 1        | Pc.  |
|      | 1808-1342        | 3233-2520-xxx      | Bracket for footrest hanger size 2                   | 1        | Pc.  |
| 2    | 1808-1343        | 3231-2411-007      | Console for footrest black anodized                  | 2        | Pc.  |
| 3    | Included         | 1111-0016          | Saddle washer PA black 20x8.4x3                      | 2        | Pc.  |
| 4    | Included         | 3231-2532          | Pivot bearing  | 2        | Pc.  |
| 5    | Included         | 1055-0006          | Washer A6,4  | 2        | Pc.  |
| 6    | Included         | 1029-0640          | button head socket screw M6x40                       | 2        | Pc.  |
| 7    | Included         | 1044-0006          | Hexagonal domed cap nut self-locking 986 M6          | 2        | Pc.  |
| 8    | Included         | 3231-2545          | Fold stop  | 1        | Pc.  |
| 9    | Included         | 3231-2544          | Nut for lock pin                                     | 2        | Pc.  |
| 10   | Included         | 1127-0009          | Lock pin with lock funktion                          | 1        | Pc.  |
| 11   | Included         | 1127-0010          | Lock pin   | 1        | Pc.  |
| 12   | Included         | 1026-0630          | Mushroom head square neck bolt M6x30                 | 2        | Pc.  |
| 13   | Included         | 1024-0635          | Hexagon socket countersunk head screw M6x35          | 2        | Pc.  |
| 14   | Included         | 1051-0006          | Washer A6,4  | 2        | Pc.  |
| 15   | Included         | 1042-0006          | Hexagon nut self-locking M6                          | 2        | Pc.  |
| 16   | Included         | 1052-0006          | Spring washer B6                                     | 2        | Pc.  |
| 17   | Included         | 1049-0068          | BILOC-nut M6   | 2        | Pc.  |
| 18   | Included         | 1141-0006          | Ribbed insert SFL 20x0,8-2,5                         | 2        | Pc.  |
|      |                  | For size 2 and 2XL | Pos. 12-13-14-15-16-17 must be replaced with 1-2-3-4 |          |      |
|      |                  |                    | See extra explosion for assembly!                    |          |      |
| 1    | Included         | 1026-0635          | Mushroom head square neck bolt M6x35                 | 4        | Pc.  |
| 2    | Included         | 1055-0006          | Washer zp A6,4                                       | 4        | Pc.  |
| 3    | Included         | 1051-0006          | Washer A6,4  | 4        | Pc.  |
| 4    | Included         | 1042-0006          | Hexagon nut self-locking M6                          | 4        | Pc.  |

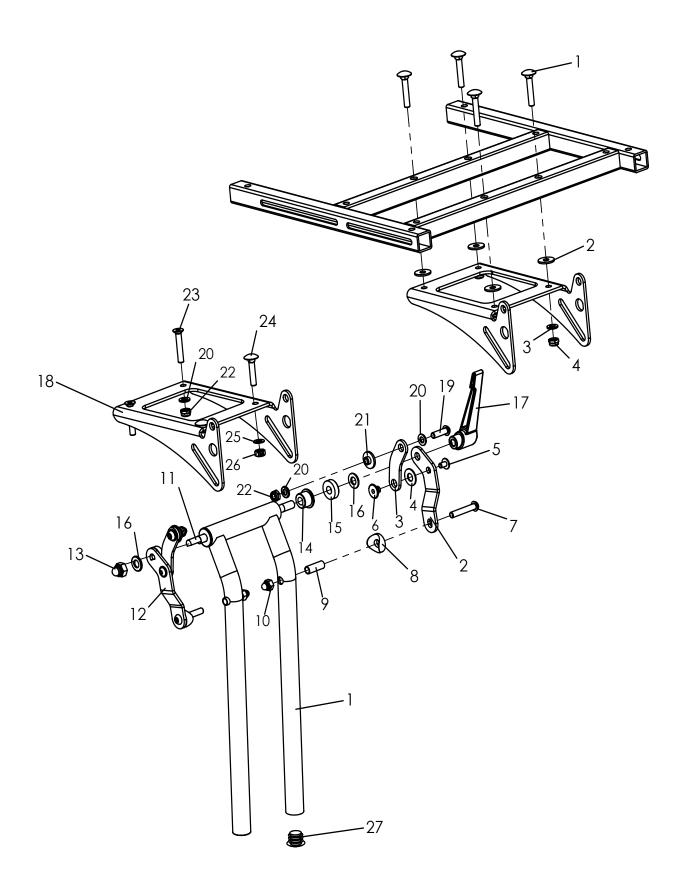


| Pos  | Medema<br>number | Article number     | Articel description                                   | Quantity | Unit |
|------|------------------|--------------------|---|----------|------|
| 1-27 | 1808-1344        | 3231-2403E-xxx     | Foot rest hanger size 1 angle adjustable 90-160°      | 1        | Pc.  |
|      | 1808-1345        | 3233-2403E-xxx     | Foot rest hanger size 2 angle adjustable 90-160°      | 1        | Pc.  |
| 1    | Included         | 3240-2870-xxx      | Brackets for footrest hanger, angle adjustable size 1 | 1        | Pc.  |
|      | Included         | 3233-2410-xxx      | Brackets for footrest hanger, angle adjustable size 2 | 1        | Pc.  |
| 2    | Included         | 3231-2434-032      | Swing lever, left                                     | 1        | Pc.  |
| 3    | Included         | 3231-2433-032      | Fold lever short                                      | 2        | Pc.  |
| 4    | Included         | 1113-0003          | Washer PA 18x8,4x1 black                              | 2        | Pc.  |
| 5    | Included         | 1030-0508          | Button head socket screw with flange M5x8             | 2        | Pc.  |
| 6    | Included         | 3231-2437          | Swing lever axel                                      | 2        | Pc.  |
| 7    | Included         | 1029-0635          | Button head socket screw M6x35                        | 2        | Pc.  |
| 8    | Included         | 1111-0016          | Saddle washer PA black 20x8.4x3                       | 2        | Pc.  |
| 9    | Included         | 1114-0012          | Bushing 8x0,8x22,5                                    | 2        | Pc.  |
| 10   | Included         | 1044-0006          | Hexagonal domed cap nut self-locking M6               | 2        | Pc.  |
| 11   | Included         | 3231-2438          | Thread rod  | 1        | Pc.  |
| 12   | Included         | 3231-2435-032      | Swing lever, right                                    | 1        | Pc.  |
| 13   | Included         | 1044-0008          | Hexagonal domed cap nut self-locking M8               | 1        | Pc.  |
| 14   | Included         | 3240-2864          | Clamp bushing   | 2        | Pc.  |
| 15   | Included         | 3231-2436          | Trust piece   | 2        | Pc.  |
| 16   | Included         | 1051-0008          | Washer A8,4   | 2        | Pc.  |
| 17   | Included         | 1121-0800          | Locking lever PA black M8                             | 1        | Pc.  |
| 18   | Included         | 3231-2411-007      | Console for footrest black anodized                   | 1        | Pc.  |
| 19   | Included         | 1029-0620          | Button head socket screw M6x20                        | 2        | Pc.  |
| 20   | Included         | 1051-0006          | Washer A6,4   | 6        | Pc.  |
| 21   | Included         | 3210-1238          | Shoulder bushing                                      | 2        | Pc.  |
| 22   | Included         | 1042-0006          | Hexagon nut self-locking M6                           | 4        | Pc.  |
| 23   | Included         | 1024-0635          | Hexagon socket countersunk head screw M6x35           | 2        | Pc.  |
| 24   | Included         | 1026-0630          | Mushroom head square neck bolt M6x30                  | 2        | Pc.  |
| 25   | Included         | 1052-0006          | Spring washer B6                                      | 2        | Pc.  |
| 26   | Included         | 1049-0068          | BILOC-nut M6  | 2        | Pc.  |
| 27   | Included         | 1141-0006          | Ribbed insert SFL 20x0,8-2,5                          | 2        | Pc.  |
|      |                  | For size 2 and 2XL | Pos. 20-22-23-24-25-26 must be replaced with 1-2-3-4  |          |      |
|      |                  |                    | See extra explosion for assembly!                     | 1        |      |
| 1    | Included         | 1026-0635          | Mushroom head square neck bolt M6x35                  | 4        | Pc.  |
| 2    | Included         | 1055-0006          | Washer zp A6,4  | 4        | Pc.  |
| 3    | Included         | 1051-0006          | Washer A6,4   | 4        | Pc.  |
| 4    | Included         | 1042-0006          | Hexagon nut self-locking M6                           | 4        | Pc.  |

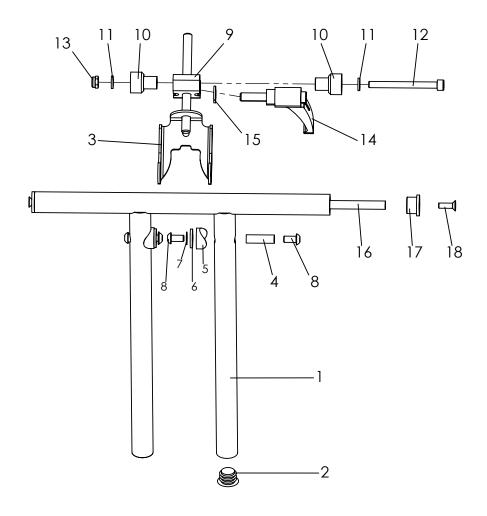


| Pos  | Medema<br>number | Article number | Articel description                                 | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-18 | 1808-1346        | 3230-2400E-032 | Foot rest hanger Mini angle adjustable              | 1        | Pc.  |
| 1    | Included         | 3230-2420-032  | Brackets for footrest hanger, angle adjustable Mini | 1        | Pc.  |
| 2    | Included         | 1141-0006      | Ribbed insert SFL 20x0,8-2,5                        | 2        | Pc.  |
| 3    | Included         | 3230-2430-033  | Suporte retainer for footrest hanger                | 1        | Pc.  |
| 4    | Included         | 1114-0062      | Threaded bushing 8xM6x26,5                          | 2        | Pc.  |
| 5    | Included         | 1111-0016      | Saddle washer PA black 20x8.4x3                     | 2        | Pc.  |
| 6    | Included         | 1113-0035      | Washer PA 18x8,4x2 black                            | 2        | Pc.  |
| 7    | Included         | 1052-0006-033  | Spring washer B6 black B6                           | 2        | Pc.  |
| 8    | Included         | 1029-0612-033  | Button head socket screw M6x12                      | 4        | Pc.  |
| 9    | Included         | 3230-2411-007  | Clamping block                                      | 1        | Pc.  |
| 10   | Included         | 3230-2413      | Axel bushing  | 2        | Pc.  |
| 11   | Included         | 1051-0006      | Washer A6,4 zp DIN/ISO 125 A6,4                     | 2        | Pc.  |
| 12   | Included         | 1021-0665      | Hexagon socket head cap screw M6x65                 | 1        | Pc.  |
| 13   | Included         | 1042-0006      | Hexagon nut self-locking M6                         | 1        | Pc.  |
| 14   | Included         | 1121-0825      | Locking lever PA black M8x25                        | 1        | Pc.  |
| 15   | Included         | 1051-0008      | Washer A8,4 zp DIN/ISO 125 A8,4                     | 1        | Pc.  |
| 16   | Included         | 3230-2441-008  | Bushing tube with M6 thread                         | 1        | Pc.  |
| 17   | Included         | 3230-2442      | Bushing for Bracket footrest hanger                 | 2        | Pc.  |
| 18   | Included         | 1024-0516      | Hexagon socket countersunk head screw M5x16         | 2        | Pc.  |

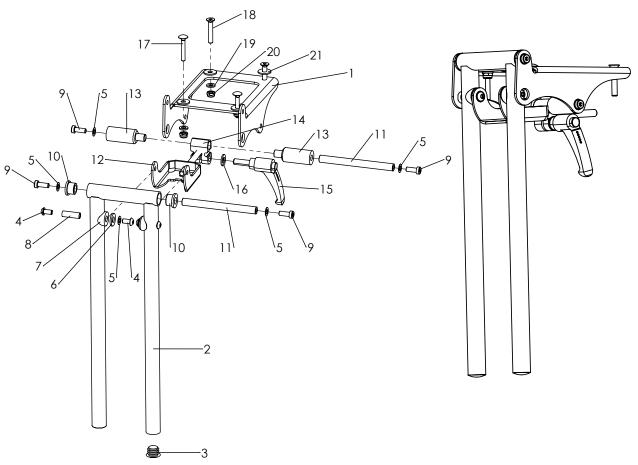


| Pos  | Medema<br>number | Article number | Articel description   | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-21 | 1808-1337        | 3232-2403E-xxx | Foot rest hanger, angle adjust. 85-175°, size 1                           | 1        | Pc.  |
|      | 1808-1338        | 3234-2403E-xxx | Foot rest hanger, angle adjust. 85-175°, size 2                           | 1        | Pc.  |
| 1    | Included         | 3232-2412-007  | Console for footrest black  | 1        | Pc.  |
| 2    | Included         | 3232-2420-xxx  | Brackets for footrest hanger, size 1                                      | 1        | Pc.  |
|      | Included         | 3234-2420-xxx  | Brackets for footrest hanger, size 2                                      | 1        | Pc.  |
| 3    | Included         | 1141-0006      | Ribbed insert SFL 20x0,8-2,5  | 2        | Pc.  |
| 4    | Included         | 1029-0612      | Button head socket screw M6x12  | 4        | Pc.  |
|      | Included         | 1029-0612-033  | Button head socket screw M6x12  | 4        | Pc.  |
| 5    | Included         | 1052-0006      | Spring washer B6  | 6        | Pc.  |
| 6    | Included         | 1113-0035      | Washer PA 18x8,4x2 black  | 2        | Pc.  |
| 7    | Included         | 1111-0016      | Saddle washer PA black 20x8.4x3   | 2        | Pc.  |
| 8    | Included         | 1114-0064      | Threaded bushing 8xM6x25,2  | 2        | Pc.  |
| 9    | Included         | 1022-0616      | Hexagon socket head cap screw M6x16                                       | 4        | Pc.  |
| 10   | Included         | 3232-2446      | Bushing for foot rest hanger  | 2        | Pc.  |
| 11   | Included         | 3232-2444      | Bushing   | 2        | Pc.  |
| 12   | Included         | 3230-2430-033  | Suporte retainer for footrest hanger                                      | 1        | Pc.  |
| 13   | Included         | 3232-2413      | Axle alves Evo  | 2        | Pc.  |
| 14   | Included         | 3230-2411-007  | Clamping block  | 1        | Pc.  |
| 15   | Included         | 1121-0825      | Locking lever PA black M8x25  | 1        | Pc.  |
| 16   | Included         | 1051-0008      | Washer A8,4   | 1        | Pc.  |
| 17   | Included         | 1026-0530      | Mushroom head square neck bolt M5x30 (front srew`s)                       | 2        | Pc.  |
| 18   | Included         | 1024-0635      | Hexagon socket countersunk head screw M6x35 (rear screw`s)                | 2        | Pc.  |
| 19   | Included         | 1051-0006      | Washer A6,4   | 4        | Pc.  |
| 20   | Included         | 1042-0006      | Hexagon nut self-locking M6   | 4        | Pc.  |
| 21   | Included         | 1055-0006      | Washer A6,4 (to be instaled between console for foot rest and seat frame) | 4        | Pc.  |

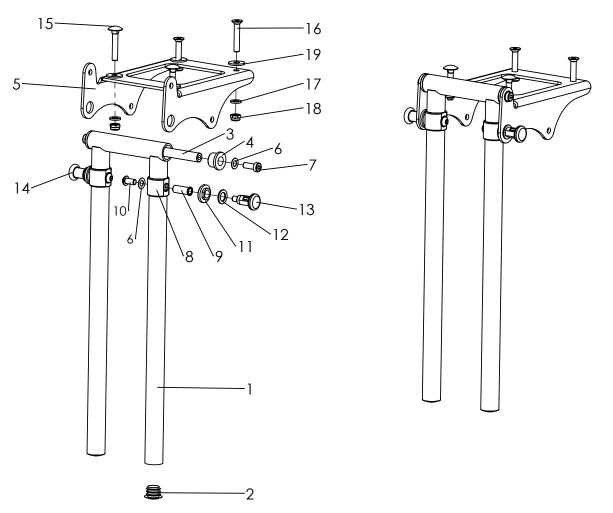


| Pos  | Medema<br>number | Article number | Articel description   | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-19 | 1808-1347        | 3232-2501E-xxx | Foot rest hanger size 1, 90° fix  | 1        | Pc.  |
|      | 1808-1348        | 3234-2501E-xxx | Foot rest hanger size 2, 90° fix  | 1        | Pc.  |
| 1    | Included         | 3232-2420-xxx  | Bracket for footrest hanger size 1  | 1        | Pc.  |
|      | Included         | 3234-2420-xxx  | Bracket for footrest hanger size 2  | 1        | Pc.  |
| 2    | Included         | 1141-0006      | Ribbed insert 20x0,8-2,5  | 2        | Pc.  |
| 3    | Included         | 3232-2444      | Axle  | 1        | Pc.  |
| 4    | Included         | 3232-2446      | Bushing for foot rest hanger  | 2        | Pc.  |
| 5    | Included         | 3232-2412-007  | Console for foot rest black anodized                                      | 1        | Pc.  |
| 6    | Included         | 1052-0006      | Spring washer B6  | 4        | Pc.  |
| 7    | Included         | 1022-0616      | Hexagon socket head cap screw M6x16                                       | 2        | Pc.  |
| 8    | Included         | 3232-2445      | Slide ring Evo  | 2        | Pc.  |
| 9    | Included         | 3232-2443      | Bolt catch  | 2        | Pc.  |
| 10   | Included         | 1029-0612      | Button head socket screw M6x12  | 2        | Pc.  |
| 11   | Included         | 3231-2544      | Nut for lock pin  | 2        | Pc.  |
| 12   | Included         | 1059-0001      | Shim ring 10x16x1,2   | 2        | Pc.  |
| 13   | Included         | 1127-0011      | Lock pin M10x1-KS with lock   | 1        | Pc.  |
| 14   | Included         | 1127-0010      | Lock pin M-10x1   | 1        | Pc.  |
| 15   | Included         | 1026-0530      | Mushroom head square neck bolt M5x30 (front srew`s)                       | 2        | Pc.  |
| 16   | Included         | 1024-0635      | Hexagon socket countersunk head screw M6x35 (rear screw`s)                | 2        | Pc.  |
| 17   | Included         | 1051-0006      | Washer A6,4   | 4        | Pc.  |
| 18   | Included         | 1042-0006      | Hexagon nut self-locking M6   | 4        | Pc.  |
| 19   | Included         | 1055-0006      | Washer A6,4 (to be instaled between console for foot rest and seat frame) | 4        | Pc.  |

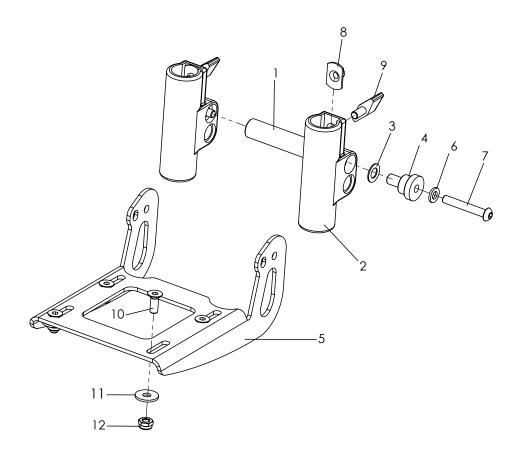


| Pos  | Medema<br>number | Article number | Articel description                               | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-11 | 1808-1313        | 3240-2901E-032 | Foot board assembly angle adjustable and foldable | 1        | Pc.  |
| 1    | 1808-1349        | 3240-2913      | Stabilizer  | 1        | Pc.  |
| 2    | 1808-1350        | 3201-2520      | Footboard holder                                  | 2        | Pc.  |
| 3    | Included         | 1052-0008      | Spring washer B8                                  | 2        | Pc.  |
| 4    | Included         | 3201-2530      | Bushing   | 2        | Pc.  |
| 5    | Included         | 3240-2911-032  | Foot rest bracket black                           | 1        | Pc.  |
| 6    | Included         | 1051-0006      | Washer A6,4                                       | 2        | Pc.  |
| 7    | Included         | 1029-0640      | Button head socket screw M6x40                    | 2        | Pc.  |
| 8    | Included         | 1049-0052      | T-nut L-769 22x14xM6x8,5                          | 2        | Pc.  |
| 9    | Included         | 1123-0612      | Wing screw Pa black M6x12 M6x BPF 20              | 2        | Pc.  |
| 10   | Included         | 1024-0616-033  | Hexagon socket countersunk head screw black M6x16 | 4        | Pc.  |
| 11   | Included         | 1055-0006      | Washer A 6,4                                      | 4        | Pc.  |
| 12   | Included         | 1042-0006      | Hexagon nut self-locking M6                       | 4        | Pc.  |

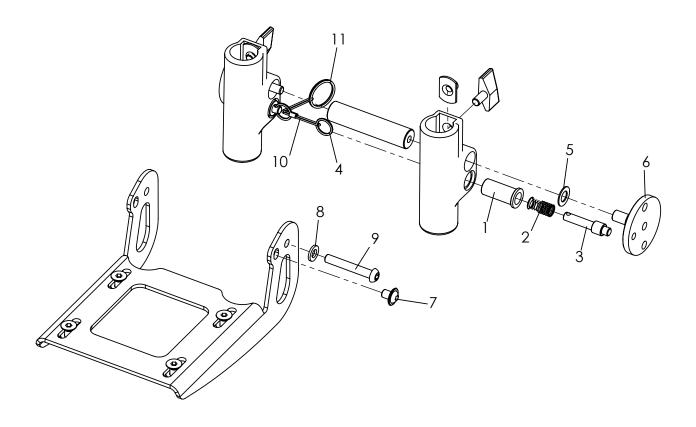


| Pos  | Medema<br>number | Article number | Articel description                      | Quantity | Unit |
|------|------------------|----------------|--|----------|------|
| 1-11 | 1403-1144        | 3240-2950E-032 | Footrest flip up lock                    | 1        | Pc.  |
| 1    | Included         | 3201-7520      | Spring housing                           | 2        | Pc.  |
| 2    | Included         | 1131-0036      | Compression spring D-127d 0,8x8,3x29     | 2        | Pc.  |
| 3    | Included         | 3201-7530      | Lock pin                                 | 2        | Pc.  |
| 4    | Included         | 1129-0036      | Key ring 16x1,6                          | 2        | Pc.  |
| 5    | Included         | 1052-0008      | Spring washer B8                         | 2        | Pc.  |
| 6    | Included         | 3201-7510      | Lock plate                               | 2        | Pc.  |
| 7    | Included         | 1030-0608      | Button head socket screw with flang M6x8 | 2        | Pc.  |
| 8    | Included         | 1051-0006      | Washer A6,4                              | 2        | Pc.  |
| 9    | Included         | 1029-0640      | Button head socket screw M6x40           | 2        | Pc.  |
| 10   | Included         | 3201-7560      | Wire rope for foot rest lock, EVO        | 0,2      | m    |
| 11   | Included         | 1129-0003      | Key ring 24x1,5                          | 1        | Pc.  |

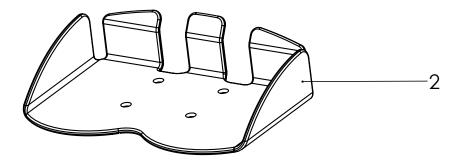


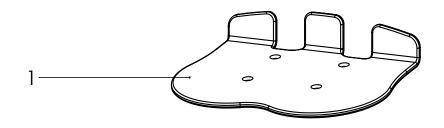


| Pos | Medema<br>number | Article number | Articel description                                  | Quantity | Unit |
|-----|------------------|----------------|--|----------|------|
| 1   | 1808-1351        | 3231-2491      | Evo foot rest with heel stop, width 27cm             | 1        | Pc.  |
|     | 1808-1318        | 3231-2492      | Evo foot rest with heel stop, width 29cm             | 1        | Pc.  |
|     | 1808-1297        | 3231-2493      | Evo foot rest with heel stop, width 31cm             | 1        | Pc.  |
|     | 1808-1352        | 3231-2494      | Evo foot rest with heel stop, width 33cm             | 1        | Pc.  |
| 2   | 1808-1311        | 3231-2551E     | Bingo-Evo foot plate in boat form size 1, width 30cm | 1        | Pc.  |
|     | 1808-1299        | 3233-2551E     | Bingo-Evo foot plate in boat form size 2, width 35cm | 1        | Pc.  |

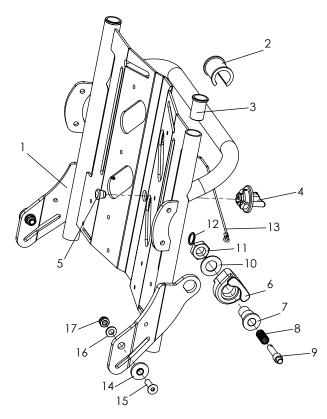


| Pos  | Medema<br>number | Article number | Articel description  | Quantity | Unit |
|------|------------------|----------------|--|----------|------|
| 1-17 | 1807-1300        | 3231-2300E-xxx | Back base size 1 complete  | 1        | Pc.  |
|      | 1807-1307        | 3233-2300E-xxx | Back base size 2 complete  | 1        | Pc.  |
|      | 1807-1308        | 3235-2300E-xxx | Back base size 2XL complete  | 2        | Pc.  |
| 1    | 1818-1146        | 3231-2310-xxx  | Back base size 1   | 1        | Pc.  |
|      | 1818-1147        | 3233-2310-xxx  | Back base size 2   | 1        | Pc.  |
|      | 1818-1148        | 3235-2310-xxx  | Back base size 2XL   | 2        | Pc.  |
| 2    | Included         | 3231-2324      | Balance bushing  | 1        | Pc.  |
| 3    | Included         | 3231-2323      | Guiding bush   | 2        | Pc.  |
| 4    | Included         | 3240-2441      | Turning knob   | 1        | Pc.  |
| 5    | Included         | 1141-0028      | Decorative plug  | 1        | Pc.  |
| 6    | Included         | 3231-2322      | Back guidance  | 2        | Pc.  |
| 7    | Included         | 3231-2321      | Locking bolt housing   | 2        | Pc.  |
| 8    | Included         | 1131-0036      | Compression spring D-127d 0,8x8,3x29                                     | 2        | Pc.  |
| 9    | Included         | 3201-7530      | Lock pin   | 2        | Pc.  |
| 10   | Included         | 1052-0012      | Spring washer B12  | 2        | Pc.  |
| 11   | Included         | 1040-0012      | Hexagon nut M12(fine pitch thread)                                       | 2        | Pc.  |
| 12   | Included         | 1129-0036      | Key ring 16x1,6  | 2        | Pc.  |
| 13   | Included         | 3231-2325      | Release cable for backrest recline size 1                                | 1        | Pc.  |
|      | Included         | 3233-2325      | Release cable for back angle adjustment size 2                           | 1        | Pc.  |
| 14   | Included         | 3231-2131      | Bushing  | 2        | Pc.  |
| 15   | Included         | 1024-0620      | Hexagon socket countersunk head screw M6x20, with frame colour 024       | 2        | Pc.  |
|      | Included         | 1024-0620-033  | Hexagon socket countersunk head screw black M6x20, with frame colour 032 | 2        | Pc.  |
| 16   | Included         | 1051-0006      | Washer A6,4  | 2        | Pc.  |
| 17   | Included         | 1042-0006      | Hexagon nut self-locking M6  | 2        | Pc.  |
| 18   | Included         | 1089-0007      | Cable ties   | 1        | Pc.  |
| 19   | Included         | 1119-0003      | Wire clamp   | 1        | Pc.  |
| 20   | Included         | 1062-0410      | Blind rivet F4x10  | 1        | Pc.  |

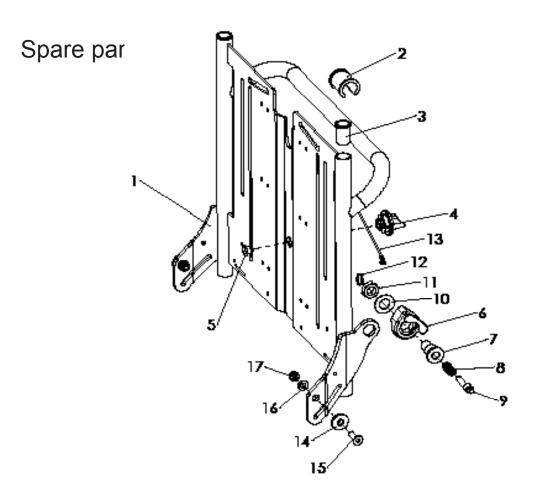

| Pos  | Medema<br>number | Article number | Articel description  | Quantity | Unit |
|------|------------------|----------------|--|----------|------|
| 1-17 | 1818-1149        | 3231-3300E-xxx | Back base OT size 1 complete   | 1        | Pc.  |
|      | 1818-1150        | 3233-3300E-xxx | Back base size OT 2 complete   | 1        | Pc.  |
| 1    | 1818-1151        | 3231-3310-xxx  | Back base size 1   | 1        | Pc.  |
|      | 1818-1152        | 3233-3310-xxx  | Back base size 2   | 1        | Pc.  |
| 2    | Included         | 3231-2324      | Balance bushing  | 1        | Pc.  |
| 3    | Included         | 3231-2323      | Guiding bush   | 2        | Pc.  |
| 4    | Included         | 3240-2441      | Turning knob   | 1        | Pc.  |
| 5    | Included         | 1141-0028      | Decorative plug  | 1        | Pc.  |
| 6    | Included         | 3231-2322      | Back guidance  | 2        | Pc.  |
| 7    | Included         | 3231-2321      | Locking bolt housing   | 2        | Pc.  |
| 8    | Included         | 1131-0036      | Compression spring D-127d 0,8x8,3x29                                     | 2        | Pc.  |
| 9    | Included         | 3201-7530      | Lock pin   | 2        | Pc.  |
| 10   | Included         | 1052-0012      | Spring washer B12  | 2        | Pc.  |
| 11   | Included         | 1040-0012      | Hexagon nut M12(fine pitch thread)                                       | 2        | Pc.  |
| 12   | Included         | 1129-0036      | Key ring 16x1,6  | 2        | Pc.  |
| 13   | Included         | 3231-2325      | Release cable for backrest recline size 1                                | 1        | Pc.  |
|      | Included         | 3233-2325      | Release cable for back angle adjustment size 2                           | 1        | Pc.  |
| 14   | Included         | 3231-2131      | Bushing  | 2        | Pc.  |
| 15   | Included         | 1024-0620      | Hexagon socket countersunk head screw M6x20, with frame colour 024       | 2        | Pc.  |
|      | Included         | 1024-0620-033  | Hexagon socket countersunk head screw black M6x20, with frame colour 032 | 2        | Pc.  |
| 16   | Included         | 1051-0006      | Washer A6,4  | 2        | Pc.  |
| 17   | Included         | 1042-0006      | Hexagon nut self-locking M6  | 2        | Pc.  |
| 18   | Included         | 1089-0007      | Cable ties   | 1        | Pc.  |
| 19   | Included         | 1119-0003      | Wire clamp   | 1        | Pc.  |
| 20   | Included         | 1062-0410      | Blind rivet F4x10  | 1        | Pc.  |

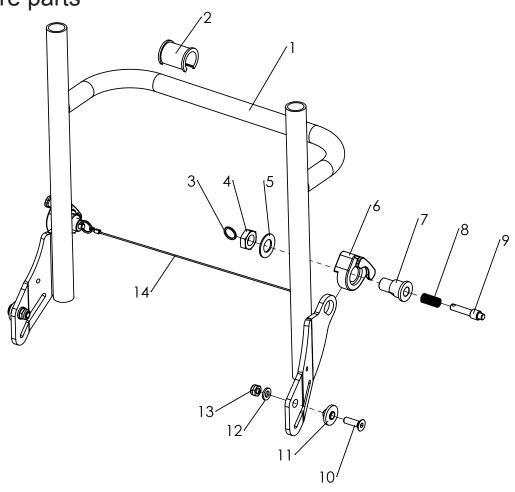


| Pos  | Medema<br>number | Article number | Articel description                         | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-14 | 1818-1153        | 3231-3600E-xxx | Backbase OT Plus size 1                     | 1        | Pc.  |
|      | 1818-1154        | 3233-3600E-xxx | Backbase OT Plus size 2                     | 1        | Pc.  |
|      | 1818-1155        | 3231-3400E-024 | Back base OT professionel size 1, silver    | 1        | Pc.  |
|      | 1818-1156        | 3231-3400E-032 | Back base OT professionel size 1, black     | 1        | Pc.  |
|      | 1818-1157        | 3233-3400E-024 | Back base OT professionel size 2, silver    | 1        | Pc.  |
|      | 1818-1158        | 3233-3400E-032 | Back base OT professionel size 2, black     | 1        | Pc.  |
| 1    | 1818-1159        | 3231-3610-xxx  | Back base OT Plus, size 1                   | 1        | Pc.  |
|      | 1818-1160        | 3233-3610-xxx  | Back base OT Plus, size 2                   | 1        | Pc.  |
|      | 1818-1161        | 3231-3410-xxx  | Back base OT professionel size 1            | 1        | Pc.  |
|      | 1818-1162        | 3233-3410-xxx  | Back base OT professionel size 2            | 1        | Pc.  |
| 2    | Included         | 3231-2324      | Balance bushing                             | 1        | Pc.  |
| 3    | Included         | 1129-0036      | Key ring 16x1,6                             | 2        | Pc.  |
| 4    | Included         | 1040-0012      | Hexagon nut M12 (fine pitch thread)         | 2        | Pc.  |
| 5    | Included         | 1052-0012      | Spring washer B12                           | 2        | Pc.  |
| 6    | Included         | 3231-2322      | Back guidance                               | 2        | Pc.  |
| 7    | Included         | 3231-2321      | Locking bolt housing                        | 2        | Pc.  |
| 8    | Included         | 1131-0036      | Compression spring D-127                    | 2        | Pc.  |
| 9    | Included         | 3201-7530      | Lock pin                                    | 2        | Pc.  |
| 10   | Included         | 1024-0620      | Hexagon socket countersunk head screw M6x20 | 2        | Pc.  |
| 11   | Included         | 3231-2131      | Bushing                                     | 2        | Pc.  |
| 12   | Included         | 1051-0006      | Washer A6,4                                 | 2        | Pc.  |
| 13   | Included         | 1042-0006      | Hexagon nut self-locking M6                 | 2        | Pc.  |
| 14   | Included         | 3231-3699      | Release cable for backrest size 1           | 1        | Pc.  |
|      | Included         | 3233-3699      | Release cable for backrest size 2           | 1        | Pc.  |

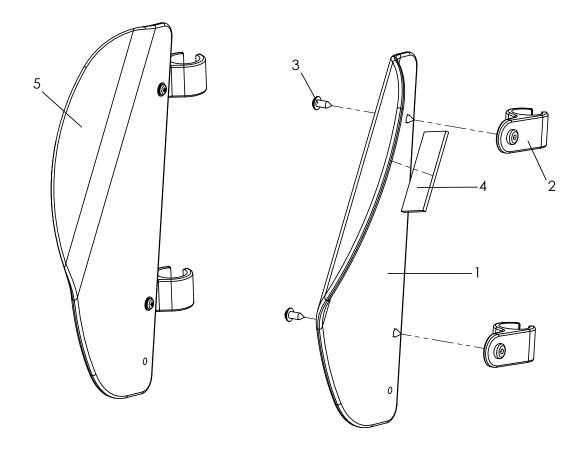

| Pos | Medema<br>number | Article number | Articel description                          | Quantity | Unit |
|-----|------------------|----------------|--|----------|------|
| 1-4 | 1818-1128        | 3231-2710E     | Lateral support left size 1 complete         | 1        | Pc.  |
|     | 1818-1129        | 3233-2710E     | Lateral support left size 2 complete         | 1        | Pc.  |
| 1   | Included         | 3201-2711      | Lateral support formed part left size 1      | 1        | Pc.  |
|     | Included         | 3203-2711      | Lateral support formed part left size 2      | 1        | Pc.  |
| 2-5 | 1818-1124        | 3231-2720E     | Lateral support right size 1 complete        | 1        | Pc.  |
|     | 1818-1125        | 3233-2720E     | Lateral support right size 2 complete        | 1        | Pc.  |
| 2   | Included         | 3231-2712      | Pad hinge                                    | 2        | Pc.  |
| 3   | Included         | 1039-0003      | Screw for plastic, pan head with flange 4x12 | 2        | Pc.  |
| 4   | Included         | 1501-0003      | Hook strap black 25mm self adhesive glue     | 0,1      | m    |
| 5   | Included         | 3201-2721      | Lateral support formed part right size 1     | 1        | Pc.  |
|     | Included         | 3203-2721      | Lateral support formed part right size 2     | 1        | Pc.  |


For Upholstery see page 52.

### Aluminum version

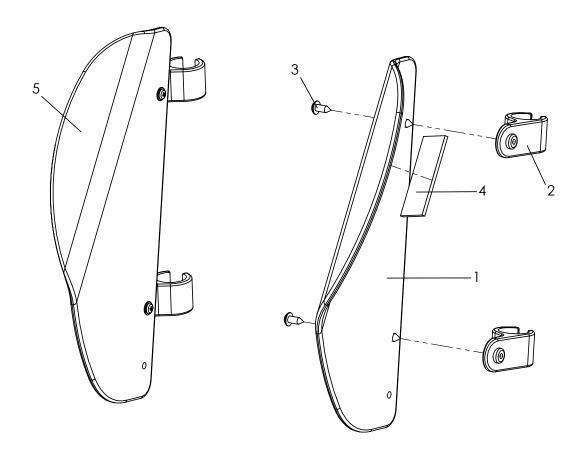


| Pos | Medema<br>number | Article number | Articel description   | Quantity | Unit |
|-----|------------------|----------------|---|----------|------|
| 1-4 | 1813-1532        | 3231-2730E     | Lateral support left size 1 complete, in alloy                          | 1        | Pc.  |
|     | 1813-1533        | 3233-2730E     | Lateral support left size 2 complete, in alloy                          | 1        | Pc.  |
|     | 1813-1534        | 3235-2710E     | Lateral support left size 2XL complete, in alloy                        | 1        | Pc.  |
| 1   | Included         | 3201-2714-032  | Lateral support formed part left size 1, in alloy                       | 1        | Pc.  |
|     | Included         | 3203-2714-032  | Lateral support formed part left size 2,, in alloy                      | 1        | Pc.  |
|     | Included         | 3235-2711-032* | Lateral support formed part left size 2XL, in alloy                     | 1        | Pc.  |
| 2-5 | 1813-1535        | 3231-2740E     | Lateral support right size 1 complete, in alloy                         | 1        | Pc.  |
|     | 1813-1536        | 3233-2740E     | Lateral support right size 2 complete, in alloy                         | 1        | Pc.  |
|     | 1813-1537        | 3235-2720E     | Lateral support right size 2XL complete, in alloy                       | 1        | Pc.  |
| 2   | Included         | 3231-2712      | Pad hinge   | 2        | Pc.  |
| 3   | Included         | 1039-0001      | Screw for plastic, pan head with flange 4x10                            | 2        | Pc.  |
| 4   | Included         | 1501-0003      | Hook strap black 25mm self adhesive glue                                | 0,1      | m    |
| 5   | Included         | 3201-2724-032  | Lateral support formed part right size 1,, in alloy                     | 1        | Pc.  |
|     | Included         | 3203-2724-032  | Lateral support formed part right size 2, in alloy                      | 1        | Pc.  |
|     | Included         | 3235-2721-032* | Lateral support formed part right size 2XL, in alloy                    | 1        | Pc.  |
|     |                  | *1039-0001     | when 2XL is needed, Screw for plastic, pan head with flange 4x10, Pos.3 | 3        | Pc.  |
|     |                  | *3231-2712     | when 2XL is needed ,Pad hinge, Pos. 2                                   | 3        | Pc.  |

For Up holstery see page 57.

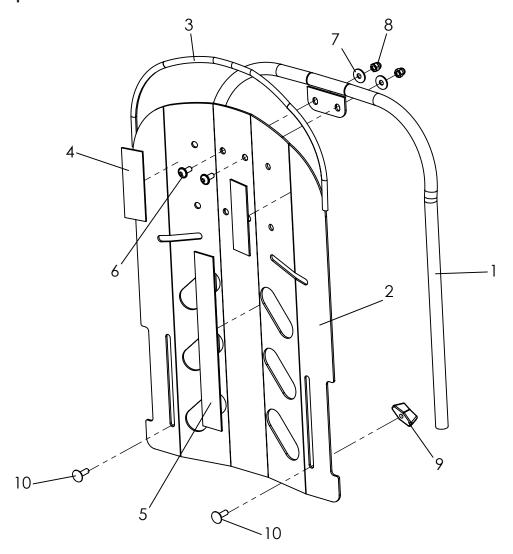


| Pos  | Medema<br>number | Article number | Articel description  | Quantity | Unit |
|------|------------------|----------------|--|----------|------|
| 1-10 | 1818-1163        | 3201-2800E-xxx | Back insert size 1, complete                                       | 1        | Pc.  |
|      | 1818-1164        | 3203-2800E-xxx | Back insert size 2 and 2XL, complete                               | 1        | Pc.  |
| 1    | Included         | 3201-2810-xxx  | Back insert size 1   | 1        | Pc.  |
|      | Included         | 3203-2810-xxx  | Back insert size 2 and 2XL   | 1        | Pc.  |
| 2-5  | Included         | 3201-2822E-xxx | Back panel size 1, with conectors                                  | 1        | Pc.  |
|      | Included         | 3203-2822E-xxx | Back panel size 2, with conectors                                  | 1        | Pc.  |
| 2    | Included         | 3201-2822-xxx  | Back panel size 1  | 1        | Pc.  |
|      | Included         | 3203-2822-xxx  | Back panel size 2  | 1        | Pc.  |
| 3    | Included         | 1144-0001      | KS-Profil PVC  | 0,5      | m    |
| 4    | Included         | 1502-0003      | Hook strap self adhesive A clue 25x200 25x200                      | 0,16     | m    |
| 5    | Included         | 1502-0003      | Hook strap self adhesive A clue 25x200 25x200                      | 0,2      | m    |
| 6    | Included         | 1030-0512      | Button head socket screw with flange M5x12                         | 2        | Pc.  |
| 7    | Included         | 1055-0005      | Washer A5,3  | 2        | Pc.  |
| 8    | Included         | 1044-0005      | Hexagonal domed cap nut self-locking M5                            | 2        | Pc.  |
| 9    | Included         | 1123-0500      | Wing nut PA black M5 M5 BPF 20                                     | 2        | Pc.  |
| 10*  | Included         | 1026-0516      | Mushroom head square neck bolt M5x16, with frame colour, 024       | 2        | Pc.  |
|      | Included         | 1026-0516-033  | Mushroom head square neck bolt black M5x16, with frame colour, 032 | 2        | Pc.  |


<sup>\*</sup> Screw`s change with frame colour!

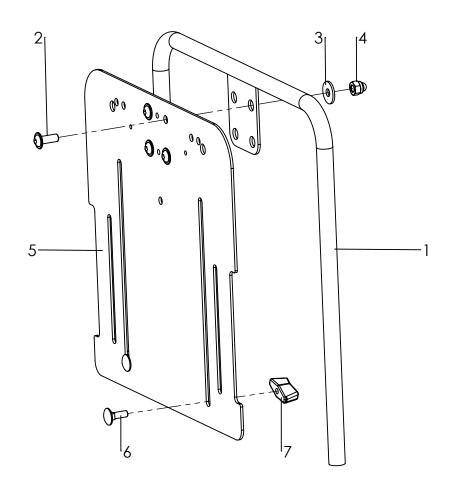

| Pos | Medema<br>number | Article number | Articel description                                       | Quantity | Unit |
|-----|------------------|----------------|---|----------|------|
| 1-7 | 1818-1165        | 3201-3800E-xxx | OT Backinsert size 1                                      | 1        | Pc.  |
|     | 1818-1166        | 3203-3800E-xxx | OT Backinsert size 2                                      | 1        | Pc.  |
| 1-4 | Included         | 3201-3810-xxx  | OT Back frame size 1 with fasteners                       | 1        | Pc.  |
|     | Included         | 3203-3810-xxx  | OT Back frame size 2 with fasteners                       | 1        | Pc.  |
| 2   | Included         | 1030-0616      | Button head socket screw with flange M6x16                | 4        | Pc.  |
| 3   | Included         | 1055-0006      | Washer A6,4   | 4        | Pc.  |
| 4   | Included         | 1044-0006      | Hexagonal domed cap nut self-locking M6                   | 4        | Pc.  |
| 5-7 | Included         | 3201-3822-xxx  | OT Back panel size 1 with fasteners                       | 1        | Pc.  |
|     | Included         | 3203-3822-xxx  | OT Back panel size 2 with fasteners                       | 1        | Pc.  |
| 6   | Included         | 1026-0516      | Mushroom head square neck bolt M5x16 with framecolour 024 | 2        | Pc.  |
|     | Included         | 1026-0516-033  | Mushroom head square neck bolt M5x16 with framecolour 032 | 2        | Pc.  |
| 7   | Included         | 1123-0500      | Wing nut PA black M5                                      | 2        | Pc.  |

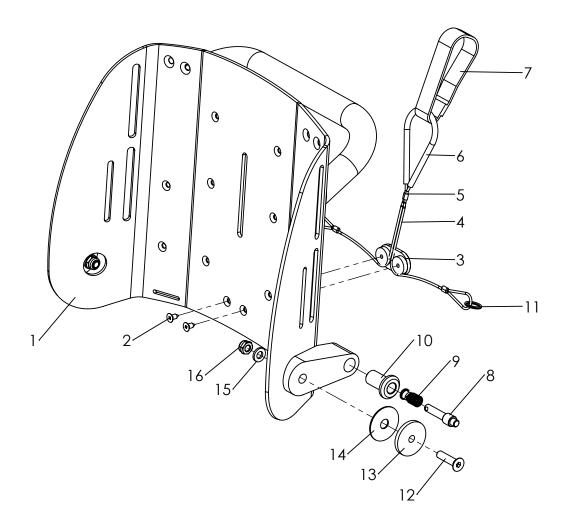

| Pos  | Medema<br>number | Article number | Articel description                             | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-16 | 1818-1167        | 3230-2301E-032 | Back base Mini, complete                        | 1        | Pc.  |
| 1    | Included         | 3230-2310-032  | Back panal Mini                                 | 1        | Pc.  |
| 2-7  | 1818-1168        | 3230-2325      | Release cable for backrest recline size Mini    | 1        | Set  |
| 2    | Included         | 1039-0004-033  | Countersunk sheet metal screw 3.9x9.5 mm        | 2        | Pc.  |
| 3    | Included         | 3230-2324      | Diverter block back adjustment                  | 1        | Pc.  |
| 4    | Included         | 3230-1129      | Feinstseil 1,2x1,8 mm PA12 natur, ca 650mm lang | 0,65     | Pc.  |
| 5    | Included         | 1089-0005      | Wire-rope 1,2x1,8 mm                            | 3        | Pc.  |
| 6    | Included         | 1143-0010      | Plastick hoseblack, 180mm long                  | 0,18     | m    |
| 7    | Included         | 1500-0014      | Back to Back on roll black 25 mm                | 0,18     | m    |
| 8    | Included         | 3201-7530      | Lock pin  | 2        | Pc.  |
| 9    | Included         | 1131-0036      | Compression spring D-127d 0,8x8,3x29            | 2        | Pc.  |
| 10   | Included         | 3230-2322      | Bolt socket                                     | 2        | Pc.  |
| 11   | Included         | 1129-0036      | key ring 16x1,6                                 | 2        | Pc.  |
| 12   | Included         | 1024-0625-033  | Hexagon socket countersunk head screw M6x25     | 2        | Pc.  |
| 13   | Included         | 3230-2321-033  | Axel bushing                                    | 2        | Pc.  |
| 14   | Included         | 1113-0065      | Washer PA nature 8,4x30x1 mm                    | 2        | Pc.  |
| 15   | Included         | 1051-0006      | Washer A6,4                                     | 2        | Pc.  |
| 16   | Included         | 1042-0006      | Hexagon nut self-locking M6                     | 2        | Pc.  |

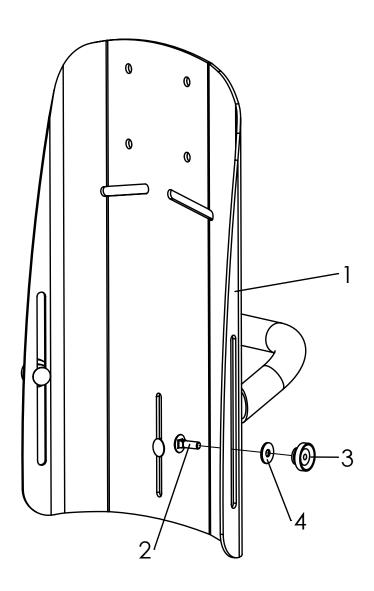


| Pos | Medema<br>number | Article number | Articel description                               | Quantity | Unit |
|-----|------------------|----------------|---|----------|------|
| 1-3 | 1818-1169        | 3230-2302E-032 | Back insert Mini, complete                        | 1        | Pc.  |
| 1   | Included         | 3230-2331-032  | Back insert Mini,                                 | 1        | Pc.  |
| 2   | Included         | 1026-0516-033  | Mushroom head square neck bolt black M5x16        | 3        | Pc.  |
| 3   | Included         | 1124-0500      | Knurled nut PA black M5                           | 3        | Pc.  |
| 4   | Included         | 1055-0005-033  | Washer A5,3                                       | 3        | Pc.  |
|     | Included         | 1502-0003      | Hock strap self adhesive, 25mm, 6 cm long, 6 St.  | 0,36     | m    |
|     | Included         | 1502-0003      | Hock strap self adhesive, 25mm, 10 cm long, 1 St. | 0,12     | m    |




| Pos | Medema<br>number | Article number | Articel description                                 | Quantity | Unit |
|-----|------------------|----------------|---|----------|------|
| 1-4 | 1821-1204        | 3230-5100E-338 | Upholstery Mini                                     | 1        | Pc.  |
|     | 1821-1161        | 3231-5100E-338 | Upholstery size 1                                   | 1        | Pc.  |
|     | 1821-1131        | 3233-5100E-338 | Upholstery size 2                                   | 1        | Pc.  |
|     | 1821-1170        | 3235-5100E-338 | Upholstery size 2XL                                 | 2        | Pc.  |
| 1   | 1821-1186        | 3230-5120E-338 | Back upholstery, Mini                               | 1        | Pc.  |
|     | 1821-1187        | 3231-5120E-338 | Back upholstery, size 1                             | 1        | Pc.  |
|     | 1821-1188        | 3233-5120E-338 | Back upholstery, size 2                             | 1        | Pc.  |
|     | 1821-1189        | 3235-5120E-338 | Back upholstery, size 2XL                           | 1        | Pc.  |
| 2   | 1818-1126        | 3201-5160E-338 | Lateral upholstery size 1                           | 1        | Pair |
|     | 1818-1127        | 3203-5160E-338 | Lateral upholstery size 2                           | 1        | Pair |
|     | 1821-1190        | 3235-5160E-338 | Lateral upholstery size 2XL                         | 1        | Pair |
| 3   | 1821-1191        | 3230-5140E-338 | Seat upholstery size Mini                           | 1        | Pc.  |
|     | 1821-1192        | 3231-5140E-338 | Seat upholstery size 1                              | 1        | Pc.  |
|     | 1821-1193        | 3234-5140-338  | Seat upholstery size 2                              | 1        | Pc.  |
|     | 1821-1194        | 3235-5140-338  | Seat upholstery size 2XL                            | 1        | Pc.  |
| 4   | 1813-1549        | 3231-9500E-338 | Side panels matching the upholstery color           | 1        | Pair |
| 5   | 1821-1196        | 3231-2920-323  | Upholstery thigh support size 1 and Mini, iron grey | 1        | Pair |
|     | 1821-1197        | 3233-2920-323  | Upholstery thigh support size 2, iron grey          | 1        | Pair |
|     | 1821-1198        | 3235-2920-323  | Upholstery thigh support size 2XL, iron grey        | 1        | Pair |
| 6   | 1821-1195        | 3230-5150-338  | Hip upholstery Mini                                 | 1        | Pair |
|     | 1821-1153        | 3201-5150E-338 | Hip upholstery size 1                               | 1        | Pair |
|     | 1821-1157        | 3203-5150E-338 | Hip upholstery size 2                               | 1        | Pair |
|     | 1821-1174        | 3235-5150E-338 | Hip upholstery size 2XL                             | 1        | Pair |

### With order replace, xxx with colour code!


| 338             | 339                  | 340             | 344                      | 345            |
|-----------------|----------------------|-----------------|--------------------------|----------------|
| sand / irongrey | turquoise / irongrey | pink / irongrey | black diamond / irongrey | red / irongrey |

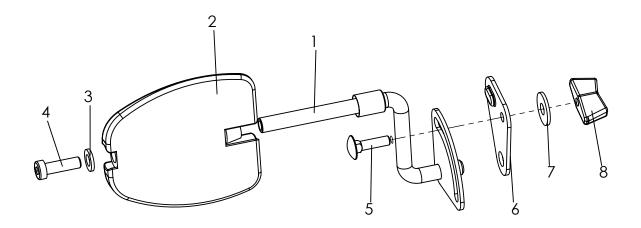


| Pos | Medema<br>number | Article number | Articel description                     | Quantity | Unit |
|-----|------------------|----------------|---|----------|------|
| 1-8 | 1813-1538        | 3231-7700      | Trunk suport size 1, without upholstery | 1        | Pair |
|     | 1813-1539        | 3233-7700      | Trunk suport size 2, without upholstery | 1        | Pair |
|     | 1813-1540        | 3235-7700      | Trunk suport size 3, without upholstery | 1        | Pair |
| 1   | Included         | 3201-7720-008  | Pad crank right size 1                  | 1        | Pc.  |
|     | Included         | 3201-7710-008  | Pad crank left size 1                   | 1        | Pc.  |
| 2   | Included         | 3231-7730      | Pad size 1                              | 2        | Pc.  |
|     | Included         | 3233-7730      | Pad size 2                              | 2        | Pc.  |
|     | Included         | 3235-7730      | Pad size 3                              | 2        | Pc.  |
| 3   | Included         | 1051-0006      | Washer A6,4                             | 2        | Pc.  |
| 4   | Included         | 1022-0620      | Hexagon socket head cap screw M6x20     | 2        | Pc.  |
|     | Included         | 1022-0640      | Cylinder head screw M6x40               | 2        | Pc.  |
|     | Included         | 1022-0650      | Hexagon socket head cap screw M6x50     | 2        | Pc.  |
| 5   | Included         | 1026-0520      | Mushroom head square neck bolt M5x20    | 2        | Pc.  |
| 6   | Included         | 3201-7717-008  | Bracket of trunk support right          | 1        | Pc.  |
|     | Included         | 3201-7716-008  | Bracket of trunk support left           | 1        | Pc.  |
| 7   | Included         | 1055-0006      | Washer A6,4                             | 2        | Pc.  |
| 8   | Included         | 1123-0500      | Wing nut PA black M5                    | 2        | Pc.  |
|     | 1821-1199        | 3231-7760-323  | Upholstery Gr. 1, iron grey             | 2        | Pc.  |
|     | 1821-1200        | 3233-7760-323  | Upholstery Gr. 2 iron grey              | 2        | Pc.  |
|     | 1821-1201        | 3203-7760-323  | Upholstery Gr. 3 iron grey              | 2        | Pc.  |

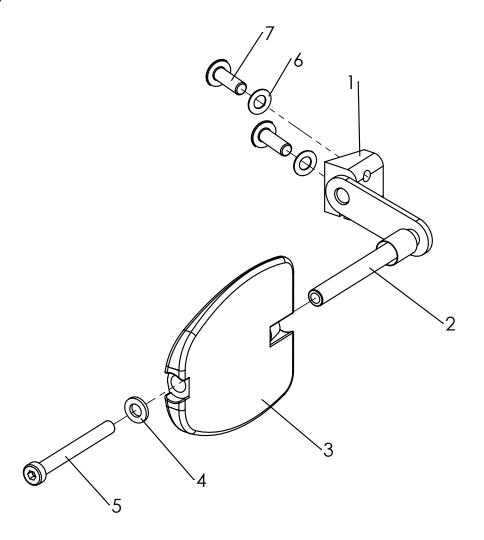


| Pos | Medema<br>number | Article number | Articel description                          | Quantity | Unit |
|-----|------------------|----------------|--|----------|------|
| 1-7 | 1813-1541        | 3230-7700      | Trunk support, size Mini with out upholstery | 1        | Pair |
| 1   | Included         | 3230-7721-007  | Clamp base Mini                              | 2        | Pc.  |
| 2   | Included         | 3230-7710      | Swivel lever Thoraxpelotte Mini              | 2        | Pc.  |
| 3   | Included         | 3231-7730      | Pad size 1                                   | 2        | Pc.  |
| 4   | Included         | 1051-0006      | Washer A6,4                                  | 2        | Pc.  |
| 5   | Included         | 1022-0620      | Hexagon socket head cap screw M6x20          | 2        | Pc.  |
| 6   | Included         | 1052-0006      | Spring washer B6                             | 4        | Pc.  |
| 7   | Included         | 1030-0620      | Button head socket screw with flange M6x20   | 4        | Pc.  |

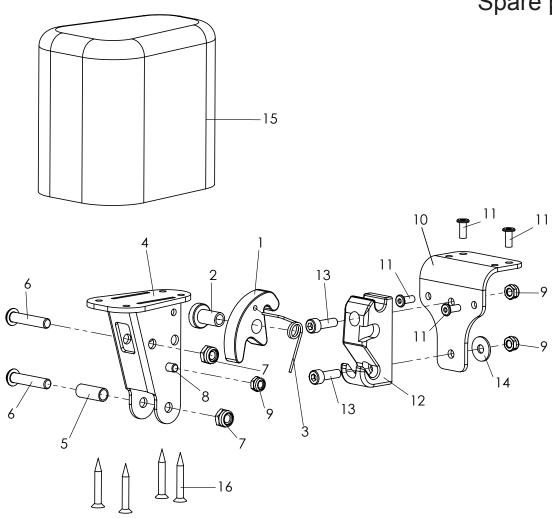


| Pos  | Medema<br>number | Article number | Articel description                                 | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-16 |                  |                |   | 1        | Pc.  |
|      | 1813-1376        | 3231-7802-323  | Abduction block size 2 swing&detachable, with cover | 1        | Pc.  |
|      | 1813-1377        | 3231-7902-323  | Abduction block size 3 swing&detachable, with cover | 1        | Pc.  |
| 1    | Included         | 3231-7841      | Arresting clamp                                     | 1        | Pc.  |
| 2    | Included         | 3231-7842      | Bearing arresting clamp                             | 1        | Pc.  |
| 3    | Included         | 1133-0020      | Leg spring 1,25 x 9,25 x 2,25                       | 1        | Pc.  |
| 4    | Included         | 3231-7820-032  | A-wedge support                                     | 1        | Pc.  |
| 5    | Included         | 1114-0013      | Bushing zp 8x0,8x20                                 | 1        | Pc.  |
| 6    | Included         | 1029-0630      | Button head socket screw M6x30                      | 2        | Pc.  |
| 7    | Included         | 1042-0006      | Hexagon nut self-locking 985 M6                     | 2        | Pc.  |
| 8    | Included         | 1021-0508      | Cylinder head screw M5x8                            | 1        | Pc.  |
| 9    | Included         | 1042-0005      | Hexagon nut self-locking M5                         | 3        | Pc.  |
| 10   | Included         | 3231-7831-032  | Base plate abduction block receiver                 | 1        | Pc.  |
| 11   | Included         | 1062-0410      | Blind rivet F4x10                                   | 4        | Pc.  |
| 12   | Included         | 3231-7832      | Arresting part for A-wedge                          | 1        | Pc.  |
| 13   | Included         | 1021-0516      | Hexagon socket head cap screw M5x16                 | 2        | Pc.  |
| 14   | Included         | 1055-0005      | Washer A5,3   | 1        | Pc.  |
| 15   | Included         | 3201-7810E     | Wood wedge size 1 with PU-foam                      | 1        | Pc.  |
|      | Included         | 3201-7910E     | Wood wedge size 1 with PU-foam                      | 1        | Pc.  |
|      | Included         | 3203-7810E     | Wood wedge size 1 with PU-foam                      | 1        | Pc.  |
| 16   | Included         | 1028-0430      | Pressboard screw countersunk head 4x30              | 4        | Pc.  |
|      | 1822-1171        | 3201-7840-323  | Cover for abduction block size 1 iron grey          | 1        | Pc.  |
|      | 1822-1172        | 3201-7940-323  | Cover abduction block size 2 iron grey              | 1        | Pc.  |
|      | 1822-1173        | 3203-7840-323  | Cover for abduction block size 3 iron grey          | 1        | Pc.  |

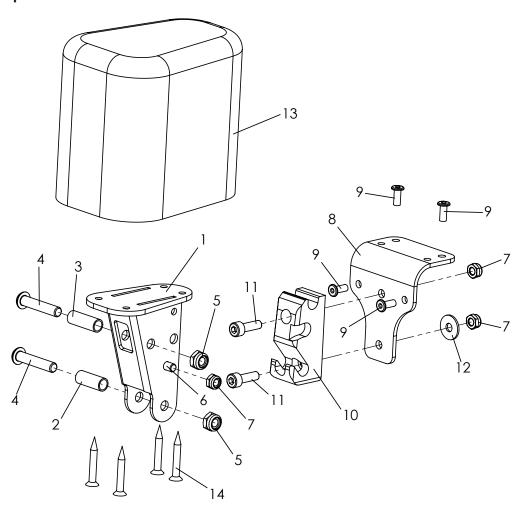


| Pos  | Medema<br>number | Article number | Articel description                                    | Quantity | Unit |
|------|------------------|----------------|--|----------|------|
| 1-14 | 1813-1545        | 3231-7801      | Bingo-Evo abduction block size 1 fixed, with out cover | 1        | Рс   |
|      | 1813-1546        | 3231-7901      | Bingo-Evo abduction block size 2 fixed, with out cover | 1        | Рс   |
|      | 1813-1547        | 3233-7801      | Bingo-Evo abduction block size 3 fixed, with out cover | 1        | Рс   |
| 1    | Included         | 3231-7820-032  | A-wedge support  | 1        | Рс   |
| 2    | Included         | 1114-0013      | Bushing zp 8x0,8x20                                    | 1        | Рс   |
| 3    | Included         | 1114-0012      | Bushing zp 8x0,8x22,5                                  | 1        | Рс   |
| 4    | Included         | 1029-0630      | Button head socket screw M6x30                         | 2        | Рс   |
| 5    | Included         | 1042-0006      | Hexagon nut self-locking M6                            | 2        | Рс   |
| 6    | Included         | 1021-0508      | Cylinder head screw M5x8                               | 1        | Рс   |
| 7    | Included         | 1042-0005      | Hexagon nut self-locking M5                            | 3        | Рс   |
| 8    | Included         | 3231-7831-032  | Base plate abduction block receiver                    | 1        | Рс   |
| 9    | Included         | 1062-0410      | Blind rivet F4x10                                      | 4        | Рс   |
| 10   | Included         | 3231-7832      | Arresting part for A-wedge                             | 1        | Рс   |
| 11   | Included         | 1021-0516      | Hexagon socket head cap screw M5x16                    | 2        | Рс   |
| 12   | Included         | 1055-0005      | Washer A5,3  | 1        | Рс   |
| 13   | Included         | 3201-7810E     | Wood wedge size 1 with PU-foam                         | 1        | Рс   |
|      | Included         | 3201-7910E     | Wood wedge size 2 with PU-foam                         | 1        | Рс   |
|      | Included         | 3203-7810E     | Wood wedge size 3 with PU-foam                         | 1        | Рс   |
| 14   | Included         | 1028-0430      | Pressboard screw countersunk head 4x30                 | 4        | Рс   |
|      | 1822-1176        | 3201-7840-323  | Cover for abduction block size 1 iron grey             | 1        | Рс   |
|      | 1822-1177        | 3201-7940-323  | Cover abduction block size 2 iron grey                 | 1        | Рс   |
|      | 1822-1178        | 3203-7840-323  | Cover for abduction block size 3 iron grey             | 1        | Рс   |

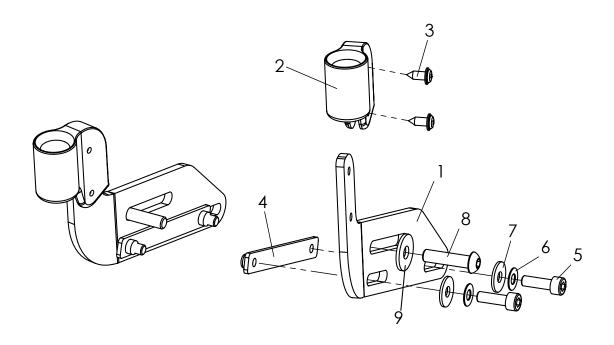


| Pos | Medema<br>number | Article number | Articel description                                      | Quantity | Unit |
|-----|------------------|----------------|--|----------|------|
| 1-9 | 1806-1255        | 3232-2203-032  | Retainer for accessories grab rail or therapy tray       | 1        | Pair |
| 1   | 1806-1333        | 3232-2232-032  | Holder for accessories retainer left side                | 1        | PC.  |
|     | 1806-1334        | 3232-2242-032  | Holder for accessories retainer right side               | 1        | PC.  |
| 2   | 1806-1335        | 3231-2232      | Retainer for accessories mould part                      | 2        | PC.  |
| 3   | Included         | 1039-0001      | Screw for plastic, cross rcessed pan head w. flange 4x10 | 2        | PC.  |
| 4   | Included         | 3232-2235      | Threaded plate black galvanized                          | 2        | PC.  |
| 5   | Included         | 1021-0516-033  | Hexagon socket head cap screw black M5x16                | 4        | PC.  |
| 6   | Included         | 1052-0005-033  | Spring washer B5 black                                   | 4        | PC.  |
| 7   | Included         | 1055-0005-033  | Washer A5,3 black  | 4        | PC.  |
| 8   | Included         | 1029-0625-033  | Button head socket screw black M6x25                     | 2        | PC.  |
| 9   | Included         | 1055-0006-033  | Washer A6,4  | 2        | PC.  |

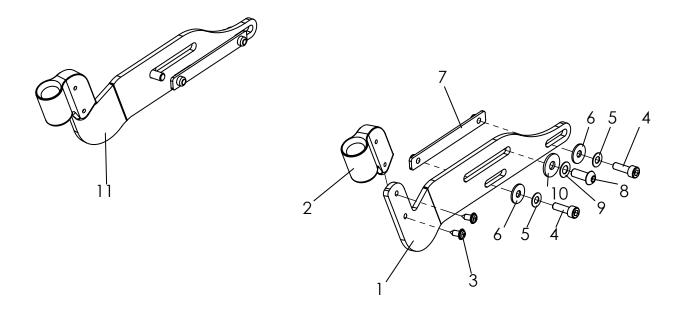


| Pos  | Medema<br>number | Article number | Articel description                                      | Quantity | Unit |
|------|------------------|----------------|--|----------|------|
| 1-10 | 1806-1336        | 3230-2201      | Retainer Mini, for accessories grab rail or therapy tray | 1        | Set  |
| 1    | 1806-1337        | 3230-2232      | Holder for accessories retainer Mini, left side          | 1        | Pc.  |
|      | 1806-1338        | 3230-2242      | Holder for accessories retainer Mini, right side         | 1        | Pc.  |
| 2    | 1806-1339        | 3231-2232      | Retainer for accessories mould part                      | 2        | Pc.  |
| 3    | Included         | 1039-0001      | Screw for plastic, cross rcessed pan head w. flange 4x10 | 4        | Pc.  |
| 4    | Included         | 1021-0516-033  | Hexagon socket head cap screw black M5x16                | 4        | Pc.  |
| 5    | Included         | 1052-0005-033  | Spring washer B5   | 4        | Pc.  |
| 6    | Included         | 1055-0005-033  | Washer A5,3  | 4        | Pc.  |
| 7    | Included         | 3232-2235      | Threaded plate black galvanized                          | 2        | Pc.  |
| 8    | Included         | 1029-0625-033  | button head socket screw black M6x25                     | 2        | St   |
| 9    | Included         | 1052-0006-033  | Spring washer B6 black                                   | 2        | St   |
| 10   | Included         | 1055-0006-033  | Washer A6,4 black  | 2        | St   |

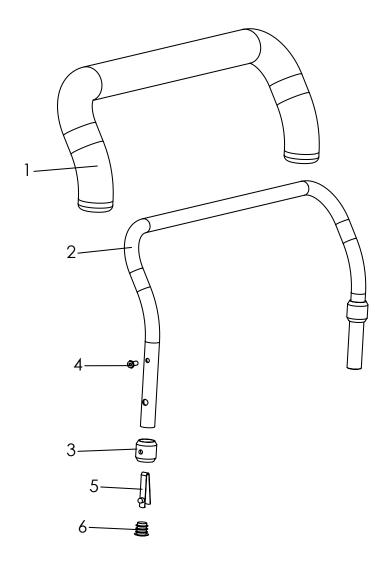


| Pos | Medema<br>number | Article number | Articel description                      | Quantity | Unit |
|-----|------------------|----------------|--|----------|------|
| 1-6 | 1806-1340        | 3230-8100-323  | Grab rail with upholstery Mini           | 1        | Pc.  |
|     | 1806-1256        | 3231-8100-323  | Grab rail with upholstery size 1         | 1        | Pc.  |
|     | 1806-1257        | 3233-8100-323  | Grab rail with upholstery size 2         | 1        | Pc.  |
| 1   | 1806-1343        | 3231-8110      | Bend Tube for grab rail size 1           | 1        | Pc.  |
|     | 1806-1344        | 3233-8110      | Bend Tube for grab rail size 2           | 1        | Pc.  |
| 2   | 1821-1202        | 3231-8130      | Upholstery foam material size 1, (60 cm) | 1        | Pc.  |
|     | 1821-1203        | 3203-8130      | Upholstery foam material size 2 (70 cm)  | 1        | Pc.  |
| 3   | Included         | 3201-8120      | Bushing                                  | 2        | Pc.  |
| 4   | Included         | 1062-0408      | Blind rivet St/St DIN/ISO 7337 F4x8      | 2        | Pc.  |
| 5   | Included         | 1134-0005      | Snap button A 102                        | 2        | Pc.  |
| 6   | Included         | 1141-0001      | Ribbed insert SFL 14x0,8-2               | 2        | Pc.  |
|     | 1822-1181        | 3230-8140-323  | Cover of grab rail Mini, iron grey       | 1        | Pc.  |
|     | 1822-1182        | 3231-8140-323  | Cover of grab rail size 1, iron grey     | 1        | Pc.  |
|     | 1822-1183        | 3233-8140-323  | Cover of grab rail size 2, iron grey     | 1        | Pc.  |

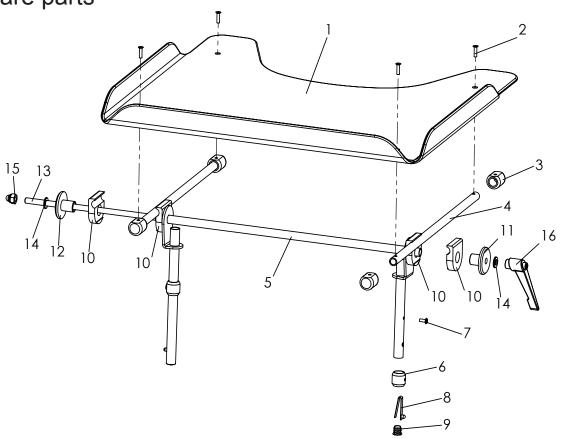


| Pos  | Medema<br>number | Article number | Articel description                     | Quantity | Unit |
|------|------------------|----------------|---|----------|------|
| 1-16 | 1814-1448        | 3231-8200      | Therapy tray size 1, clear              | 1        | Pc.  |
| 1-16 | 1814-1451        | 3233-8200      | Therapy tray size 2, clear              | 1        | Pc.  |
| 1-16 | 1814-1458        | 3230-8200      | Therapy tray Mini, clear                | 1        | Pc.  |
| 1    | Included         | 3231-8290      | Tray plate only, clear                  | 1        | Pc.  |
|      | Included         | 3231-8280      | Tray plate clear, with rails            | 1        | Pc.  |
|      | 1814-1446        | 3231-8201      | Therapy tray size 1, gray               | 1        | Pc.  |
|      | 1814-1449        | 3233-8201      | Therapy tray size 2, gray               | 1        | Pc.  |
|      | 1814-1459        | 3230-8201      | Therapy tray Mini, gray                 | 1        | Pc.  |
|      | Included         | 3231-8291      | Tray plate only, grey                   | 1        | Pc.  |
|      | Included         | 3231-8281      | Tray plate gray, with rails             | 1        | Pc.  |
| 2    | Included         | 1063-0410      | Blind rivet S4x10                       | 4        | Pc.  |
| 3    | Included         | 3201-8280      | Bearing block                           | 4        | Pc.  |
| 4    | Included         | 3201-8270      | Rail                                    | 2        | Pc.  |
| 5-9  | Included         | 3231-8210E-032 | Table base frame, size 1                | 1        | Pc.  |
|      | Included         | 3233-8210E-032 | Table base frame, size 2                | 1        | Pc.  |
|      | Included         | 3230-8210E-032 | Table base frame, size 3                | 1        | Pc.  |
| 6    | Included         | 3201-8120      | Bushing                                 | 2        | Pc.  |
| 7    | Included         | 1062-0410      | Blind rivet F4x10                       | 2        | Pc.  |
| 8    | Included         | 1134-0005      | Snap button A 102                       | 2        | Pc.  |
| 9    | Included         | 1141-0001      | Ribbed insert SFL 14x0,8-2              | 2        | Pc.  |
| 10   | Included         | 3201-8230      | Clamp half                              | 4        | Pc.  |
| 11   | Included         | 3201-8240      | Clamp disk D 8,5                        | 1        | Pc.  |
| 12   | Included         | 3201-8250      | Clamp disk M8                           | 1        | Pc.  |
| 13   | Included         | 1039-0002      | Thread M8x47,2mm                        | 0,47     | m    |
| 14   | Included         | 1051-0008      | Washer A8,4                             | 2        | Pc.  |
| 15   | Included         | 1044-0008      | Hexagonal domed cap nut self-locking M8 | 1        | Pc.  |
| 16   | Included         | 1121-0800      | Locking lever PA black M8               | 1        | Pc.  |

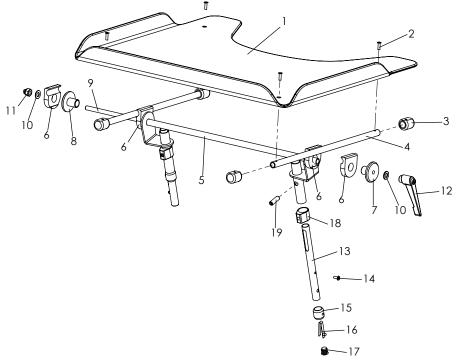


| Pos   | Medema<br>number | Article number | Articel description                           | Quantity | Unit |
|-------|------------------|----------------|---|----------|------|
| 1-16  | 1814-1460        | 3230-8202      | Therapy tray Mini, clear, height adjustable   | 1        | Pc.  |
|       | 1814-1447        | 3231-8202      | Therapy tray size 1, clear, height adjustable | 1        | Pc.  |
|       | 1814-1450        | 3233-8202      | Therapy tray size 2, clear, height adjustable | 1        | Pc.  |
| 1     | Included         | 3231-8290      | Tray plate only, clear                        | 1        | Pc.  |
| 1-4   | Included         | 3231-8280      | Tray plate clear, with rails                  | 1        | Pc.  |
|       | 1814-1461        | 3230-8203      | Therapy tray Mini, gray, height adjustable    | 1        | Pc.  |
|       | 1814-1445        | 3231-8203      | Therapy tray size 1, gray, height adjustable  | 1        | Pc.  |
|       | 1814-1453        | 3233-8203      | Therapy tray size 2, gray, height adjustable  | 1        | Pc.  |
|       | Included         | 3231-8291      | Tray plate only, grey                         | 1        | Pc.  |
| 1-4   | Included         | 3231-8281      | Tray plate gray, with rails                   | 1        | Pc.  |
| 2     | Included         | 1063-0416      | Blind rivet S4x10                             | 4        | Pc.  |
| 3     | Included         | 3201-8280      | Bearing block                                 | 4        | Pc.  |
| 4     | Included         | 3201-8270-016  | Rail  | 2        | Pc.  |
| 5     | Included         | 3230-8220-032  | Table base frame, Mini, height adjustable     | 1        | Pc.  |
|       | Included         | 3231-8220-032  | Table base frame, size 1, height adjustable   | 1        | Pc.  |
|       | Included         | 3233-8220-032  | Table base frame, size 2, height adjustable   | 1        | Pc.  |
| 6     | Included         | 3201-8230      | Clamp half                                    | 4        | Pc.  |
| 7     | Included         | 3201-8250      | Clamp disk M8                                 | 1        | Pc.  |
| 8     | Included         | 3201-8240      | Clamp disk D 8,5                              | 1        | Pc.  |
| 9     | Included         | 1039-0002      | Thread rod M8x1000                            | 0,47     | m    |
| 10    | Included         | 1051-0008      | Washer A8,4                                   | 2        | Pc.  |
| 11    | Included         | 1044-0008      | Hexagonal domed cap nut self-locking M8       | 1        | Pc.  |
| 12    | Included         | 1121-0800      | Locking lever PA black M8                     | 1        | Pc.  |
| 13-17 | Included         | 3231-8215E     | Support tube, adjustablet left                | 1        | Pc.  |
|       | Included         | 3231-8216E     | Support tube, adjustablet right               | 1        | Pc.  |
| 14    | Included         | 1062-0408      | Blind rivet F4x8                              | 2        | Pc.  |
| 15    | Included         | 3201-8120      | Bushing                                       | 2        | Pc.  |
| 16    | Included         | 1134-0005      | Snap button A 102                             | 2        | Pc.  |
| 17    | Included         | 1141-0001      | Ribbed insert SFL 14x0,8-2                    | 2        | Pc.  |
| 18    | Included         | 3201-8221      | Clamp ring for height adjustable tray         | 2        | Pc.  |
| 19    | Included         | 1031-0825      | Hexagon socket set screw M8x25                | 2        | Pc.  |

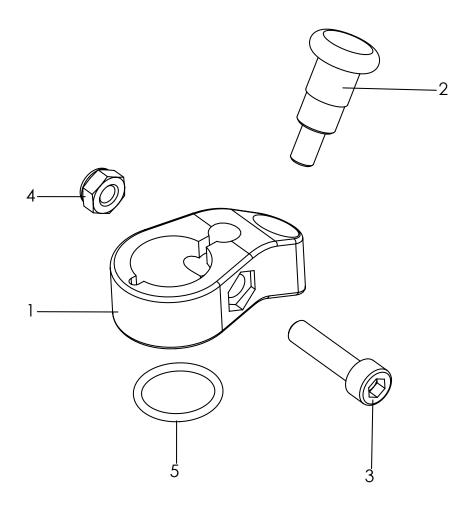


| Pos | Medema<br>number | Article number | Articel description                 | Quantity | Unit |
|-----|------------------|----------------|-------------------------------------|----------|------|
| 1-5 | 1803-1307        | 3202-7100      | Swivel lock, for two castor fork's  | 1        | Set  |
| 1   | Included         | 3202-7110      | Swivel lock housing                 | 2        | Pc.  |
| 2   | Included         | 1127-0009      | Lock pin, with interlocksystem      | 2        | Pc.  |
| 3   | Included         | 1021-0625      | Hexagon socket head cap screw M6x25 | 2        | Pc.  |
| 4   | Included         | 1042-0006      | Hexagon nut self-locking M6         | 2        | Pc.  |
| 5   | Included         | 1150-0001      | O-ring NBR 18x2                     | 2        | Pc.  |

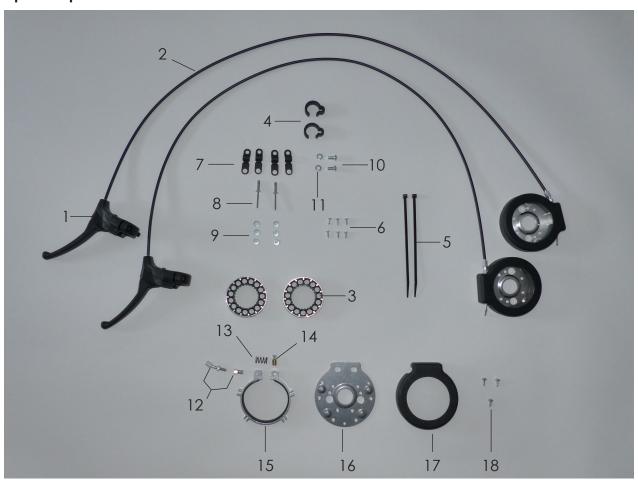

| Pos   | Medema<br>number | Article number | Articel description  | Quantity | Unit |
|-------|------------------|----------------|--|----------|------|
| 1-11  | 1803-1220        | 3231-7201      | Attendant brake, two hand use,mounted at the Push bar        | 1        | Set  |
|       | 1803-1345        | 3231-7202      | Attendant brake, two hand use, mounted at the ratchet joints | 1        | Set  |
|       | 1803-1346        | 3235-7201      | Attendant brake, two hand use,mounted at the Push bar        | 1        | Set  |
|       | 1803-1347        | 3235-7202      | Attendant brake, two hand use, mounted at the ratchet joints | 1        | Set  |
| 1     | Included         | 1209-0021      | Hand brake 80° no lock 22 mm                                 | 2        | Pc.  |
|       | Included         | 1209-0022      | Hand brake 36° no lock 22 mm                                 | 2        | Pc.  |
| 2     | Included         | 1209-0001      | Break wire set 1,5m cable                                    | 2        | Pc.  |
| 3     | Included         | 1204-0005      | Brake disc with aluminium-ring                               | 2        | Pc.  |
| 4     | Included         | 1149-0001      | Cable clip D 25  | 2        | Pc.  |
| 5     | Included         | 1089-0004      | Cable tie, black, 5x180                                      | 2        | Pc.  |
| 6     | Included         | 1039-0006      | Tapping screw with countersunk head                          | 6        | Pc.  |
| 7     | Included         | 1149-0003      | Cable clip d 6,4 / 8 mm, black                               | 4        | Pc.  |
| 8     | Included         | 1062-0516      | Blind rivet F5x16  | 2        | Pc.  |
| 9     | Included         | 1051-0005      | Washer A5,3 zp DIN/ISO 125 A5,3                              | 6        | Pc.  |
| 10    | Included         | 1029-0616      | button head socket screw M6x16                               | 2        | Pc.  |
| 11    | Included         | 1050-0006      | Washer A6,4  | 2        | Pc.  |
| 12-18 |                  |                | Spare parts, Attendant brake                                 |          |      |
| 12    | Included         | 3231-7217      | Setscrew for cable, 160° angle                               | 1        | Pc.  |
| 13    | Included         | 1131-0073      | Compression spring for hub brake                             | 1        | Pc.  |
| 14    | Included         | 1099-0015      | Screw nipple   | 1        | Pc.  |
| 15    | Included         | 3231-7213      | Alu-profile bracket with brake pad                           | 1        | Pc.  |
| 16    | Included         | 3231-7214      | Base plate for hub brake                                     | 1        | Pc.  |
| 17    | Included         | 3231-7212      | Cover for hub brake  | 1        | Pc.  |
| 18    | Included         | 1039-0001      | Screw for plastic, cross rcessed pan head w. flange zp 4x10  | 3        | Pc.  |

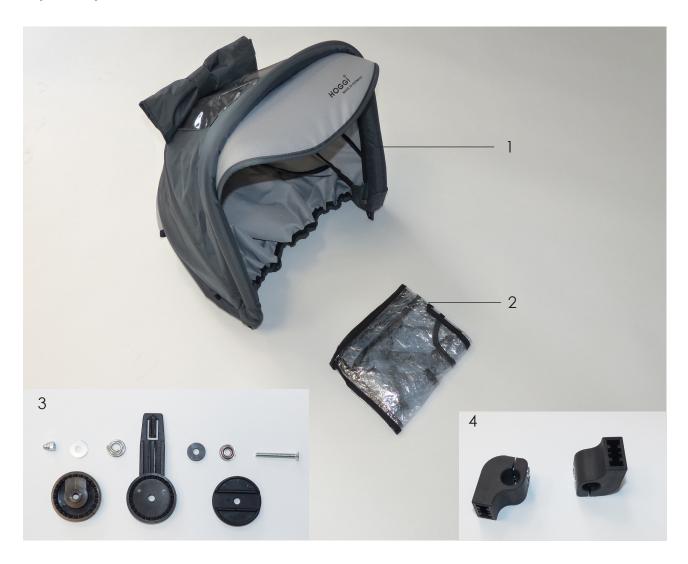

| Pos | Medema<br>number | Article number | Articel description   | Quantity | Unit |
|-----|------------------|----------------|---|----------|------|
| 1-4 | 1816-1457        | 3202-9300-323  | Canopy size 1, incl. adapter and rain protection, iron gray   | 1        | Set  |
|     | 1816-1511        | 3235-9300-323  | Canopy size 2XL, incl. adapter and rain protection, iron gray | 1        | Set  |
| 1   | Included         | 7000-0007-323  | Canopy with rain cover, gray                                  | 1        | Pc.  |
| 2   | Included         | 7000-0008      | Rain cover for canopy, BINGO                                  | 1        | Pc.  |
|     | Included         | 7000-0037      | Rain cover for canopy Zip size 2                              | 1        | Pc.  |
| 3   | Included         | 7000-0004      | Ratchet joint for canopy with fasteners                       | 2        | Pc.  |
| 4   | Included         | 3201-9310E     | Canopy adapter with fastener complete                         | 1        | Set  |

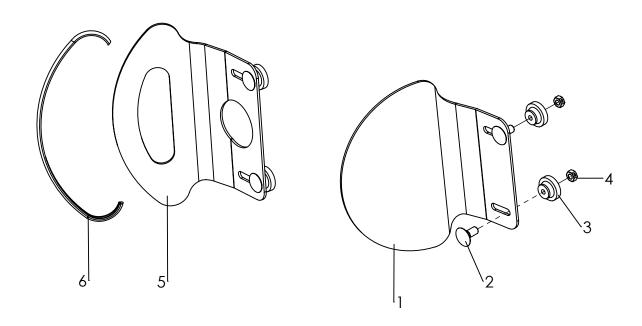

| Pos | Medema<br>number | Article number | Articel description  | Quantity | Unit |
|-----|------------------|----------------|--|----------|------|
| 1-4 | 1813-1458        | 3230-7602-323  | Headrest pads size Mini irongrey, with upholstery              | 1        | Pair |
|     | 1813-1386        | 3201-7600-323  | Headrest pads size 1, irongrey, with upholstery                | 1        | Pair |
|     | 1813-1387        | 3203-7600-323  | Headrest pads size 2 irongrey, with upholstery                 | 1        | Pair |
| 1   | Included         | 3230-7631-032  | Headrest size Mini   | 2        | Pc.  |
|     | Included         | 3201-7610-032  | Headrest size 1  | 2        | Pc.  |
|     | Included         | 3203-7610-032  | Headrest size 2  | 2        | Pc.  |
| 2   | Included         | 1026-0520      | Mushroom head square neck bolt M5x20                           | 4        | Pc.  |
| 3   | Included         | 1124-0500      | Knurled nut PA black M5  | 4        | Pc.  |
| 4   | Included         | 1049-0066      | BILOC-Nut M5   | 4        | Pc.  |
| 2-6 | 1813-1493        | 3230-7601-323  | Headrest pads low version size Mini iron grey, with upholstery | 1        | Pair |
|     | 1813-1395        | 3231-7600-323  | Headrest pads low version size 1, iron grey, with upholstery   | 1        | Pair |
|     | 1813-1462        | 3233-7600-323  | Headrest pads low version size 2, iron grey, with upholstery   | 1        | Pair |
| 5   | Included         | 3230-7611-032  | Headrest low version size Mini                                 | 2        | Pc.  |
|     | Included         | 3231-7611-032  | Headrest low version size 1                                    | 2        | Pc.  |
|     | Included         | 3233-7611-032  | Headrest low version size 2                                    | 2        | Pc.  |
| 6   | Included         | 1144-0001      | KS-Profil PVC  | 2        | Pc.  |

