# Control Medi Standard

## Manual

Art. Nr: 429220-1

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|      |                                                           |    |
|------|-----------------------------------------------------------|----|
| 11.  | Sammanställning snabbguide.....                           | 26 |
|      | GB: English Manual Control Medi Standard .....            | 29 |
| 1.   | Introduction .....                                        | 29 |
| 1.1  | Product description.....                                  | 29 |
| 1.2  | Intended purpose.....                                     | 29 |
| 1.3  | General information about Control Medi .....              | 30 |
| 1.4  | Symbols .....                                             | 31 |
| 1.5  | Inserting Batteries.....                                  | 31 |
| 1.6  | Change the batteries .....                                | 31 |
| 2.   | Important information .....                               | 32 |
| 2.1  | Safety .....                                              | 32 |
| 2.2  | Safety instructions for operation .....                   | 32 |
| 2.3  | Serious incidents .....                                   | 32 |
| 2.4  | Conformance with requirements for medical devices .....   | 32 |
| 2.5  | Product liability .....                                   | 32 |
| 3.   | Settings.....                                             | 33 |
| 3.1  | Alerts when programming .....                             | 33 |
| 3.2  | Cancel Programming.....                                   | 33 |
| 3.3  | Levels in Control Medi Standard.....                      | 33 |
| 3.4  | Locking/Unlocking the Settings Lock .....                 | 35 |
| 3.5  | Opening/Closing a Level.....                              | 35 |
| 3.6  | Recording IR Channels from another Transmitter.....       | 36 |
| 3.7  | Adding a GewaLink channel .....                           | 37 |
| 3.8  | Adding a 4096-code .....                                  | 37 |
| 3.9  | Copying a button .....                                    | 39 |
| 3.10 | Feedback when pressing buttons .....                      | 39 |
| 3.11 | Setting an acceptance delay.....                          | 40 |
| 3.12 | Repeating the last function .....                         | 40 |
| 3.13 | Creating an automatic sequence (Macro) .....              | 41 |
| 3.14 | Direct control .....                                      | 42 |
| 3.15 | Setting the channel range.....                            | 43 |
| 4.   | Button Labelling .....                                    | 44 |
| 5.   | Control Medi Holders .....                                | 44 |
| 6.   | Backups and reinstallation .....                          | 45 |
| 7.   | Cleaning .....                                            | 45 |
| 8.   | Default settings in your Control Medi .....               | 46 |
| 8.1  | Resetting to Default settings .....                       | 46 |
| 9.   | Technical data .....                                      | 47 |
| 9.1  | Technical data .....                                      | 47 |
| 9.2  | Accessories included with your Control Medi Standard..... | 47 |
| 9.3  | Other accessories availale for Control Medi.....          | 47 |
| 10.  | Storage and disposal.....                                 | 48 |
| 10.1 | Storage.....                                              | 48 |
| 10.2 | Disposal .....                                            | 48 |

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+ 2 sek, , välj nivå med , välj knapp .

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+ 2 sek, , , , .

# **GB: English Manual Control Medi Standard**

## **1. Introduction**

### **1.1 Product description**

Control Medi Standard is a programmable IR transmitter, which replaces ordinary remote controls for TV, DVD, stereo etc. It has in all 28 different programming positions on four levels. Simply press the buttons to send signals, and change level by pressing the *Level* button. You can also connect an external control switch that can directly control one button on the transmitter.

When you press a button, confirmation is given as vibration, sound and light.

The Control Medi is a part of the “GewaLink system” and therefore can be used together with all of these products.

### **1.2 Intended purpose**

Control Medi is a simple to use remote control intended to support people who are unable to use a standard remote control due to motor, cognitive or visual impairments. The device can help the user control electronic equipment at home, including TV sets, lighting, and entrance and garage door openers. The device is intended for indoor and outdoor use.

### 1.3 General information about Control Medi

To be able to use your Control Medi there are certain things that you should know:


**⚠ WARNING:** Make sure that the user always can get help through a backup system if the product does not operate correctly.

**⚠ WARNING:** Before you use the product, make sure that all connected appliances and electronic equipment are correctly installed, used as intended and safe to use.

- The Control Medi is delivered preset with GewaLink channels, meaning that it is possible to use the transmitter immediately after the batteries has been inserted. See section *Inserting Batteries*.
- To enter the programming mode, press first the *Level* button and then **simultaneously** one of the *Function* buttons for 2 seconds. Different *Function* buttons are chosen depending on what is to be programmed.
- If you press the wrong button or make a mistake when programming you can always press the *Level* button for two seconds. It works as a *Reset* button.
- It is important not to wait too long between the different steps when programming. After a certain time your Control Medi automatically leaves the programming mode to save power.
- If a button on your Control Medi or control switch is pressed for more than 1 minute, your Control Medi will stop transmitting to save power.
- The settings can be protected by using the Settings Lock.
- You can only change the settings of levels that are unlocked.

This picture shows the various terms and functions which are described in the text.

Fig 1

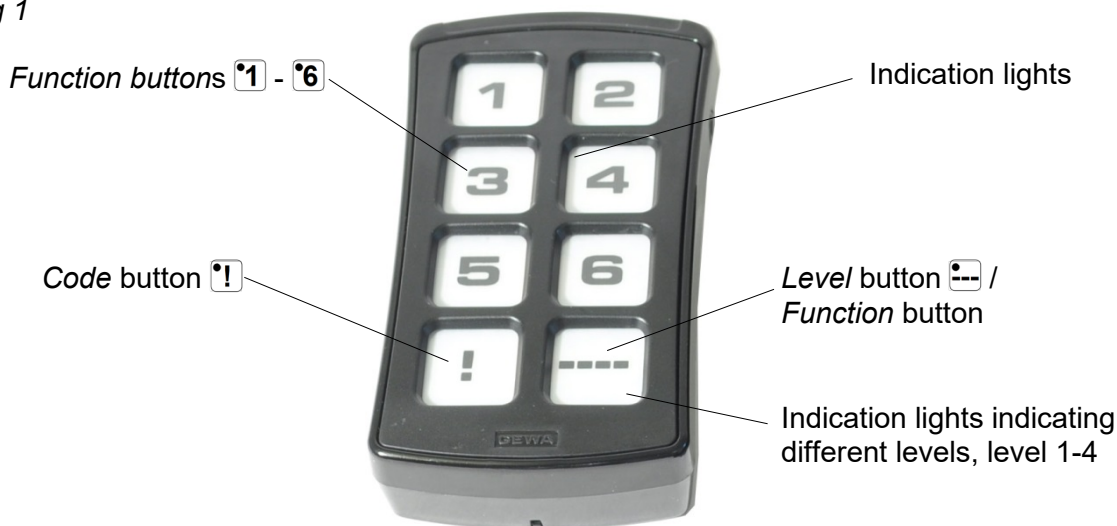## 1.4 Symbols

The following symbols appear in the text throughout this manual:

|                                                                                                                                                                                                 |                                                                           |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------|
|  <b>X</b> and  <b>Y</b>       | Any button on your Control Medi                                           |
|  <b>Y</b>                                                                                                      | Any button on another transmitter to be programmed                        |
|  <b>1</b> ...  <b>6</b>       | Buttons 1-6                                                               |
|  <b>!</b>                                                                                                      | <i>Code</i> button                                                        |
|  <b>---</b>                                                                                                    | <i>Level</i> button                                                       |
|  <b>---</b> 2 sec                                                                                              | <i>Level</i> button held pressed for approximately 2 seconds              |
|  <b>---</b> +  <b>1</b> 2 sec | <i>Level</i> button and button 1 held pressed for approximately 2 seconds |

## 1.5 Inserting Batteries

Unscrew the battery lid screws on the back of your Control Medi using a screwdriver and insert three AAA batteries (alkaline LR03 1.5 V). Insert the batteries according to the guidelines at the bottom of the battery compartment.

**NOTE:** Use non-rechargeable alkaline batteries for Control Medi Standard.

## 1.6 Change the batteries

When the batteries need replacing, the LED indicator blinks red and one long and three short beeps are repeated twice.

## 2. Important information

Before you use the product, read and understand the contents of this user manual.

### 2.1 Safety

#### Safety definitions

 **WARNING:** Risk of injury or death if the instructions are not followed.

 **CAUTION:** Risk of damage to the product if the instructions are not followed.

**NOTE:** Information that is necessary in a given situation.

### 2.2 Safety instructions for operation

 **WARNING:** Read the following warning instructions before you use the product.

- Make sure that the user always can get help through a backup system if the product does not operate correctly.
- Before you use the product, make sure that all connected appliances and electronic equipment are correctly installed, used as intended and safe to use.
- The string lock is a safety feature to prevent strangling. Make sure that the ends of the neck strap can be pulled out eg. if the transmitter gets caught on something. Never make knots in the neck strap!

### 2.3 Serious incidents

If a serious incident related to the product occurs, this must be reported to the manufacturer and the competent authority of your country.

An incident is regarded serious if it directly or indirectly led, might have led or might lead to either of these situations:

- The death of a patient, user or other person.
- The temporary or permanent serious deterioration of a patient's, user's or other person's state of health.
- A serious public health threat.

### 2.4 Conformance with requirements for medical devices

Control Medi is marketed as a technical aid for people with disabilities. It complies with all necessary requirements, regulations and directives for medical devices.

### 2.5 Product liability

The product has been developed and risk-assessed in accordance with ISO 14971. The user manual and the technical specifications are made according to the safety assessments in the risk analysis. Always consider the safety when using the product. If the product is used in other ways than the intended use, it may involve risks for which Abilia AB does not take responsibility. The estimated life of the product applies when used in accordance with its intended use and maintenance instructions.

Abilia AB is not liable for any loss, damage or consequential loss if the product has been used or installed in any way other than as specified in this document. Any form of damage or tamp.

## 3. Settings

Below is described how to program your Control Medi for different setting options.

To enter the programming mode, you should first press the *Level* button  and then simultaneously press the button  for 2 seconds for your desired setting option (see below). When you release the buttons a “Setting” alert is heard, and the indication lamp on the button for the setting option flashes. An “Error” alert will sound if you select an invalid setting option.

| Press simultaneously                                                                                                                                                        | Setting option |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------|
|  +  2 sec | Function       |
|  +  2 sec | System         |
|  +  2 sec | Reset          |

This is followed by different button presses depending on what is to be set, see the separate descriptions below.

It is important not to wait too long between the different steps when programming. After a certain time your Control Medi automatically leaves the programming mode to save power.


If your programming is successful, a “Success” alert will sound. If something has gone wrong or takes too long, an “Error” alert will sound.

### 3.1 Alerts when programming

Whilst you are programming, your Control Medi gives feedback with flashes and different alerts:

“Setting” -  (three increasing tones).

“Success” -  (three positive tones).

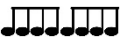“Error” -  (eight negative tones).

### 3.2 Cancel Programming

To cancel the programming, you can press the *Level* button  and keep it pressed for two seconds. It works as a *Reset* button.

### 3.3 Levels in Control Medi Standard

Your Control Medi Standard has totally four levels with seven programmable buttons. On delivery the buttons are factory-programmed with GewaLink channels 16-39 on level 1-4, see *Table 1*. You can freely use one, several or all four levels. Four indication lights indicate the current level (see *Fig 1*).

Your Control Medi Standard is delivered with only level 1 open, other levels are closed. To open further levels, see section *Opening/closing a level*.

When only one level is open, the *Level* button  automatically contains one channel which is always is the first channel on the next level. In this way, it is possible to control eight functions even if only one level is open. See <sup>1</sup> in *Table 1*.

When several levels are active you can change to another level by pressing the *Level* button . Each time you press the button the next level is chosen.

**Table 1: Default setting Gewa Link channels**

Table 1 shows the channels and codes for Control Medi Standard on delivery.

| GewaLink channels position                                                                        |                           |      |      |      |
|---------------------------------------------------------------------------------------------------|---------------------------|------|------|------|
| Button on Control Medi Standard                                                                   | Level 1-4 (red indicator) |      |      |      |
|                                                                                                   | 1                         | 2    | 3    | 4    |
|  1               | 16                        | 22   | 28   | 34   |
|  2               | 17                        | 23   | 29   | 35   |
|  3               | 18                        | 24   | 30   | 36   |
|  4               | 19                        | 25   | 31   | 37   |
|  5               | 20                        | 26   | 32   | 38   |
|  6               | 21                        | 27   | 33   | 39   |
|  ! <sup>2</sup>  | code                      | code | code | code |
|  -- <sup>1</sup> | 22                        | 28   | 34   | 16   |

<sup>1</sup> If only one level is open the *Level/* button automatically becomes one channel. It is the first channel on the next level.

<sup>2</sup> Same code on all levels.

**NOTE:** *The table above shows GewaLink channels on delivery. It is possible to select another set of GewaLink channels, see section Setting the channel range.*

### 3.4 Locking/Unlocking the Settings Lock

In order to protect the programmed settings in your Control Medi, Settings Lock can be activated.

When the Setting Lock is enabled, the settings are protected. If you try to change the settings without first unlocking, the “Error” alert is given.

Do as follows to lock/unlock the Settings Lock:

1. First press  and then simultaneously press  for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press .
3. Press  to unlock the Settings Lock, or  to lock.
4. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

#### Locking/Unlocking the Settings Lock

Press  +  2 sec, , then press your desired option  or .

 = Unlock.  = Lock.

*Default setting: Unlocked*

### 3.5 Opening/Closing a Level

Your Control Medi has four different levels, which means that each button can have four different functions. You can select the number of levels that are open and hence, how many functions each button should have. You should close levels that are not to be used.

1. First press  and then simultaneously press  for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press . The indicator light for the current level glows.
3. Select the level to be opened or closed by pressing .
4. Press  to close the selected level or  to open the selected level.  
The indicator light for the levels automatically moves to the next level, which then directly can be opened or closed.
5. Finish by pressing  for 2 seconds.
6. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

#### Opening/Closing a Level

Press  +  2 sec, , choose level with , then press the desired option  or , then  2 sec.

 = Close.  = Open.

*Default setting: Level 1 open*

### 3.6 Recording IR Channels from another Transmitter

Make sure the batteries in both transmitters are fully charged. Do not use the transmitters in intense light (e.g., sunlight) when programming.

Place the two transmitters on a table with the original transmitter directed towards the rear of your Control Medi, see picture.

1. First  and then simultaneously press  for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press .
3. Choose the level where the channels are to be recorded with .
4. Press the button  where the channel is to be recorded. The indicator light for the selected button glows steadily and the “Press” alert sounds.
5. Press and hold the corresponding button  on the original transmitter pressed. Release the button when the indicator light for the selected button goes off and the “Release” alert sounds. This may take up to 10 seconds. If the indicator light glows again and the “Press” alert sounds, you should press the button one or more times until the “Success” alert sounds.

If the “Error” alert sounds, the recording has failed. Try again from step 4.

6. If the recording of the first button was successful, repeat steps 4 and 5 until all buttons are programmed.
7. Finish by pressing  for 2 seconds.
8. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.
9. Now test your recorded functions on the device that is to be controlled.

Original transmitter

#### Recording IR Channels

Press  +  2 sec, , choose the level with , select button , select button , then continue or finish with  2 sec

**Important!** When just **one level** is used, the *Level* button automatically becomes the next levels first button, see *Table 1*. It means that it is possible to record an IR channel on the *Level* button, but only when just one level is being used.

### 3.7 Adding a GewaLink channel

Your Control Medi Standard comes with GewaLink channels 16-39 (see Table 1). If you want to use another GewaLink channel on a certain button, you can insert a chosen GewaLink channel like this:

1. First press and then simultaneously press for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press and then press . The indicator light for the current level flashes.
3. Select the level where the GewaLink channel should be added with .
4. Select the button where the GewaLink channel is to be saved.
5. Select your **Channel range**, = GewaLink 00-63. = GewaLink 64-127.
6. Select your **Channel group** according to *Table 2* below.
7. Press the button that corresponds to your desired GewaLink channel in the selected channel group according to *Table 2* below.
8. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

**Table 2: Channel group and button to add GewaLink channel**

|        | <b>Channel range : GewaLink 00-63</b>  |           |           |           |           |           |           |           |
|--------|----------------------------------------|-----------|-----------|-----------|-----------|-----------|-----------|-----------|
| Button | Ch. group                              | Ch. group | Ch. group | Ch. group | Ch. group | Ch. group | Ch. group | Ch. group |
|        | 00                                     | 08        | 16        | 24        | 32        | 40        | 48        | 56        |
|        | 01                                     | 09        | 17        | 25        | 33        | 41        | 49        | 57        |
|        | 02                                     | 10        | 18        | 26        | 34        | 42        | 50        | 58        |
|        | 03                                     | 11        | 19        | 27        | 35        | 43        | 51        | 59        |
|        | 04                                     | 12        | 20        | 28        | 36        | 44        | 52        | 60        |
|        | 05                                     | 13        | 21        | 29        | 37        | 45        | 53        | 61        |
|        | 06                                     | 14        | 22        | 30        | 38        | 46        | 54        | 62        |
|        | 07                                     | 15        | 23        | 31        | 39        | 47        | 55        | 63        |
|        | <b>Channel range : GewaLink 64-127</b> |           |           |           |           |           |           |           |
| Button | Ch. group                              | Ch. group | Ch. group | Ch. group | Ch. group | Ch. group | Ch. group | Ch. group |
|        | 64                                     | 72        | 80        | 88        | 96        | 104       | 112       | 120       |
|        | 65                                     | 73        | 81        | 89        | 97        | 105       | 113       | 121       |
|        | 66                                     | 74        | 82        | 90        | 98        | 106       | 114       | 122       |
|        | 67                                     | 75        | 83        | 91        | 99        | 107       | 115       | 123       |
|        | 68                                     | 76        | 84        | 92        | 100       | 108       | 116       | 124       |
|        | 69                                     | 77        | 85        | 93        | 101       | 109       | 117       | 125       |
|        | 70                                     | 78        | 86        | 94        | 102       | 110       | 118       | 126       |
|        | 71                                     | 79        | 87        | 95        | 103       | 111       | 119       | 127       |

#### Adding a GewaLink channel

Press + 2 sec, , , choose the level with , choose the button , choose the **Channel range**, choose the **Channel group**, then choose GewaLink channel.

See *Table 2* for your choice of Channel groups and GewaLink channels.

Example: You want to add GewaLink channel 28 onto a certain button .

Press + 2 sec, , , choose level, , , , .

### 3.8 Adding a 4096-code

Control Medi is delivered with a special coded channel (4096) on the *Code* button . The code is selected randomly. This coded channel is normally used for functions that require strict authorization such as apartment or house doors. The code has 4096 different combinations.

The *Code* button always has the same code irrespective of which level you use. If you have to program a new coded channel there are two methods of doing so. You can set the 4096 code on any level and button. It is recommended to keep the 4096 code on the *Code button*.


**NOTE!** When you add a new coded channel it is added to just one level. If you want to use the *Code button* with the same code on several levels, you must add it to each level.

**Method one:** record the code from another transmitter, as described under *Recording IR channels from another Transmitter*.

**For method two** a binary code is entered. Make up the binary code yourself or else use a code that is already set in an IR receiver or IR transmitter.

There are two types of IR receivers: an older type of receiver where you set the code with a 12-pole switch, or a newer type where the IR receiver is programmable. The principle is the same when it comes to creating the binary code. *Fig 3* shows a 12-pole switch. Switches 1, 4, 6 and 12 are in the “On” position, the others are in the “Off” position.

Fig 3

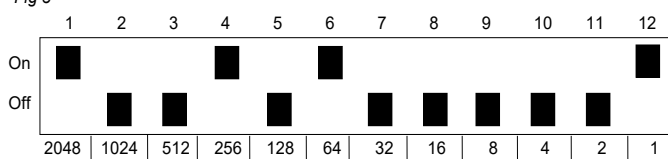The following example describes how the binary code is created according to method two:

1. First press and then simultaneously press for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
  2. Press and then press again.
  3. Select the level where the 4096 code is to be set with .
  4. Select the button where the 4096 code should be saved. (Button is recommended.)
  5. Enter the code by pressing on or .
- = switch in Off position and = switch in On position.  
Press twelve times in turn; , , , , , , , , , , , , (Fig 3).
6. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.
  7. The code is now set on your selected button. NOTE! The code above is just an example and should be different for different users. Make a note of the chosen code for future usage.

**NOTE!** If you have added a coded channel to the *Code button* and you want to use the *Code button* with the same code on several levels, you must add it to each level individually.

### Adding a 4096 code

Press + 2 sec, , , choose the level with , choose the button , then press or 12 times. = Off position. = On position.

*Default setting: Random code*

### 3.9 Copying a button

Using your Control Medi, it is possible to copy a buttons' function to another button.

NOTE! The code/channel remains in its original place.

1. First press and then simultaneously press for 2 seconds. When you release the buttons the "Setting" alert sounds and the indication lamp on button 1 flashes.
2. Press .
3. Select the level with and press the button from which the code is to be copied.
4. Select the level where the code should be copied to with . Press the button where you want to copy the code to.
5. If the action is successful, the "Success" alert sounds. If something goes wrong or takes too long, the "Error" alert sounds.

#### Copying a button

Press + 2 sec, , choose your level and button to be copied , then choose level and button to which you want to copy the code.

### 3.10 Feedback when pressing buttons

Your Control Medi always gives feedback in the form of light signals when you press a button. You can also get feedback as sound alerts and vibration. It is possible to get feedback from all three ways simultaneously, or you can set up what will be on or off (light is always on).

Do as follows to set the levels of sound and vibration.

1. First press and then simultaneously press for 2 seconds. When you release the buttons the "Setting" alert sounds and the indication lamp on button 2 flashes.
2. Press .
- 3a. Sound level:  
Press to lower the volume, press to increase the volume. Repeat pressing until the desired volume is set.  
*and/or:*
- 3b. Vibration level:  
Press to lower the vibration, press to increase the vibration. Repeat pressing until the desired vibration is set.
4. Finish by pressing for 2 seconds.
5. If the action is successful, the "Success" alert sounds. If something goes wrong or takes too long, the "Error" alert sounds.

#### Set level for sound and vibration

Press + 2 sec, , choose the setting, then 2 sec.

= Lower volume. = Increase volume. = Lower vibration. = Increase vibration.

*Default setting: Sound alert = 1 (0 – 5) Vibration = 1 (0 – 5)*

### 3.11 Setting an acceptance delay

An acceptance delay can be set for pressing the buttons. Acceptance delay is the time a button has to be pressed for before a function is activated. An acceptance delay can be set from 1 to 4. 1 is no delay at all (off) and 4 is the maximum delay (1 sec.).

1. First press  and then simultaneously press  or 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 2 flashes.
2. Press .
3. Press your desired value for the acceptance delay,  is off, and  is max.
4. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

#### Setting an acceptance delay

Press  +  2 sec, , then choose the setting.

 = 0 sec,  = 0.3 sec,  = 0.6 sec,  = 1 sec

*Default setting: Acceptance delay = OFF (0 sec)*

### 3.12 Repeating the last function

Your Control Medi Standard can have a button that repeats the last sent function.

Do as follows to add Repeat Last Function to a selected button:

1. First press  and then simultaneously press  for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 2 flashes.
2. Press .
3. Press  to choose the level.
4. Press the desired button  on the selected level.
5. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

To delete the Repeat Last Function, you will need to reset the button. See *Reset to default settings, reset a button*.

#### Repeating the last function

Press  +  2 sec, , choose the level and button .

*Default setting: OFF (none)*

### 3.13 Creating an automatic sequence (Macro)

Your Control Medi is able to send sequences of codes. When you press **one** button, Control Medi automatically carries out what is equivalent to pressing several buttons. To find a TV menu page for instance you need to press several buttons. Control Medi however, can be programmed in such a way that you only need to press one button. Other available sequences are dialling long telephone numbers or muting the TV at the same time as you answer the telephone. Your Control Medi can also be programmed to change levels and remember how long the button is pressed, you can transmit for a maximum of 25 seconds per press. Other programmed sequences can also be a part of a new sequence.

A sequence can be terminated at any time by pressing any other button.

When a sequence is programmed, your Control Medi transmits at the same time what is already on the key. A maximum of 50 button presses per sequence can be programmed. NOTE! The button used for the sequence must not be part of the sequence.

1. First press  and then simultaneously press  for 2 seconds. When you release the buttons the "Setting" alert sounds and the indication lamp on button 1 flashes.
2. Press .
3. Press  to start the recording.
4. Press the buttons to be used for the sequence.
5. First press  and then simultaneously press . When you release the buttons the "Setting" alert sounds and the indication lamp on button 1 flashes.
6. Press .
7. Press  to finish the recording.
8. Select where you want the sequence to be with button .
9. If the action is successful, the "Success" alert sounds. If something goes wrong or takes too long, the "Error" alert sounds.

#### Creating an automatic sequence (Macro)

Press  +  2 sec, , , create the sequence,  + , , , select the button  to be saved.

*Max 25 seconds per press, 50 presses*

### 3.14 Direct control

Your Control Medi can directly control a button with an external control switch. You can choose which button that should be activated when you press the control switch.

It is also possible to direct control two buttons some seconds apart. The time can be from 2 to 8 seconds. (Direct Control of two buttons applies from ver. 1.0.6).

A control switch with 3.5mm plug will fit in to the side of your Control Medi.

Do as follows to activate/deactivate Direct control:

1. First press  and then simultaneously press  for 2 seconds. When you release the buttons the "Setting" alert sounds and the indication lamp on button 2 flashes.
2. Press .

3a. Activate direct control.

Press , and then press the button  that you want to be activated when pressing the control switch.

or:

3b. Activate direct control of two buttons.

Press , and then press the first button  that you want to be activated when pressing the control switch. Press the second button  to be activated. Select time before the second button should be activated with  - .

or:

3c. Press  to turn off direct control, (Resetting scanning).

4. If the action is successful, the "Success" alert sounds. If something goes wrong or takes too long, the "Error" alert sounds.

#### Activating/deactivating Direct control

Activate direct control: Press  +  2 sec, , , choose the button .

Activate direct control of two buttons:  +  2 sec, , , choose the first button , choose the second button . Select time  - .

 = 2 sec,  = 4 sec,  = 6 sec,  = 8 sec

Deactivate direct control: Press  +  2 sec, , .

Default setting: OFF (none)

### 3.15 Setting the channel range

Your Control Medi has the option to set different channel ranges for the transmitter to operate within. When the change of channel range is applied, programmed channels will disappear and a new random code is added on the *Code* button (!).

Normally, the channel range does not need to be changed. If several GewaLink transmitters are to be used in the same area and you do NOT want to control the same equipment, it may be necessary to use different channel ranges.

Table 3 shows the different settings.

**Table 3: Channel range**

|         | Channel range, GewaLink channels |            |            |                     |             |              |
|---------|----------------------------------|------------|------------|---------------------|-------------|--------------|
|         | 00-23<br>❶                       | 16-39<br>❷ | 32-55<br>❸ | 56-79<br>❹          | 80-103<br>❺ | 104-127<br>❻ |
| Level 1 | 00-05                            | 16-21      | 32-37      | 56-61               | 80-85       | 104-109      |
| Level 2 | 06-11                            | 22-27      | 38-43      | 62 <sup>1</sup> -67 | 86-91       | 110-115      |
| Level 3 | 12-17                            | 28-33      | 44-49      | 68-73               | 92-97       | 116-121      |
| Level 4 | 18-23                            | 34-39      | 50-55      | 74-79               | 98-103      | 122-127      |

<sup>1</sup> Channel 62 is a stop code and cannot control any function. On the other hand you can store channels/codes on this button which in turn can control a function.

Do as follows to set the channel range:

1. First press and then simultaneously press for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 2 flashes.
2. Press .
3. Choose the channel range to according to table 3 above.
4. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

#### Setting the channel range

Press + 2 sec, , choose the channel range.

= 00-23, = 16-39, = 32-55, = 56-79, = 80-103, = 104-127

*Default setting: GewaLink channel 16-39*

## 4. Button Labelling

The buttons on your Control Medi are marked with numbers as standard. The numbered label can be replaced with a personalised design. It is possible to create your own button labels using pre-designed templates with symbols and text, refer to [www.abilia.com](http://www.abilia.com).

The document can also be downloaded. The labeling is printed on a clear sheet or white paper and inserted through the slit on your Control Medi. Before the button label is inserted, the keyguard must be removed.

Remove the keyguard by gripping the middle, and then lifting it straight up, see photo.

An A4 sheet for making notes is included with your Control Medi.

## 5. Control Medi Holders

Rubber feet are mounted on the underside so that your Control Medi can lie still on flat surfaces. Your Control Medi can also be equipped with a clip (accessory) which can be attached to a belt or pocket.

As an optional accessory, there is also a holder that fits to your Control Medi. This holder can be assembled for mounting on to a table or attaching to a pipe. The holder can also be mounted on a mobile stand.

Another option is to hang your Control Medi around the neck. A neck strap is supplied with your Control Medi.

Do as follows to attach the neck strap:

1. Unscrew the battery lid.
2. Place the thin loop of the neck strap in the groove around the screw mount.
3. Screw the battery lid back down.
4. Adjust the length of the neck strap.

**⚠ WARNING:** The string lock is a safety feature to prevent strangling. Make sure that the ends of the neck strap can be pulled out eg. if the transmitter gets caught on something. Never make knots in the neck strap

## **6. Backups and reinstallation**

Your Control Medi can be connected to a computer in order to make a backup of the settings and recordings of your transmitter. The backup copy can be used to reinstall the settings and recordings onto your transmitter. Use the Control Tool USB Interface accessory (sold separately). The software can be downloaded [www.abilia.com](http://www.abilia.com).

## **7. Cleaning**

Your Control Medi can be cleaned with a damp cotton wool pad with surgical spirit, Chlorhexidine or ethyl alcohol.

It can also be cleaned with a damp cloth. Wipe dry after cleaning.

## 8. Default settings in your Control Medi

On delivery your Control Medi Standard will be set with certain default settings as follows:

Acceptance delay is set to value OFF

Feedback sound level is set to value 1 (0 – 5)

Feedback vibration level is set to value 1 (0 – 5)

Only level 1 is open, other levels are closed

GewaLink channels 16-39 are on level 1-4

A random code on the  button

### 8.1 Resetting to Default settings

It is possible to reset one button, reset all buttons on one level, or reset the entire Control Medi to default settings. See *Table 1 Default setting Gewa Link channels*, which shows you which channels are set after a reset. Other settings will also return to default settings, see above. If the settings lock is activated, the device must first be unlocked. See *Locking/Unlocking the Settings Lock*

Do the following to restore:

First press  and then simultaneously press  for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 3 flashes.

2a. Reset **one button**:

Press .

Choose the level with .

Choose the button  to be restored.

or:

2b. Reset **all buttons on one level**:

Press .

Choose the level with .

Press .

or:

2c. Reset to **factory settings** (all buttons on all levels):

Press .

Press , ,  in sequence.

If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

#### Resetting to default settings

To reset one button:

Press  +  2 sec, , choose level with , then choose button .

To reset one level:

Press  +  2 sec, , then choose level with , .

To reset to factory settings:

Press  +  2 sec, , , , .

## 9. Technical data

### 9.1 Technical data

|                                          |                                                                                                                        |
|------------------------------------------|------------------------------------------------------------------------------------------------------------------------|
| Power consumption at transmitting        | Approx. 40 mA (GewaLink channel)                                                                                       |
| Input control switch type:               | 3.5mm jack plug 2-pole                                                                                                 |
| Battery life (button use only):          | Approx. 2 years (100 seconds transmitting per day with GewaLink channel)                                               |
| Programming locations:                   | 28 places                                                                                                              |
| Operating force buttons:                 | < 160g                                                                                                                 |
| Range:                                   | 40-50m (GewaLink channels)                                                                                             |
| EMC-standard emission:<br>" immunity:    | EN 301 489-3 V1.4.1:2002, EN(IEC) 61000-6-2:2005, EN(IEC) 61000-6-3:2007, selected parts from EN 300 220-2 V2.3.1:2010 |
| Computer Connection:                     | Via 429252 Control Tool USB interface                                                                                  |
| Measurements (L x W x H):                | 100 x 55 x 22 mm (incl. rubber feet)                                                                                   |
| Weight (incl. batteries):                | 105g                                                                                                                   |
| Operating temperature (excl. batteries): | -20°C to +45°C                                                                                                         |

### 9.2 Accessories included with your Control Medi Standard

| Art.no | Name                     | Description                                  | Qty   |
|--------|--------------------------|----------------------------------------------|-------|
| 429255 | Control Medi Labelling 8 | For labeling of buttons                      | 1 pcs |
| -      | Note sheet               | A4-sheet for noting your programmed channels | 1 psc |
| 429257 | Control Medi Keyguard 8  | Keyguard for 8 buttons                       | 1 pcs |
| 890410 | Neck strap               |                                              | 1 pcs |
| 8375   | Battery                  | LR03 (AAA) 1,5 V                             | 3 pcs |

### 9.3 Other accessories available for Control Medi

| Art.no | Name                           | Description                                                      |
|--------|--------------------------------|------------------------------------------------------------------|
| 429249 | Control Medi Holder 6 mm Hole  | For mounting a Control Medi to different systems with a 6mm hole |
| 429250 | Control Medi Holder 10 mm Pipe | For mounting a Control Medi to a 10mm pipe system                |
| 429251 | Control Medi Belt Clip         |                                                                  |
| 429252 | Control Tool USB interface     | Adapter for connecting a Control Medi to PC                      |

## 10. Storage and disposal

### 10.1 Storage

Keep the product indoors at room temperature and away from moist.

### 10.2 Disposal

The Control Medi contains electronics components that must be recycled. Dispose the product at an applicable disposal location.

# 11. Summary and Quick guide

## Locking/Unlocking the Settings Lock

Press  +  2 sec, , then press your desired option  or .

 = Unlock.  = Lock.

Default setting: Unlocked

## Opening/Closing a Level

Press  +  2 sec, , choose level with , then press the desired option  or , then  2 sec.

 = Close.  = Open.

Default setting: Level 1 open

## Recording IR Channels

Press  +  2 sec, , choose the level with , select button , select button , then continue or finish with  2 sec

## Adding the 4096 code

Press  +  2 sec, , , choose the level with , choose the button , then press  or  12 times.

 = Off position.  = On position.

Default setting: Random code

## Copying a button

Press  +  2 sec, , choose your level and button to be copied , then choose level and button  to which you want to copy the code.

## Setting the levels for sound and vibration

Press  +  2 sec, , choose the setting, then  2 sec.

 = Lower volume.  = Increase volume.  = Lower vibration.  = Increase vibration.

Default setting: Sound alert = 1 (0 – 5) Vibration = 1 (0 – 5)

### Setting the acceptance delay

Press  $\text{---} + \text{2}^{2 \text{ sec}}$ ,  $\text{2}$ , then choose the setting.

$\text{1} = 0 \text{ sec}$ ,  $\text{2} = 0.3 \text{ sec}$ ,  $\text{3} = 0.6 \text{ sec}$ ,  $\text{4} = 1 \text{ sec}$

Default setting: Acceptance delay = OFF (0 sec)

### Repeat latest function

Press  $\text{---} + \text{2}^{2 \text{ sec}}$ ,  $\text{5}$ , choose the level and button  $\text{X}$ .

Default setting: OFF (none)

### Creating an automatic sequence (Macro)

Press  $\text{---} + \text{1}^{2 \text{ sec}}$ ,  $\text{4}$ ,  $\text{1}$ , create the sequence,  $\text{---} + \text{1}$ ,  $\text{4}$ ,  $\text{2}$ , select the button  $\text{X}$  to be saved.

Max 25 seconds per press, 50 presses

### Activating/deactivating Direct control

Activate direct control: Press  $\text{---} + \text{2}^{2 \text{ sec}}$ ,  $\text{3}$ ,  $\text{2}$ , choose the button  $\text{X}$ .

Activate direct control of two buttons:  $\text{---} + \text{2}^{2 \text{ sec}}$ ,  $\text{3}$ ,  $\text{3}$ , choose the first button  $\text{X}$ , choose the second button  $\text{X}$ . Select time  $\text{1} - \text{4}$ .

$\text{1} = 2 \text{ sec}$ ,  $\text{2} = 4 \text{ sec}$ ,  $\text{3} = 6 \text{ sec}$ ,  $\text{4} = 8 \text{ sec}$

Deactivate direct control: Press  $\text{---} + \text{2}^{2 \text{ sec}}$ ,  $\text{3}$ ,  $\text{1}$ .

Default setting: OFF (none)

### Setting the channel range

Press  $\text{---} + \text{2}^{2 \text{ sec}}$ ,  $\text{4}$ , choose the channel range.

$\text{1} = 00-23$ ,  $\text{2} = 16-39$ ,  $\text{3} = 32-55$ ,  $\text{4} = 56-79$ ,  $\text{5} = 80-103$ ,  $\text{6} = 104-127$

Default setting: GewaLink channel 16-39

### Resetting to default settings

To reset one button:

Press  $\text{---} + \text{3}^{2 \text{ sec}}$ ,  $\text{2}$ , choose level with  $\text{---}$ , then choose button  $\text{X}$ .

To reset one level:

Press  $\text{---} + \text{3}^{2 \text{ sec}}$ ,  $\text{3}$ , then choose level with  $\text{---}$ ,  $\text{1}$ .

To reset to factory settings:

Press  $\text{---} + \text{3}^{2 \text{ sec}}$ ,  $\text{1}$ ,  $\text{1}$ ,  $\text{2}$ ,  $\text{3}$ .

## **Contact Information**

### **Sweden**

Abilia AB, Råsundavägen 6, 169 67 Solna, Sweden

Phone +46 (0)8- 594 694 00 | support@abilia.se | [www.abilia.se](http://www.abilia.se)

### **Norway**

Abilia AS, Tevlingveien 23, 1081 Oslo, Norway

Phone +47 23 28 94 00 | support@abilia.no | www.abilia.no

### **United Kingdom**

Abilia UK Ltd, Unit12 Buckingway Business Park, Swavesey,  
Cambridge, CB24 4AE, United Kingdom

Phone +44 (0) 1954 281 210 | support@abilia.uk | [www.abilia.uk](http://www.abilia.uk)

### **International**

[www.abilia.com/intl/get-products/distributors](http://www.abilia.com/intl/get-products/distributors)