# ribcap®

1. **Instructions for use** ..... English
2. **Gebruiksaanwijzing**..... Dutch
3. **Brugsanvisning** ..... Danish
4. **Gebrauchsanweisung**..... German
5. **Käyttöohjeet** ..... Finnish
6. **Instructions d'utilisation** ..... French
7. **Istruzioni per l'uso**..... Italian
8. **Instrucciones de uso** ..... Spanish
9. **Bruksanvisning**..... Swedish
10. **Návod k použití**..... Czech
11. **Naudojimo instrukcija** ..... Lithuanian
12. **Instrukcja użycia** ..... Polish
13. **Instruções de Utilização** ..... Portuguese

# 1. Instruction for Use - English

## 1. PRODUCT DESCRIPTION

### 1.1 Name of the Medical Device

Device Name: Cap with sewn-in protectors

Trade Name: RIBCAP®

### 1.2 List of any accessories covered

RIBCAP® head protectors have an adjustable chinstrap and side-releasing safety buckle. Except: Chessy, Bieber, Palmer and some baseball cap series. Non-exhaustive list.

Adjust the straps to ensure a good fit of the head protector, but not too tight.

## 2. PURPOSE OF USE

RIBCAP® head protector models serve to protect from head injuries caused by uncontrolled falls or sequences of movements, they cover fall hazards with minor or medium potential for injury through the protective effect of the shock-absorbing foam padding.

The top of the head is almost completely enclosed by pads. The head protector comes in different sizes in a range from 47cm to 65cm. (18.5" to 25.6").

## 3. INTENDED USERS

People who need a helmet due to weakened stability. i.e., people with balance issues due to age or a medical condition.

## 4. INTENDED PATIENT POPULATION

The intended patient population is the intended users of this device.

## 5. MEDICAL CONDITIONS

The medical conditions that require a medical helmet use include but are not limited to:

- Neurological conditions (i.e. epilepsy, autism, neuromuscular disorders, spasticity)
- Reduced mobility
- Oncological diseases
- The necessity of head protection after an operation

## 6. INDICATIONS

The medical conditions that require a medical helmet use include but are not limited to: Neurological conditions (i.e., epilepsy, autism, neuromuscular disorders, spasticity) Reduced mobility Oncological diseases Necessity of head protection after an operation. There are no known contraindications.

Note: RIBCAP® is not certified as a sports or riding helmet (such as skiing, skating, cycling, ... etc.).

## 7. RESIDUAL RISK

The evaluation of all risks by the GLOMARKET RIBCAP®'s risk management showed that the remaining residual risk is as low as reasonably possible.

### 7.1 Safety guidelines

Please ensure the head protector is in a faultless condition before use.

The head protector series bears the CE quality mark and complies with the requirements of: EU/2017/745

### 7.2 Safety and wearing guidelines.

· RIBCAP® head protectors have an adjustable chinstrap and side-releasing safety buckle. Except: Chessy, Bieber, Palmer and some baseball cap series. Non-exhaustive list.

· Adjust the straps to ensure a good fit of the head protector, but not too tight.

· Please ensure both straps are firmly locked! Make sure the wearer understands how the buckle works before using.

· The RIBCAP® head protector may only be used for the indications listed and must not be used for any other purposes (for example as a helmet in general, or for a helmet used for bike riding, horseback riding and other sports!).

· Ensure the head protector is always handy, so that it can be put on quickly and properly! The label inside the RIBCAP® must be at the back when placed on the head.

· When the product is not being used, store it in normal indoor conditions. When disposing of the packaging and the product, comply strictly with the legal norms of your community.

### 7.3 Limitations;

This product;

- cannot be used as a sports helmet

**ribcap**

[www.ribcap.com](http://www.ribcap.com)

- does not support safety in case extreme falls without any reflexes.

#### 7.4 Contraindications

There are no known contraindications.

#### 7.5 Complications and side effects

Rare Complications: Extreme sweating and weakened neck muscles can be considered complications and side effects.

#### 7.6 Warnings

The product must be replaced if required by one or more of the following criteria:

- The fit changes, for example, through growth, changes in head shape, etc.
- Exceptional strain to the head protector
- Frequency of wear: Although RIBCAP® does not lose its quality, there's a certain life span on protective foams.
- Do not use if the product is damaged.
- Hygienic conditions of the head protector, influenced by (for example) medication and clinical picture.
- Please ensure both straps are firmly locked! Make sure the wearer understands how the buckle works before using.

The patient's general condition is an important part of the risk assessment. In individual cases, the decision must be made by the attending physician, who must weigh the risks versus the expected benefits

#### 7.7 Precautions

- People with self-harming behavior should not be left alone with the product.
- Instructions for care of the product should be followed

#### 7.8 Adverse reactions

The device has no known adverse reactions.

### 8. CLEANING AND CARE OF THE HEAD PROTECTOR

RIBCAP® head protectors may be cleaned using mild soap and a damp cloth or sponge. Strong solvents may contain substances that can damage the head protector and impair its protective effect.

The head protector may be hand washed at a temperature of 30°C. Do not expose the head protector to high temperatures! Washing instructions are on the labels inside the RIBCAP®. To dry the RIBCAP®, leave it to dry at room temperature. Do not hang up or iron, and do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

### 9. FITTING INSTRUCTIONS

Select the right size by measuring around the head at forehead level. Fit the headgear onto the wearer's head, making sure that the front is correctly positioned, and adjust the chinstrap to the required tightness to ensure that the headgear is stable and secure.

### 10. WARRANTY

There is a 2-year warranty on this product. The warranty period begins on the date of purchase (shown on the purchase receipt). During the warranty period, you can submit any complaints about a defective RIBCAP® to your dealer or to the service address specified. In this period, we will repair all defects due to faulty materials or operation free of charge. Improper handling, falls, impacts and the like are excluded from the warranty. A new warranty period does not start if the RIBCAP® is repaired or exchanged; the 2-year warranty period as of the purchase date continues unchanged.

Any warranty expires in the event of non-compliance such as use other than as intended, repairs and alterations made to the head protector or its components by unauthorised people.

### 11. OPERATING LIFE

To obtain optimum protection, the product must be treated with due care and diligence. Ensure a precise fit. The RIBCAP® must be replaced after 2 years with a new RIBCAP®, as optimum protection can then no longer be guaranteed due to the deterioration of the material properties. The product must be replaced if required by one or more of the following criteria:

- The fit changes, for example through growth, changes in head shape, etc.
- Exceptional strain to the head protector
- Frequency of wear, frequency of falls
- Degree of care
- Hygienic conditions of the head protector, influenced by (for example) medication and clinical picture.

RIBCAP® guarantees all its products as long as they have not been manipulated or altered from their original state. The guarantee does not apply for products that have been altered from their original state as a result of misuse, deficiencies or breakage of any kind. If you observe any deficiency or anomaly, immediately inform the establishment from which it was obtained in order for it to be changed.

**ribcap** 

[www.ribcap.com](http://www.ribcap.com)

## 12. EXPECTED CLINICAL BENEFITS AND CLAIMS

The Clinical Evaluation Report of RIBCAP® focus on confirming the following clinical claim:  
· RIBCAP® aims to prevent or minimize injuries caused by damage to users' heads.

## 13. RE-USE

RIBCAP® is a reusable, single-patient use product.

## 14. NATURE AND DURATION OF CONTACT WITH THE PATIENT

According to EN ISO 10993-1:2020, RIBCAP® is in contact with intact skin, and RIBCAP®'s contact duration is long-term exposure (C) – medical devices whose cumulative sum of single, multiple, or repeated contact time exceeds 30d. Therefore, although the daily use is less than 24 hours, it was evaluated in the long-term duration category (C >30d) to correctly assess the potential cumulative effect and take precautions against the worst scenario when repeated use is considered.

Table 7-1 Nature and duration of contact according to EN ISO 10993-1:2020

Nature of body contact		Contact duration
Surface medical device	Intact skin	C – Long term

## 15. STORAGE INFORMATION

- Ensure the head protector is always handy, so that it can be put on quickly and properly!
- The label inside the RIBCAP® must be at the back when placed on the head.
- When the product is not being used, store it in normal indoor conditions.

## 16. USE ENVIRONMENT

RIBCAP® does not cycling cycling or other sports helmet, but RIBCAP® is intended for use during daily life and low risk activities

## 17. DISPOSAL

When disposing of the packaging and the product, comply strictly with the legal norms of your community.

## 18. NOTICE REGARDING SERIOUS INCIDENT

For a patient/user/third party in the European Union and in countries with an identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority. The contact information for the manufacturer of this device to report a serious incident is as follows:  
RIBCAP® (Glomarket bv, Zwaarveld 34, 9220 Hamme, Belgium) e-mail: contact@Ribcap.com

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**ribcap** 

[www.ribcap.com](http://www.ribcap.com)

Symbol	Symbol Title	Symbol Definition
	Consult instructions for use or consult electronic instructions for use	Indicates the need for the user to consult the instructions for use
	Manufacturer	Indicates the medical device Manufacturer name and address
	Catalog Number	Indicates the manufacturer's catalogue number so that the medical device can be identified
	Batch Code	Indicates the manufacturer's batch code so that the batch or lot can be identified
	Caution	Indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences
	Keep away from sunlight	Indicates a medical device that needs protection from light sources
	Keep dry	Indicates a medical device that needs to be protected from moisture

**ribcap** 

[www.ribcap.com](http://www.ribcap.com)

Symbol	Symbol Title	Symbol Definition
	Medical Device	Indicates the item is a medical device
	Single Patient	Indicates a medical device that may be used multiple times (multiple procedures) on a single patient
	Unique Device Identifier	Indicates a carrier that contains unique device identifier information
	Patient Information Site	Indicate a website where a patient can obtain additional information on the medical product


**ribcap** 

[www.ribcap.com](http://www.ribcap.com)

## STANDARD HEAD CIRCUMFERENCE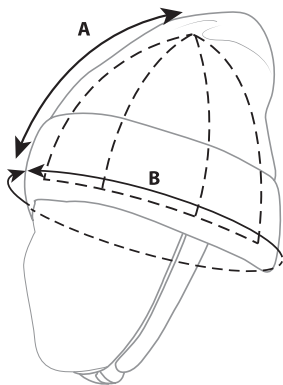

## SIZING KIDS/ADULT

Size	A	B
Mini	16 cm   6,3"	47 - 49 cm   18,5 - 19,3"
Midi	16,5 cm   6,5"	50 - 52 cm   19,7 - 20,5"
Maxi	17,5 cm   6,9"	53 - 55 cm   20,8 - 21,6"
Small	17,5 cm   6,9"	53 - 55 cm   20,8 - 21,6"
Medium	18,5 cm   7,3"	56 - 58 cm   22,0 - 22,8"
Large	19,5 cm   7,7"	59 - 61 cm   23,2 - 24,0"
Extra Large*	20,5 cm   8,1"	62 - 65 cm   24,4 - 25,6"

When you're in between sizes, please choose the larger size

\*only available in Lenny Grey and Anthracite


**ribcap** 

[www.ribcap.com](http://www.ribcap.com)

## STANDARD HEAD CIRCUMFERENCE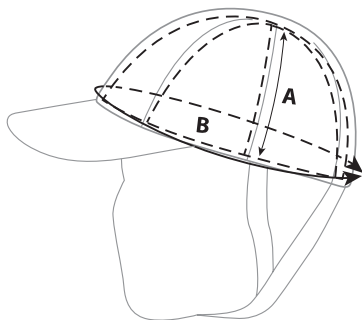

## SIZING BASEBALL CAPS

Size	A	B
Midi / Maxi for kids baseball cap	16,5 cm   6,5 "	50 - 55 cm   19,7 - 21,6 "
Small / Medium	17,5 cm   6,9 "	53 - 58 cm   20,8 - 22,8 "
Large / Extra Large	18,5 cm   7,3 "	59 - 65 cm   23,2 - 25,6 "


When you're in between sizes, please choose the larger size

**ribcap** 

[www.ribcap.com](http://www.ribcap.com)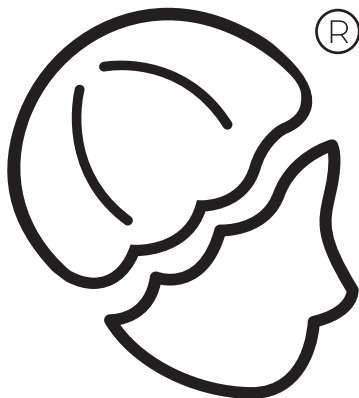

**ribcap**<sup>®</sup>

**Office & Warehouse:**

Glomarket bv

Zwaarveld 34 - 9220 Hamme - Belgium

VAT: BE 0550.525.676  
SRN: BE-MF-000000730

**[www.ribcap.com](http://www.ribcap.com)**  
**[contact@ribcap.com](mailto:contact@ribcap.com)**

Version: 11/2022