RECONDITIONING GUIDE:

ROHO Cushions, Cushioning Devices, Covers, Foam Components, Mattress Overlay

Operation manuals are available at permobil.com.

General Safety Information (Applies to all ROHO products listed on this page.)

- Warnings:**
- Cleaning and disinfecting are separate processes. Cleaning must precede disinfection. Before use by a different individual: clean, disinfect, and check the product for proper functioning and confirm that all package contents are provided with the product. Refer to the operation manual provided with the product.
 - Clean the product regularly and when it becomes soiled.
 - Use of the following may cause damage: abrasives (steel wool, scouring pads); caustic, automatic dishwasher detergents; cleaning products that contain petroleum or organic solvents, including acetone, toluene, Methyl Ethyl Ketone (MEK), naphtha, dry cleaning fluid, adhesive remover; oil-based lotions, lanolin; ozone gas; ultraviolet light cleaning methods.
 - Follow all manufacturer bleach container and germicidal disinfectant safety guidelines. For a list of acceptable disinfectants, contact the equipment provider or local health authority.
 - High temperatures accelerate aging and will damage the product assembly. ROHO products are not packaged in a sterile condition, nor are they intended or required to be sterilized prior to use. If institution protocol demands sterilization: First, follow the cleaning and disinfecting instructions; then, open the inflation valve and use the lowest possible sterilizing temperature, but not to exceed 70°C (158°F), for the shortest time possible. DO NOT autoclave.
 - After cleaning and disinfecting, and before using, check all components for proper functioning and replace if necessary.

Cushions and Cushioning Devices: DRY FLOATATION Cushion**, HYBRID ELITE Cushion Insert, Toilet Seat Cushion, Commode Cushion

** Includes the following: ROHO LOW and HIGH PROFILE® Single Compartment Cushions; HIGH PROFILE Sensor Ready® Cushion; LOW, MID, and HIGH PROFILE QUADTRO SELECT® Cushions; CONTOUR SELECT® Cushion; ENHANCER® Cushion

Note: Deflated air cells may be easier to thoroughly clean. To deflate the air cells: with the inflation valve(s) open, roll up and/or compress the cushion, and then completely close the inflation valve(s).

- Warnings for WASHING and DISINFECTING by HAND or by MACHINE; and for DRYING CABINET Use:**
- Washing and disinfecting the cushion by hand and air drying is strongly recommended.
 - Valves MUST be closed: DO NOT allow water or cleaning solution to enter the cushion. Valve(s) MUST be closed before cleaning or disinfecting. Water in a Sensor Ready Cushion may cause a false reading when using Smart Check.
 - Before cleaning or disinfecting the Sensor Ready Cushion, remove the Smart Check.
 - Before cleaning or disinfecting the HYBRID ELITE Cushion Insert, carefully remove the cushion insert from the contoured foam base.
 - DO NOT expose cushions to temperatures above 70°C (158°F), which may damage cushion components. Overheating neoprene rubber can cause permanent product damage.
 - Machine washing can cause wear and damage, including cuts and tears to the cushion and the detachment of required product information. High water temperature may cause cushion discoloration and label detachment.
 - For machine washing or disinfecting, make sure the washing machine capacity is sufficient. A washing machine that isn't large enough may not remove all soil or contaminants from a cushion.
 - ALWAYS confirm that any hook fastener strips remain attached and in the correct position, as they may loosen after machine washing or disinfecting the cushion or after exposing the cushion to a hand held dryer or drying cabinet.
 - Using a damaged cushion will reduce or eliminate the benefits of the cushion and increase risk to skin and other soft tissue.
- Caution:** Rinse thoroughly. Detergent residue may cause air cells to stick together.

<p>HAND WASH Instructions: Remove the cover and close the inflation valve(s). Hand wash, using mild liquid hand soap, hand dishwashing liquid, laundry detergent, or a multipurpose detergent (follow product label instructions). Gently scrub all cushion surfaces, using a soft plastic bristle brush, a sponge or a cloth. Rinse with clean water. Air dry. DO NOT expose to direct sunlight.</p> 	<p>HAND DISINFECTING Instructions: Repeat cleaning instructions, using 1 part household liquid bleach per 9 parts water. Keep the product wet with the bleach solution for 10 minutes. Rinse thoroughly with clean water. Air dry.</p> 
<p>Prepare for MACHINE WASHING or MACHINE DISINFECTING: 1) Remove the cover and close the inflation valve(s). 2) Examine the cushion for damage, and patch any holes (refer to Repair Kit instructions provided with the product). 3) Remove stains or odors as needed: Place the cushion in warm water and a disinfectant cleaning agent. Gently scrub, using a soft plastic bristle brush, and rinse with clean water. 4) Deflate the cushion: Open the inflation valve and roll up the cushion, starting at the corner opposite the valve. After removing as much air as possible, CLOSE THE INFLATION VALVE. 5) Inspect for any sharp edges in the washing machine that may damage the cushion.</p>	<p>MACHINE WASHING Instructions: 1) Place the cushion in a laundry bag and then into the washing machine. When folding the cushion, make sure the air cells face outward. 2) Use a washing machine that has a sanitize cycle. 3) Use warm water 30° - 40°C (85° - 105°F) and laundry detergent or machine- and cushion-safe cleaner/biocide, placed in the detergent dispenser. 4) Gentle cycle: Use a machine setting that does not include a high-speed spin cycle. 5) Rinse thoroughly. Detergent residue may cause air cells to stick together. 6) Remove the cushion from the laundry bag and air or cabinet dry.</p> 
<p>DRYING CABINET Instructions: For optimal drying of the air cells after machine washing and/or disinfecting, open the inflation valve and place the cushion in the drying cabinet with the valve pointing down for 30 minutes. Do not exceed temperatures above 70°C (158°F). HANDHELD DRYER Instructions: For faster drying, use a hand-held hair dryer, hold no closer than 6 in. (15 cm) from the cushion, and use a sweeping motion.</p> 	<p>MACHINE DISINFECTING Instructions: Follow the machine washing instructions and use hot water 60°C (140°F). Use biocide (refer to the manufacturer instructions). Bleach with care: Only use a dilution of 1 part household liquid bleach per 9 parts water. Rinse with clean water.</p> 
<p>Recommended Overinflation Test: After machine washing and/or after using a handheld dryer or drying cabinet, confirm proper functioning: 1) Overinflate the cushion with the supplied hand pump until all air cells touch each other, and close the inflation valve; 2) Allow the cushion to remain overinflated for at least 12 hours; 3) Inspect the cushion for damage or air loss; if necessary, refer to the repair kit instructions provided with the product or contact Customer Care.</p>	

Hybrid Elite Foam Base and Foam Insert

- Warning:** DO NOT machine wash or machine dry the contoured foam base or the foam insert.  
- | | |
|---|---|
| <p>Hybrid Elite Foam Base - Cleaning and Disinfecting: Remove the cover, foam insert and cushion insert.
To clean, wipe gently with a cloth dampened with water and a household detergent, then rinse with cool, clean water.
To disinfect, follow the cleaning instructions, but use disinfectant instead of household detergent. After cleaning or disinfecting, allow the foam to air dry thoroughly before putting the cover on.</p>  | <p>Hybrid Elite Foam Insert - Cleaning and Disinfecting:
To clean, moisten a cloth, then wring to remove excess water. Wipe gently with the cloth and a household detergent, then wipe again with a clean, slightly damp cloth.
To disinfect, follow the cleaning instructions, but use disinfectant instead of household detergent. After cleaning or disinfecting, allow the foam to air dry thoroughly before putting the cover on.</p>  |
|---|---|

AirLITE® Cushion and Convex Insert

- Warnings:** Disinfectants are not effective on porous surfaces such as certain types of foam. If the foam becomes soiled or contaminated, DO NOT use with multiple individuals. Replace soiled or contaminated foam components. DO NOT machine wash or sanitize. DO NOT machine dry.  
- AirLITE Cushion and Convex Insert - Cleaning:** Remove the cover. Wipe gently with a cloth dampened with water and a household detergent. Rinse the cloth and re-wipe. Allow the foam to air dry thoroughly before putting the cover on or replacing the Convex Insert. 

ROHO Cushion Covers: Standard Cushion Cover, Heavy Duty Cushion Cover, HYBRID ELITE Cover, AirLITE Cover

- Before cleaning and disinfecting, remove the cushion from the cover. Remove any hook fastener pieces from the bottom of the cover.
- | | |
|---|--|
| <p>Cover - Cleaning: To Clean the Cover: Machine wash in cold water (30°C) with mild detergent, gentle cycle, or wipe clean with neutral detergent and warm water. Tumble dry low.</p>    | <p>Cover - Disinfecting: To Disinfect the Cover: Hand wash in cold water (30°C) using 1 part household liquid bleach per 9 parts water. Keep the product wet with the bleach solution for 10 minutes, then rinse thoroughly. Or machine wash in hot water (60°C). Tumble dry low.</p>     |
|---|--|

General Safety Information (Applies to all ROHO products listed on this page.)

Warnings:

- Valves **MUST** be closed: **DO NOT** allow water or cleaning solution to enter the cushioning product.
- **DO NOT** machine wash or sanitize or machine dry mattress overlay products.
- Cleaning and disinfecting are separate processes. Cleaning must precede disinfection. Before use by a different individual: clean, disinfect, and check the product for proper functioning and confirm that all package contents are provided with the product. Refer to the operation manual provided with the product.
- Clean the product regularly and when it becomes soiled.
- Use of the following may cause damage: abrasives (steel wool, scouring pads); caustic, automatic dishwasher detergents; cleaning products that contain petroleum or organic solvents, including acetone, toluene, Methyl Ethyl Ketone (MEK), naphtha, dry cleaning fluid, adhesive remover; oil-based lotions, lanolin; ozone gas; ultraviolet light cleaning methods.
- Follow all manufacturer bleach container and germicidal disinfectant safety guidelines. For a list of acceptable disinfectants, contact the equipment provider or local health authority.
- High temperatures accelerate aging and will damage the product assembly. ROHO products are not packaged in a sterile condition, nor are they intended or required to be sterilized prior to use. If institution protocol demands sterilization: First, follow the cleaning and disinfecting instructions; then, open the inflation valve and use the lowest possible sterilizing temperature, but not to exceed 70°C (158°F), for the shortest time possible. **DO NOT** autoclave.
- After cleaning and disinfecting, and before using, check all components for proper functioning and replace if necessary.

DRY FLOATATION Mattress Overlay System


To Clean the Overlay:

Remove all linens and straps. Unsnap overlay sections. Close valve(s) and place each overlay section in a large sink. Hand wash, using mild liquid hand soap, hand dishwashing liquid, laundry detergent, or a multipurpose detergent (follow the product label instructions). Gently scrub all product surfaces, using a soft plastic bristle brush, a sponge or a cloth. Rinse with clean water. Air dry. **DO NOT** expose to direct sunlight.

To Disinfect the Overlay:


Repeat the cleaning instructions, using 1 part household liquid bleach per 9 parts water. Keep the product wet with the bleach solution for 10 minutes. The mattress overlay may also be disinfected by wiping with an alcohol-based disinfectant or with an oxidizing surface disinfectant with a surfactant-based additive. Rinse thoroughly with clean water. Air dry.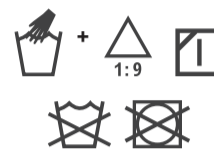

ROHO Reusable Mattress Overlay System Cover


To Clean the Reusable Cover:

Remove the cover from the overlay. Machine wash in cold water (30°C) with mild detergent, gentle cycle, or wipe clean with neutral detergent and warm water. Tumble dry low.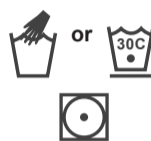

To Disinfect the Reusable Cover:

Hand wash, using 1 part household liquid bleach per 9 parts water. Keep the product wet with the bleach solution for 10 minutes. Quaternary-type disinfectants may also be used in concentrations recommended by the manufacturer. Rinse the product thoroughly. Or machine wash in hot water (60°C). Note: Repeated washing at high temperatures can accelerate deterioration of the cover and is not recommended). Tumble dry low.

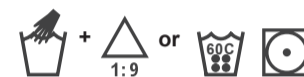Caution: **DO NOT** dry clean the Reusable Overlay Cover.

Care Symbols

Hand wash.	Line dry in the shade.	Bleach (1 part bleach: 9 parts water).	Machine wash, warm, gentle.	Machine wash hot, normal, at the temperature shown.	Use a handheld dryer.
Hand wash cold.	Dry flat in the shade.	Line dry.	Tumble dry low.	Machine wash cold, gentle.	Wipe with a damp cloth.
Do not machine wash.	Do not machine dry.	Do not dry clean.			

ROHO, Inc., 1501 South 74th St., Belleville, IL 62223 U.S.A.
 U.S.A.: 800-851-3449 • Fax: 888-551-3449 • Customer Care: orders.roho@permobil.com
 Outside the U.S.A.: 618-277-9150 • Fax: 618-277-6518 • intlorders.roho@permobil.com
 permobil.com

Products may be covered by one or more U.S. and foreign patents and trademarks, including AirLITE®, CONTOUR SELECT®, ENHANCER®, DRY FLOATATION®, HIGH PROFILE®, HYBRID ELITE®, MID PROFILE®, LOW PROFILE®, QUADTRO SELECT®, ROHO®, Sensor Ready®, and Smart Check®.

Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

ROHO, Inc. is part of Permobil Seating and Positioning.

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© 2018, 2021 Permobil Rev. 2021-03-04
ROHO, Inc. ingår i Permobil Seating and Positioning.