Manual

# Gewa Duocom

**SE, GB, DE, NO, DK, FR, NL**

Abilia AB, Kung Hans väg 3, Sollentuna, Box 92, 191 22 Sollentuna, Sweden  
Phone +46 (0)8-594 694 00 | Fax +46 (0)8-594 694 19 | [info@abilia.se](mailto:info@abilia.se) | [www.abilia.se](http://www.abilia.se)

  
**ABILIA**  
GEWA AB | FALCK IGEL

<b>SE BRUKSANVISNING FÖR DUOCOM</b>	<b>3</b>
<b>GB HOW TO USE DUOCOM</b>	<b>5</b>
<b>DE GEBRAUCHSANWEISUNG FÜR DUOCOM</b>	<b>7</b>
<b>NO BRUKERVEILEDNING FOR DUOCOM</b>	<b>9</b>
<b>DK SÅDAN ANVENDER DU DUOCOM</b>	<b>11</b>
<b>FR MODE D'EMPLOI DU DUOCOM</b>	<b>13</b>
<b>NL GEBRUIK VAN DE GEWA DUOCOM</b>	<b>15</b>

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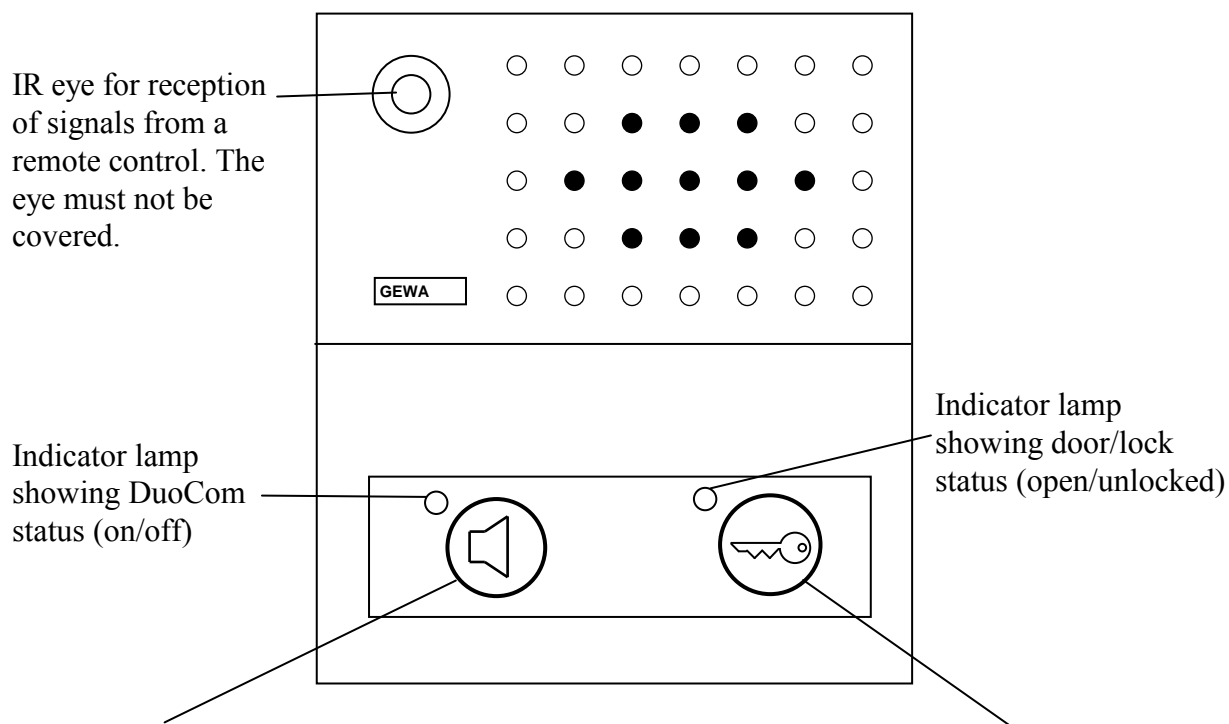
# GB How to Use DuoCom

The figure shows the functions for DuoCom room units.

IR eye for reception of signals from a remote control. The eye must not be covered.

Indicator lamp showing DuoCom status (on/off)

Indicator lamp showing door/lock status (open/unlocked)

## SPEECH BUTTON

- Answer Door

### When Using recorded messages.

- Press once: DuoCom starts. Message 1 is activated consisting of the phrase: Hello, who's there?
- Press a second time: Message 2 is activated consisting of the phrase: Please can you call back later?

\* The phrases are only examples.

## Cleaning

Use only a damp rag with a small amount of washing liquid.

## OPENING BUTTON

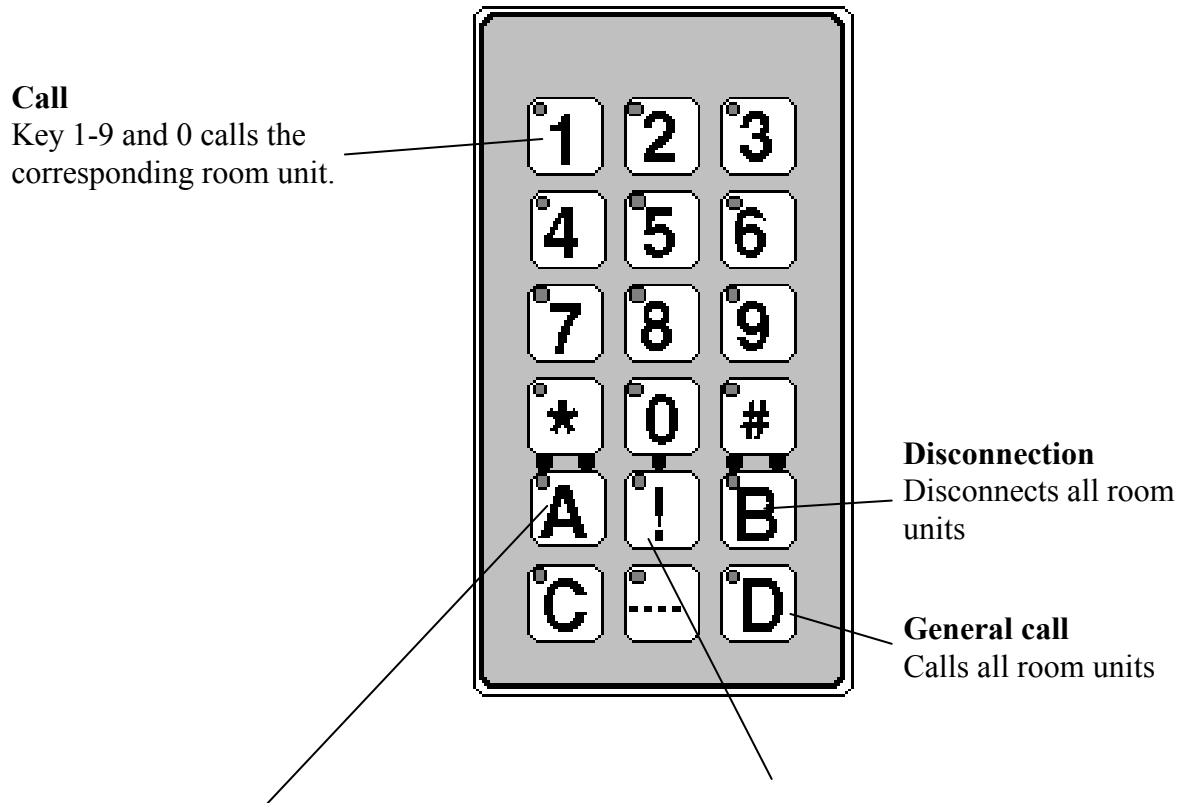
- Lock Release

### With recorded messages and automatic door openers connected.

- **Short push:** Opening of lock. Message 3 is activated consisting of the phrase: Do come in, I'll open the door!
- **Long push:** Opening of lock and activation of automatic door opener. Message 3 is activated consisting of the phrase: Do come in, I'll open the door!

# How to Use Intercom Telephone

If you want to use DuoCom as an intercom telephone an IR transmitter is required. The figure shows the functions for DuoCom as an intercom telephone.

## SPEECH BUTTON

- Answer Door

### When Using recorded messages.

- Press once: DuoCom starts. Message 1 is activated consisting of the phrase: Hello, who's there?
- Press a second time: Message 2 is activated consisting of the phrase: Please can you call back later?

\* The phrases are only examples.

## OPENING BUTTON

- Lock Release

### With recorded messages and automatic door opener connected.

- **Short push:** Opening of lock. Message 3 is activated consisting of the phrase: Do come in, I'll open the door!
- **Long push:** Opening of lock and automatic door opener. Message 3 is activated consisting of the phrase: Do come in, I'll open the door!

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