TurnSafe2

SystemRoMedic

Instructions for use – English

Bruksanvisning – Svenska

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REF

20201010

SWL: 150 kg/ 330 lbs

Table of contents

English	3
Svenska	6
Norsk	9
Dansk	12
Suomi	15
Deutsch	18
Nederlands	21
Français	24
Italiano	27
Español	30
Dimensions	33
Symbols	33

TurnSafe activates and supports users with impaired balance and strength in the legs during transfers from bed to wheelchair or from wheelchair to the toilet. TurnSafe features a large foot plate with an anti-slip surface, adjustable height and lower leg supports which are adjustable both sideways and in height. These features, together with the stable and grip-friendly handle, enable the user to actively participate in the sit-to-stand and transfer.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Check to ensure that the material are free from damage. If there are signs of wear, the product must be inspected and serviced.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Special requirements and conditions

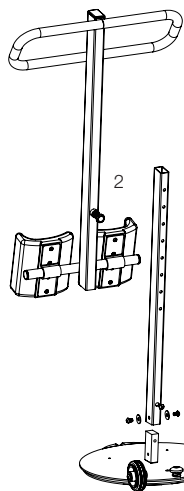
- The caregiver must always keep a foot on the brake while the patient is standing up, sitting down or standing still.
- The caregiver must always perform the transfer slowly and gently.
- TurnSafe must not be used to transport users.
- The patient must always hold the lower part of the support handle.

To transfer with TurnSafe, the patient must be able to:

- support weight while standing
- stand up/sit down and maintain balance (with support)
- grasp handles
- understand instructions

Assembly


TurnSafe consists of three sections that can be easily disassembled for storage and transport.

The lower mast is secured to the footplate with three screws (1). Insert the upper mast with leg supports and support handle, and push into place (2).

Adjustment of leg supports and support handles

The support handle and the leg supports can be set at eight different positions by pressing the safety buttons and moving to the desired height. Ensure that the safety button is properly secured in the hole.

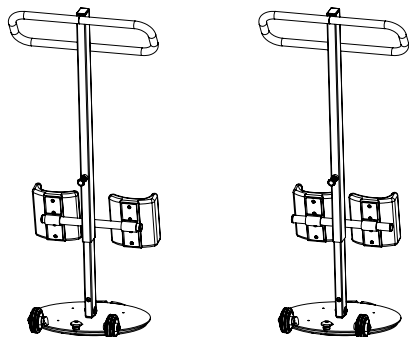The leg support can also be set in different sideways positions. Make sure that the leg supports are symmetric to each other.

The footplate and placement of the feet

The feet is to be placed centered och each side of the foot plate.

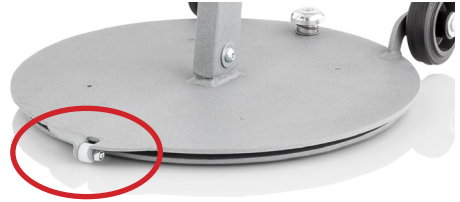
The weight of the user must be inside the area of the foot plate, i. e the user must be in a standing position.

The weight and gravity point of the user may never be outside of the foot plate!

Using

The product may only be used on a level floor and never used in the shower.


Place TurnSafe right in front of the user. The user moves the feet to the sides and by slightly tilt TurnSafe forward on the small wheel in the front TurnSafe can easily be moved in to place between the users feet. In this position the user can easily place the feet on the foot plate.


Adjust the leg supports to the height just below the knees of the user. (Approximately 2 cm/ 1 inch)

Make sure that the surface that the user is to move to is placed as close as possible.

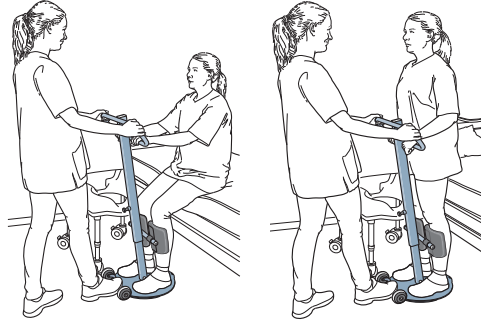The caregiver has a firm grip on the upper handles and one foot on the brake until both the user and the caregiver are ready to preform the turn/transition.

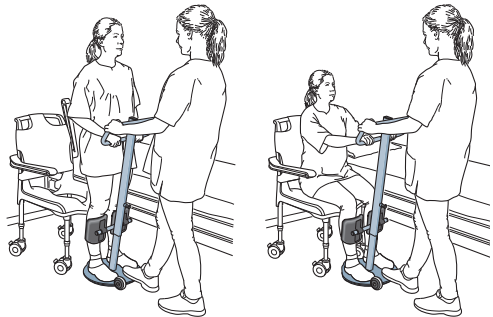The user grasps the lower handles and stands up straight. The caregiver can now let go of the brake and slowly turn the user around.

When the other surface is straight behind the user the caregiver again places one foot on the brake and the user can sit right back down.

Care of the product

The surface of the product can be cleaned with warm water or a soap solution. If disinfection is needed, 70 % ethanol, 45 % isopropanol or similar should be used.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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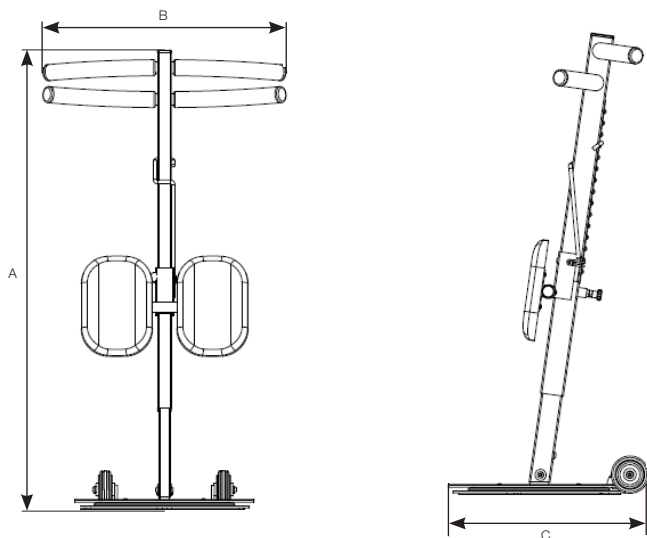
Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

SystemRoMedic®

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

 Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.