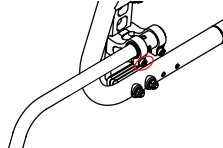
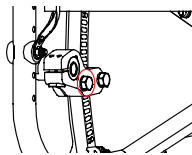
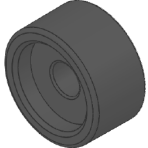
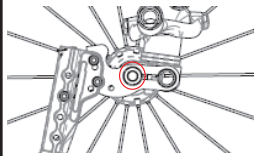
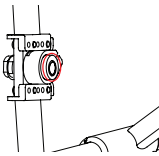
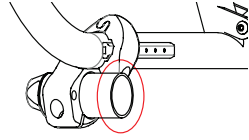
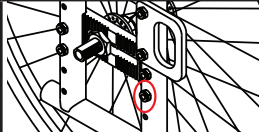
## Folder Adaptor Bar Insert Instructions

*NOTE: Refer to the separate directions provided for folder bar mounting.*

### Insert Use

If the quick-release axle tips are recessed or not accessible so the standard folder adaptor bar end cap are not able to engage them for the wheelchair, it may be necessary to use one of the provided inserts. Chair attachment point should be as close to rear wheel axle height as possible. Use the chart below and provided screws to attach the correct insert to the threaded inserts in the folder adaptor bar end caps.

Insert Image	Insert Diameter	Attachment Image	Chair Attachment
	0.39" Inside Diameter		Quickie 2 Lite [with Anti-Tippers] bolt head
	0.67" Inside Diameter for SAE Bolt		TiLite Aero X / 2GX [No Axle Tips] - hex bolt head
	20 mm Inside Diameter for Metric Bolt		
	22 mm Inside Diameter		Quickie / Sopur Neon / Xenon - over axle plug
	1.0" Inside Diameter		TiLite [No Axle Tips] - over axle plugs
	1.25" Inside Diameter		TiLite TX Series 2 - over axle/camber tube section
No Insert	0.5" Inside Diameter [of end caps]		Quickie 2 [with One-Arm Drive / No Axle Tips] - axle plate nut

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