






Instructions for use TOPRO Olympos ATR


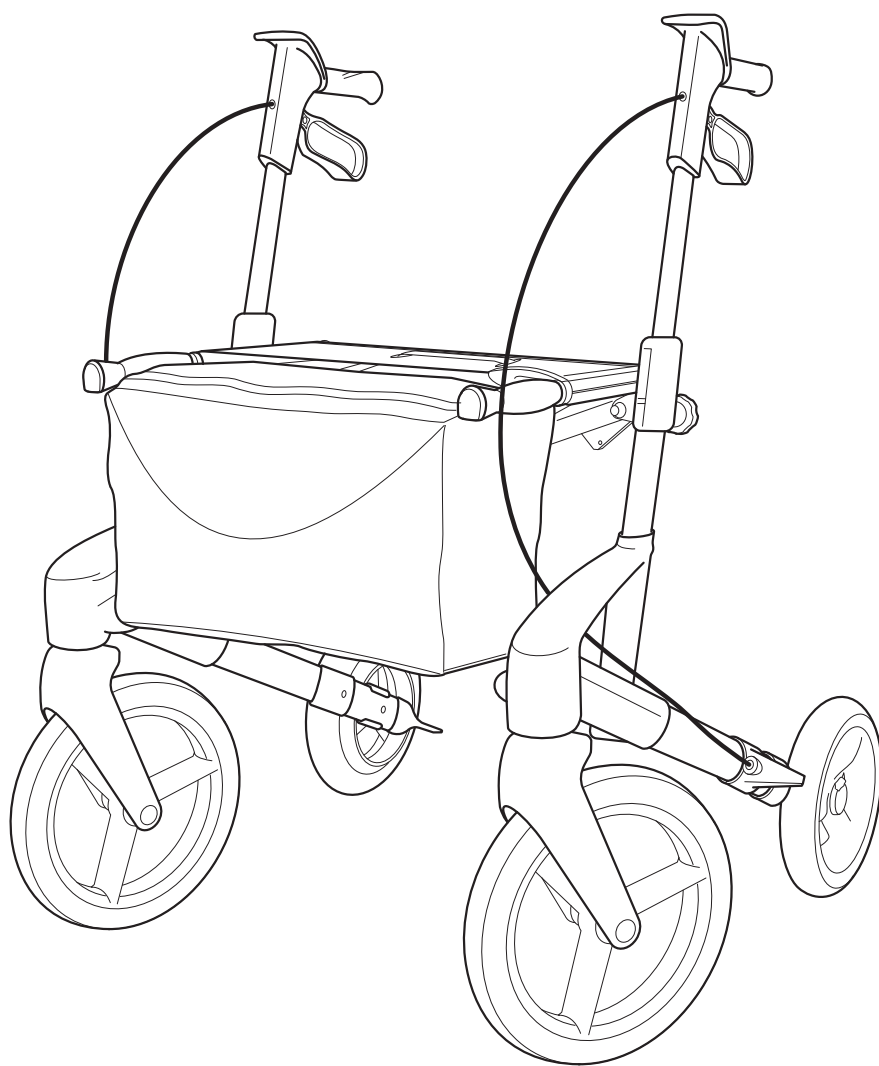






1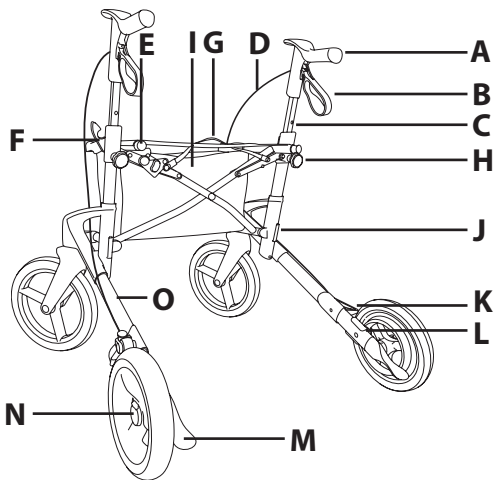



TOPRO OLYMPOS ATR M	815600
TOPRO OLYMPOS ATR M Slim	815620
TOPRO OLYMPOS ATR S	815610
TOPRO OLYMPOS ATR S Slim	815630

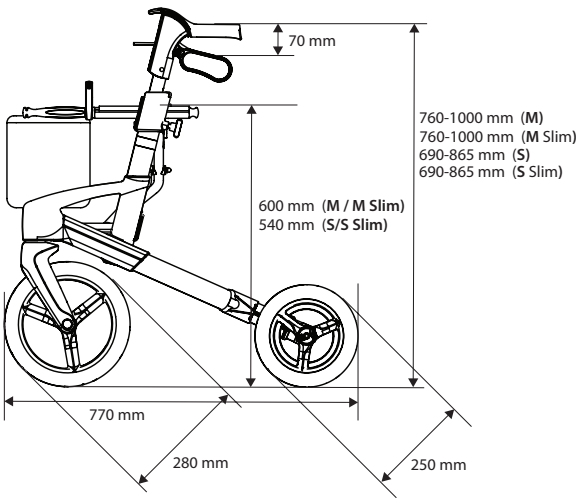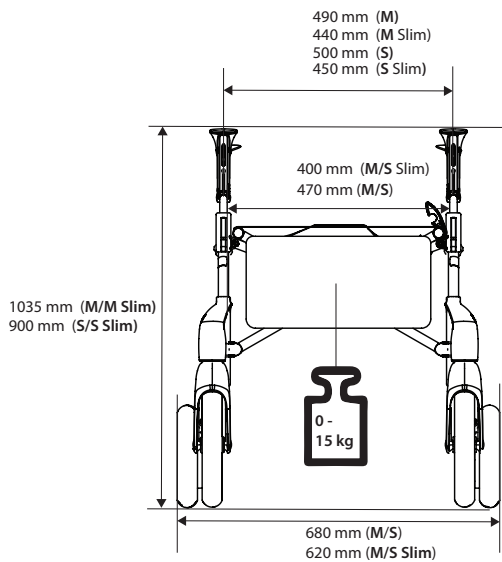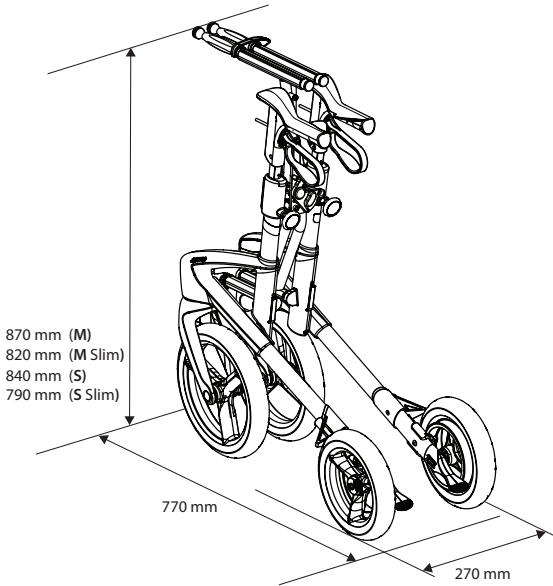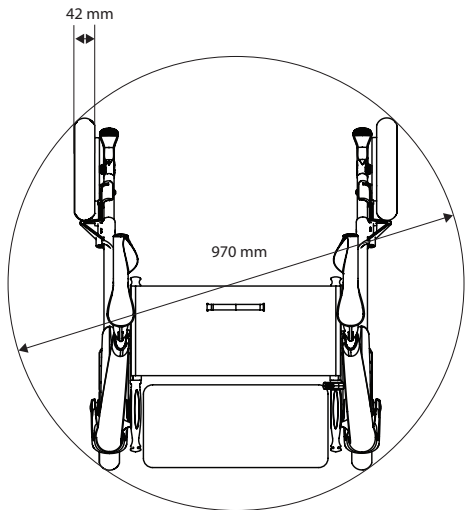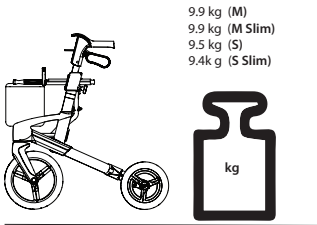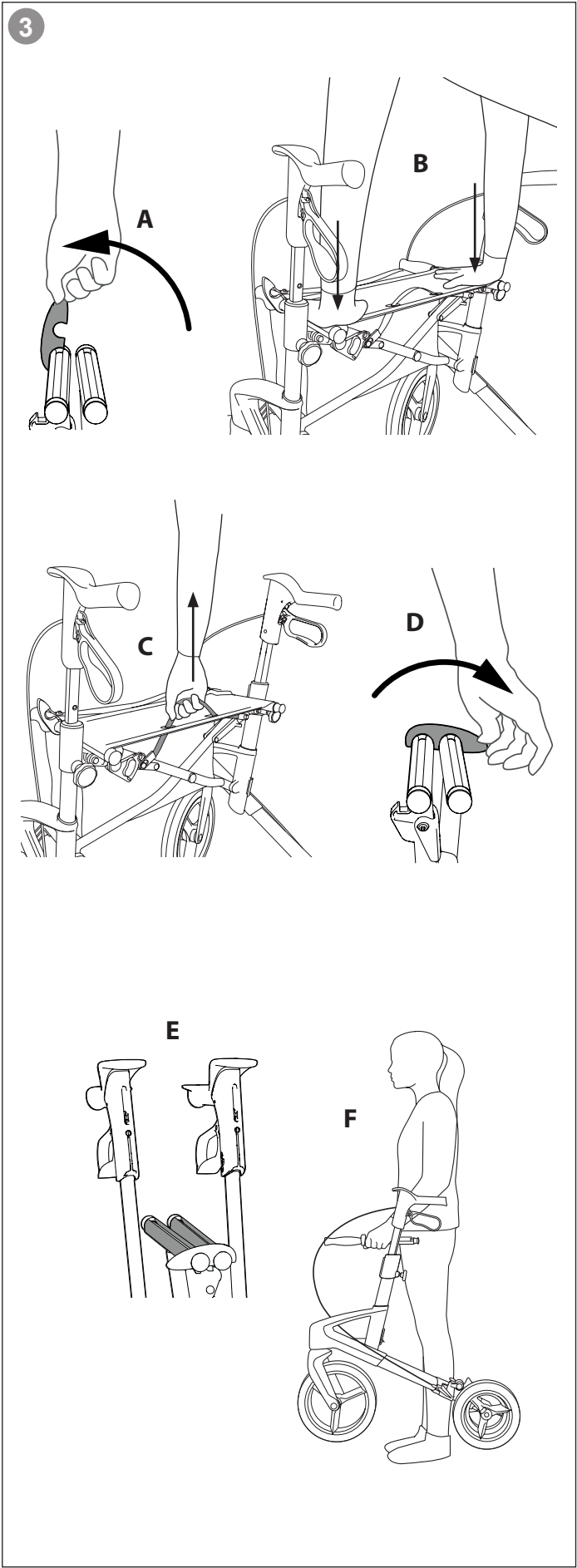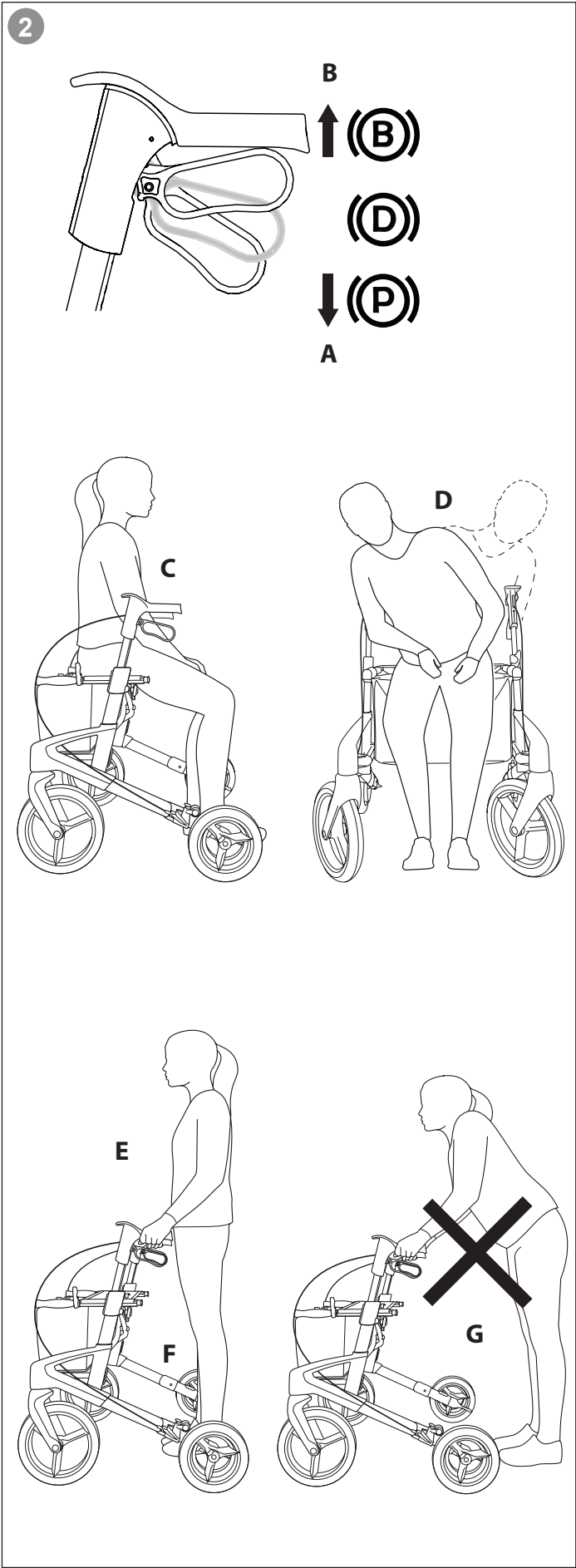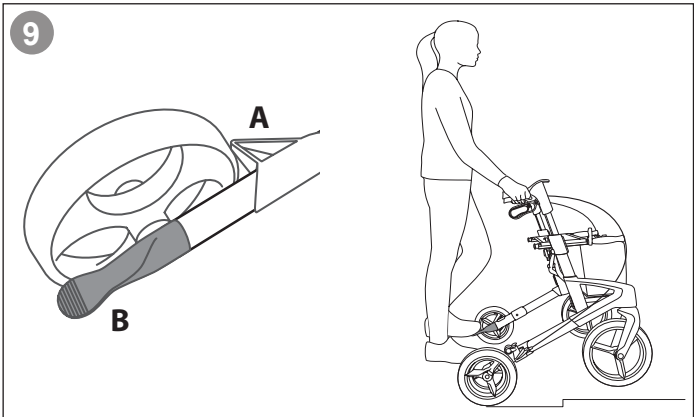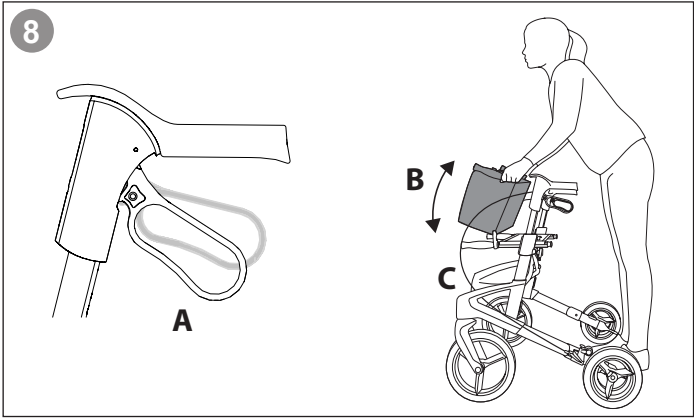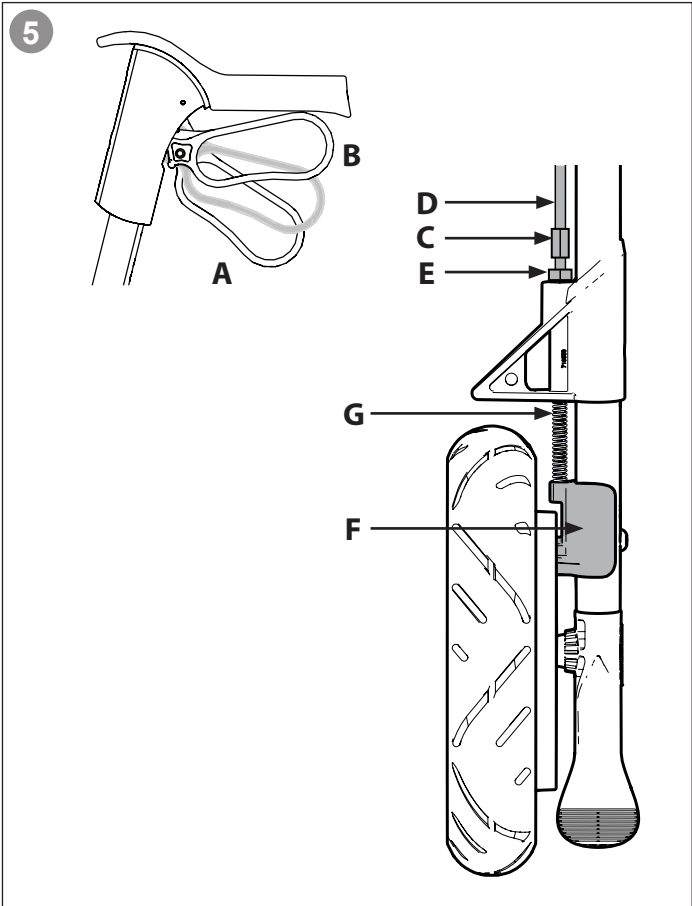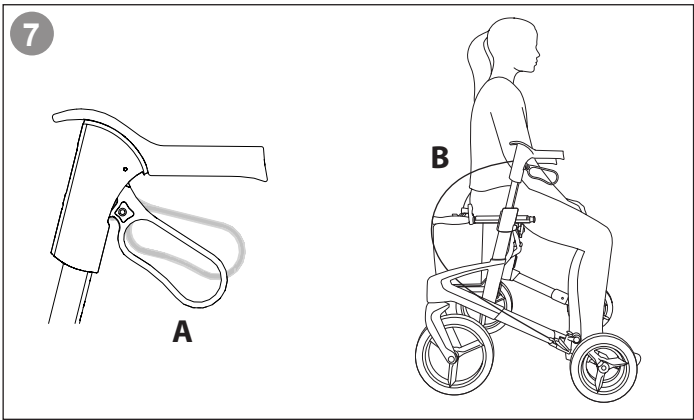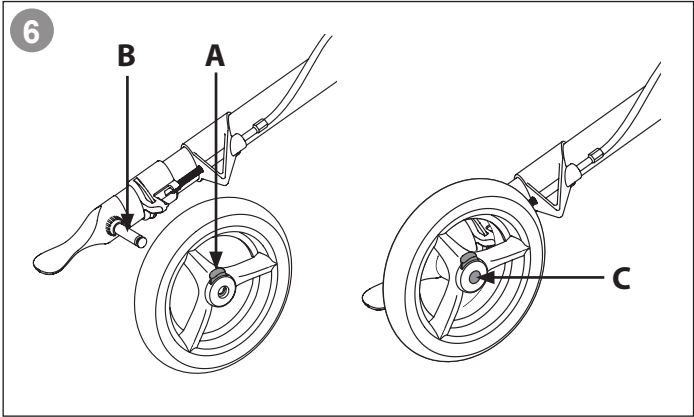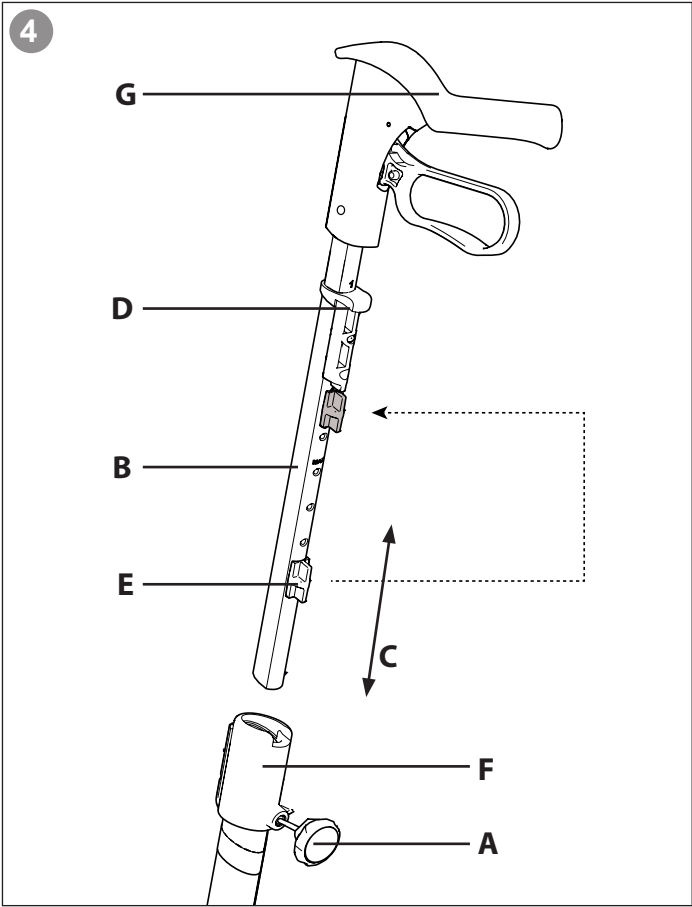10

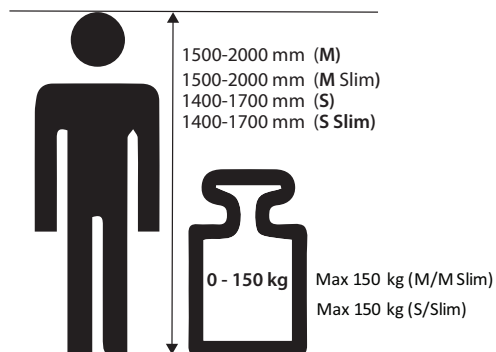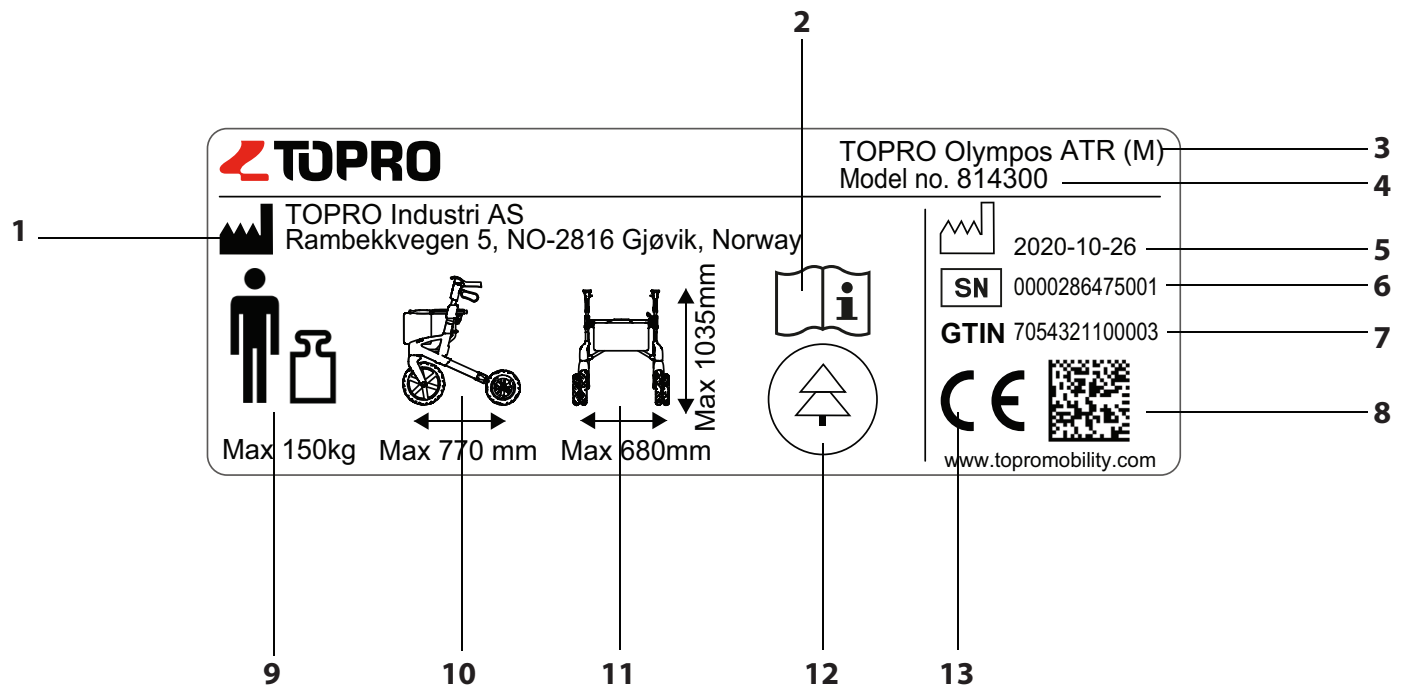Congratulations with your choice of a new TOPRO rollator

Every TOPRO product is developed, designed and made in Norway. The product is declared to be in conformity with Medical Device Regulation (EU) 2017/745. It is tested and approved according to EN ISO 11199-2:2005. On receipt of the goods, please check your product. If you have any queries, please contact your mobility shop or dealer immediately. If you have problems reading this user manual, an electronic version is available on www.topromobility.com.

Intended use

The TOPRO Olympos ATR shall give support to users with reduced balance and/or reduced walking ability. It is designed to be pushed, not pulled. The intended user is an adult. Height and weight restrictions apply.

Contraindications: The product is designed and approved for outdoor use but can also be used indoors. The rollator is intended to be an aid for walking on roads and paths. It is not suitable in rough and/or steep terrain, neither for stairs. The product is not suitable for persons with low strength in arms, with very poor balance, or with considerable cognitive disabilities.

To get to know the rollator and learn how to use it correctly, training by a skilled person is advised.

The grey numbered circles in this user manual refers to the numbered information graphic and illustrations inside the cover

1 Get to know your TOPRO product

A	Handlebar	H	Lock wheel for the handle
B	Brake handle	I	Shopping bag
C	Handlebar tube	J	Side frame
D	Brake wire	K	Edge guard
E	Seat bars/carrying handle	L	Brake block
F	Lock clamp	M	Tilt function
G	Seat with strap for folding	N	Detachable rear wheels
		O	Product label

2 Safety

When folding and unfolding, be aware so you do not hurt your fingers.

Be aware not to put your fingers in open holes or between stationary parts.

Make sure that the lock wheel for height adjustment for the handles is locked tight.

When stationary or parked, make sure the parking brakes on both sides are in locked position **A**.

Do not use the rollator on stairs. Be careful when descending kerbs.

Check the brakes before use every time, and that the rollator locks in unfolded position.

Do not use the device to transport heavy loads or persons.

Be aware that some parts of the rollator might feel hot or cold when exposed to extreme temperatures.

Do not modify the product as this might put your safety at risk and the guarantee will become void.

Before sitting on the seat **C**, make sure the parking brakes on both sides are in locked position **A**.

To avoid the risk of falling, do not lean or bend sideways or over the back of the rollator whilst in the seated position. Do not tilt the rollator too much sideways **D**, to avoid the risk of falling. For the same reason, bags etc. must never be hung on the handlebars.

To give the right support and safety, walk straight and make sure the rollator is kept as close to your body as possible **E**. Walk between the rear wheels **F**, not behind them **G**.

Be aware that the rollator may roll faster than you when walking downhill, which may put you in risk of falling. Apply the brakes to adjust the speed **B**.

Be careful when wearing flared skirts and trousers as they could get caught in the wheels, which could lead to falling. Wear sturdy footwear to avoid injuries.

For safety reasons it is prohibited to sit on the rollator during public transportation. Store your rollator securely and make sure the parking brakes on both sides are in locked position **A**.

Keep the rollator away from fire and hot items.

Do not climb on the rollator.

3 Unfolding / folding

Unfolding: Release the lock clamp positioned at the seat bars **A**. Press the seat bars down and out. The side frames will slide apart. Press down the seat bars **B**. You will hear a clicking sound. The rollator is now locked in its open position.

Folding: Pull the strap on the seat **C** and press the side frames together. Lock the clamp **D** over the seat bars.

Carrying the rollator: The rollator must not be carried by the seat strap or the brake wires. Fold the rollator together as shown on the illustration **E**. Carry the rollator by the folded seat bars **F**.

4 Handles / memory function

Height adjusting: Unscrew the lock wheel **A** (approx. 2 turns). Adjust the handlebar tubes to a suitable height then pull them slightly up and down **C** until you hear a clicking sound.

Tighten the lock wheel **A** again. Adjust both handlebars to the same height by comparing the number scale on each side. It is recommended that the handlebars are set to the same height as your wrists when your arms are hanging alongside your body.

Memory Function: Decide on the height of the handlebars (see number scale). Count visible holes at the back of the handlebar tubes. Unscrew the lock wheel **A**. Pull the tube **B** out of the frame. The fastening piece **D** will now follow the handlebar tube. Move the fastening piece **D** on the tube **B** so the number of holes corresponds to the number counted earlier. Fit the red memory chip **E** on the handlebar tube in the first available hole under the fastening piece **D**.

Push the handlebar tube **B** down and into the side frame **F** until the fastening piece is locked. Screw the lock wheel **A** half in. Pull the handlebars **G** up until they stop, and then tighten the lock wheel **A** properly. From now on, when the handlebars are in the lowest position, they will stop in the pre-set position automatically when pulled up.

5 Brakes

Make sure the brakes are working before each trip. Each brake handle **A** works on each of the rear wheels. Do not drive or push the rollator while the parking brakes are in use! **Parking brakes:** Push both brake handles **A** down until they lock. Both rear wheels are now locked. Squeeze both brakes handles upwards to release the parking brakes. **Driving brakes:** Pull both brake handles **B** upwards to reduce speed. *Do not use continuously while the rollator is in motion.*

Adjusting / checking: Loosen the nut **C**. Tighten the brake wire **D** with the adjustment screw **E**. Adjust the brake block **F** as close as possible to the inner drum brake, and then tighten the nut **C**. Turn the wheel around to check that the brake block is not touching the wheel. **Maintaining:** Clean the brake block **F** and the brake spring **G** frequently. The wheels and brakes can be replaced if they are worn out or damaged.

6 Rear wheels

If you want to remove the rear wheels for transportation or replacement, follow these instructions:

Removal: Loosen the brake lever. Push down the red button on the hub **A** and slide the wheel off the axle **B** while still pushing the button down.

Attachment: Loosen the brake lever. Slide the wheel onto the axle **B**. Push the wheel as far as possible inwards on the axle to make the wheel snap onto the axle. You will hear a "click" when the wheel is in place. The wheel is securely fastened when the axle is completely visible in the middle of the hub **C**. Gently pull the wheel to check that it is securely fastened to the axle.

After the new wheel has been applied to the rollator, make sure that the brakes are functioning properly.

Readjust them if necessary (see section Brakes).

7 Seat

The parking brakes must be on before using the seat. Push both brake handles downwards until they lock **A**. Both rear wheels are now locked. Sit with your back towards the bag **B**.

8 Shopping bag

The bag holds up to 15 kilos and can be washed gently at 60 °C.

Mounting: Apply the parking brake **A**. Lift the bag onto the carrying handles **B**. When it is placed correctly, you will hear a click. Ensure that the brake wire still runs unrestrained along the frame **C**.

Removal: Apply the parking brake. Place your hands towards the front of the bag, on both sides. Lift the bag up while pressing it slightly together.

9 Edge guard / tilt function

Edge guard: The edge guards **A**, which are in front of the rear wheels prevent the rear wheels from catching on corners, door frames etc.

Tilt function: To use the tilt function use the front of the foot and press down on the tilt pedal **B**, which is beside the rear wheel. This helps to tilt the rollator over thresholds, edges etc. Please note: Take care when descending kerbs.

TOPRO Ergo Grip

TOPRO Ergo Grip offers ergonomic grip and support surfaces. The front of the handles gives you a better gripping position when getting up from a chair or couch.

Always make sure that the parking brake is locked when you do this. The hand grip also offers extra support while standing. When pushing the rollator, your hands should always be kept in the original grip position in order to make the best possible use of the brakes

Transport

The rollator should be transported in the folded and locked position (see paragraph *Unfolding / Folding*). Ensure that the lock clamp is on, so that the rollator cannot unfold unintentionally. Be careful when loading the rollator into a vehicle, giving attention to securing of the item.

Storage

The rollator should be stored in an upright position. If stored outdoors, then under cover to protect it. Do not leave your rollator unattended. Do not put heavy items on top of the rollator during storage.

Cleaning

Machine washing in the dish cabinet used at aid centers can be used. Do not use abrasives. Avoid flushing directly against bearings (wheels, front forks).

Disinfection

Disinfection must only be carried out by authorized personnel and by wearing sufficient protective equipment. The surface of the rollator must be cleaned with a disinfectant containing 70–80 % ethanol. We advise against disinfectant that contains chlorine and phenol.

The manufacturer cannot be held responsible for any damage or injuries that might occur by the use of harmful disinfectant or disinfection carried out by unauthorized personnel.

Inspection / maintenance / re-use

It is recommended that inspection and maintenance is carried out at least once a year (frequency is determined by how and how often the rollator is used). Check the following items: Frame, screws, handles, handlebar tubes, brakes, brake parts, wheels, seat and accessories. This also applies when the rollator is to be prepared for re-use. Please see detailed instructions in this user manual about maintenance of brakes and cleaning/disinfection.

Material / recycling

The rollator is made of plastic-coated aluminium pipes, plastic couplings and material made of plastic and polyester. There are ball bearings in all wheels and forks. Most rollator parts can be recycled. Dispose the rollator and its packaging according to the applicable regulations in your country. For information, please contact your public authorities.

Warranty

TOPRO OLYMPOS ATR is guaranteed free for faults and defects for 7 years. Damage caused by incorrect use or parts that are exposed to natural wear and tear (e.g. brake block, brake wire, wheel, bag, seat and grip) are exempt from the 7-year warranty. For repairs during the warranty period, please contact your local mobility shop or dealer. The warranty is void if unauthorised spare parts and accessories have been or are being used on the product. The expected life span of the product is estimated to be 10 years, provided that it is used in the correct manner according to this user manual and its safety and maintenance instructions.

10 CE labelling

- 1 - Manufacturer
- 2 - Consult Instructions for use
- 3 - Model name
- 4 - Model number
- 5 - Date of manufacture
- 6 - Serial number
- 7 - Global Trade Item Number
- 8 - GS1 DataMatrix
- 9 - Maximum user weight
- 10 - Maximum length of rollator
- 11 - Maximum width / height of rollator
- 12 - Intended for outdoor use
- 13 - CE marking

Accessories

	TOPRO Item No.
Back support with padding (67 cm)	815270
Back support with padding (75 cm)	815272
Tray (NB: not for size S)	815212
Anti-slip mat for tray	814059
Crutch holder	814040
Bottle holder	814043
LED lamp with clip-on and USB plug	815260
Lock	814023
Umbrella with attachment (black)	814068
Off-road wheels for IBS, set of four	815304
Studded tyres for IBS, pair of two rear wheels	815332
One-handed brake	814026
Guiding handle for the visually impaired	814014

Accessories mounted on the rollator may influence stability, we therefore advise you to use them carefully. The accessories are to be ordered separately to enable a TOPRO OLYMPOS ATR to be customised to the needs of an individual user. Contact your mobility shop, dealer or TOPRO for an updated overview of the accessories or visit our homepage www.topromobility.com.

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TOPRO Mobility AB