

IBIS HD

För Förskrivare/ Tekniker

- Förskrivarinformation
- Serviceanvisning



Första delen

1. Användningssmöjligheter	3
2. Service	4
3. Declaration of conformity	5

Därefter följer

Användar manualen och Snabbguiden

Användnings- möjligheter

IBIS kan användas direkt mot TFT skärm eller s.k "platt-TV".
En splitbox (extra tillbehör) gör att man också kan ansluta den och använda den med stationär dator.



IBIS HD mot TFT skärm



IBIS HD mot "platt-TV"



IBIS HD mot stationär dators TFT skärm

Service

IBIS har få delar, vid service är oftast skicka det lättaste. Ring oss för samråd av lämplig åtgärd.

Vid fotens fäste använd Insexnyckel



Lampa—vid byte 2 st torxs kruvar, 2 st stjärnskruvar



En servicefilm visar hur man kan byta tex kontakter.



DECLARATION OF CONFORMITY

Product:

Model	IBIS HD RED	Type no.:	967000
Model	IBIS HD BLUE	Type no.:	967001
Model	IBIS HD GREY	Type no.:	967002
Art:	CCTV-low vision equipment		

Manufacturer:

Name	TAGARNO AS
Address	Sandøvej 4
ZipCode/City	DK-8700 Horsens
Country	Denmark
Phone	+45 7625 1111
Fax	+45 7625 1199

Description:

TAGARNO AS hereby declare that the products listed above, consisting of a camera unit and a 12V power adapter, to which this declaration relates, are in conformity with the essential requirements of the Directive (COUNCIL DIRECTIVE 93/42/EEC of 14 June 1993 concerning medical devices) and the following Standards and Normative Documents: NS-EN 12182.
In addition, a medical risk analysis has been carried out according to ISO14971:2007(E)

The products comply to above mentioned requirements according to the following harmonised standards:

EMC:	EN60601-1-2
Safety:	EN60601-1

The declaration is issued by:

Manufacturer	TAGARNO AS
Date	November 1 st 2011
Signed	

General Manager, Klaus Beck



EN60601-1