FUTURA HP

SV BRUKSANVISNING

NO BRUKSANVISNING

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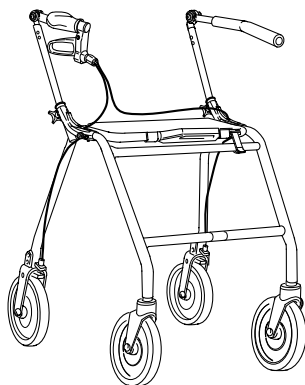
GB OPERATING INSTRUCTIONS

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FR MANUEL D' UTILISATION

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FUTURA HP

SV BRUKSANVISNING

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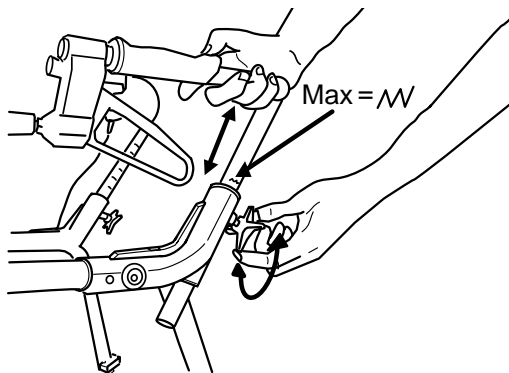
FUTURA HP

GB HANDLEBARS

For adjustment; Loosen the wing, adjust the handle to suitable height and tighten the wing properly. Therapist may suggest different heights depending on specific need of the client.

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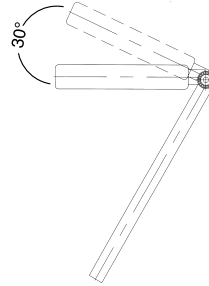
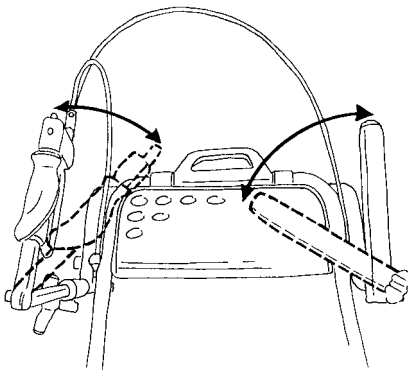
GB HANDLEBARS

The handles can be angled maximum 45° towards the center of the walker. The outer position of the handles must be parallel with the frame. Handles should be placed horizontally maximum 30° upwards

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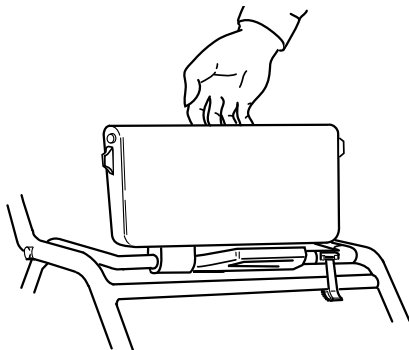
GB FOLDABLE SEAT AND FOLDING

The seat is foldable for increased space between the handles. When the seat is folded up the blockage will drop out of position, this does not effect the function.

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GB MAINTENANCE – SAFETY

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the rollator with solvent-free household cleaner. Keep the wheels clean.
- **Ensure that screws and adjustments** are properly tightened.
- Lubricate the wheels regularly with grease.
- **Immediately contact the nearest dealer if a fault should appear. Never use a defective walker.** Do not attempt repairs yourself!

N.B. Your walker should only be used as a walking aid! User's max. weight: 275 lbs. (125 kg, 20 stones.) Dolomite will **not** accept responsibility for adaptations, faulty repairs, etc, that are outside their control.

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DOLomite AB
AQUATEC GmbH & Co.KG
MOBITEC Mobilitätshilfen GmbH