Invacare® Bedco

en	Backrest Support User Manual	1
da	Sengeryglæn Brugsanvisning	4
fi	Selkänoja Käyttöohje	6
no	Ryggstøtte Bruksanvisning	9
sv	Sängryggstöd Bruksanvisning	11

I General

en

I.1 About this manual

This user manual contains important information about the handling of the product. In order to ensure safety when using the product, read the user manual carefully and follow the safety instructions.

All indications of right and left are based on a user lying on his back in the bed.

Please note that there may be sections in this user manual, which are not relevant to your product, since this manual applies to all existing modules (on the date of printing).

If a problem should arise in connection with the delivered product, please contact your *Invacare®* supplier. An address list is shown on the back side of this manual.

Symbols in this manual

In this user manual, warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

I.2 Intended Use

The backrest support is indicated for persons, who need support to raise up into a sitting position or who must not lay flat for medical reasons.

The backrest support is intended for use in home care and nursing home environments. It can be mounted on standard private beds or care beds.

The backrest support is not intended for children under 12 years age or psychiatric patients.

I.3 Service life

The expected service life of this product is eight years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

Yes, you can.®

2 Safety

2.1 General warnings

WARNING!

Risk of injury


Improper use of this product may cause injury or damage.

- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment.

Continued use of the product with damaged parts could lead to the product malfunctioning, causing injury to the user and/or caregiver.

- Check ALL components and carton for damage, and test components before use. In case of damage or if the product is not working properly, contact a technician or Invacare for repair.

2.2 Labels and symbols on the product

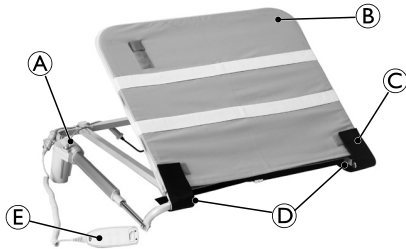- (A) Motor with identification label
- (B) Backrest
- (C) Pivot Joint Protector
- (D) Pivot Joints
- (E) Hand Control

Max. User Weight

Manufacturer

WEEE Compliant

Alternating Current

Direct Current

Type B Applied Part

Insulation Class II

EIP Pollution Control Mark (China ROHS)

C-Tick (Australian EMC)


XXXXX

Read Manual

3 Usage

3.1 Installing Backrest Support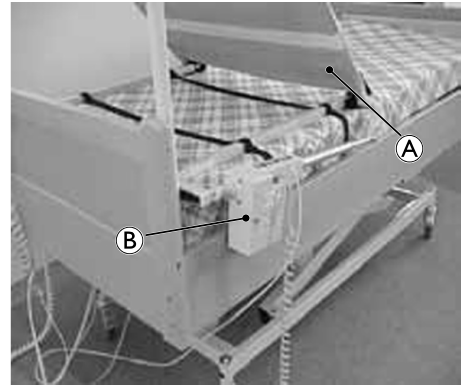

1. Place the Backrest Support (A) on the head end of the bed. The backrest can be used on top or under the mattress depending on the mattress type in use.
2. Place motor (B) beside mattress.

3.2 Placing Backrest Support on Bed


CAUTION!

Risk Of Injury

To avoid injury from product misuse:

- Ensure straps are free from slides and other moving parts.
- Periodically check that the tension and placement of the backrest support straps remains intact. Adjust as necessary.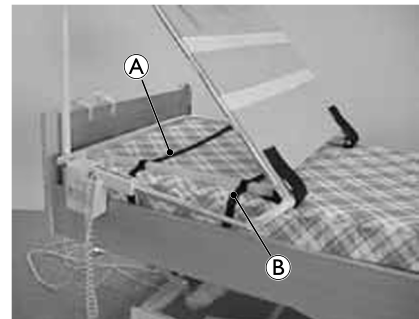

1. Place the first strap (A) as close as possible to the Backrest Support hinges.
2. Place second strap (B) as close as possible to head end of bed.
3. If possible, wrap straps around both bed frame and mattress, then tighten. Be careful not overtighten straps.

3.3 Adjusting Backrest Support Angle


WARNING!

Risk Of Injury

To avoid injury due to tripping hazards, and/or entangled, pinched, or severed cords:

- Ensure that all cord(s) are routed and secured properly.
- Keep all moving parts free of obstruction (i.e. blankets/sheets, tubing, wiring, cords, etc.) during operation.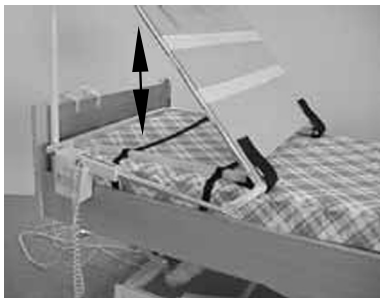


1. Press the up ↑ and down ↓ arrows on the hand controls to adjust the angle of the Backrest Support.
2. Release button when desired angle is reached.

3.4 Changing Motor Position

WARNING! Risk of Injury

To avoid injury from spinning motor:

- Do not rotate motor unit more than one complete turn.
- Ensure cable is not too tight.
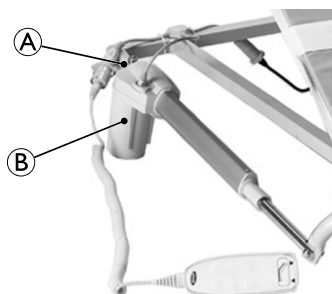


The motor can be turned to four different positions:

1. Remove bolt/pin A from end of motor B unit.
2. Rotate motor B unit to the desired position and carefully replace the bolt/pin A.

3.5 Mounting Motor on the Opposite Side

The motor can be mounted either on the left or on the right side of the Backrest Support.

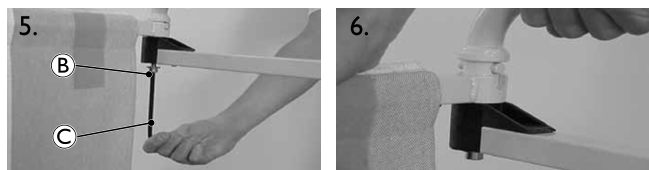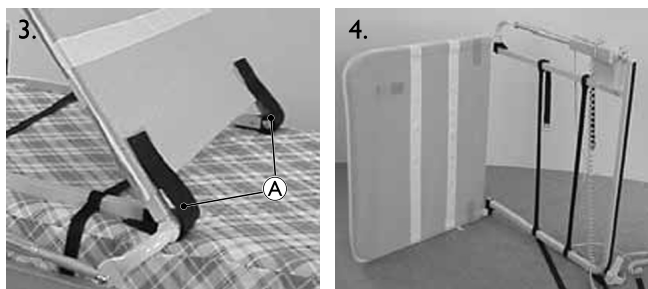1. Remove the Backrest Support from the bed.
2. Using the hand control, raise Backrest Support to the highest position.
3. Remove the pivot joint protectors A.
4. Place the Backrest Support on side opposite of motor.
5. Loosen screw B with 6 mm Allen wrench C. Loosen screw only as much as necessary to rotate arm freely.
6. Rotate arm 180 degrees.

7. Reverse steps 3-5 to reassemble Backrest Support.
8. Reinstall Backrest Support onto bed. Refer to 3.2 Placing Backrest Support on Bed, page 2.

3.6 Accessory Bag

The optional accessory bag is offered for storing the hand control and power supply for the Invacare Bedco Backrest Support.

4 Maintenance

4.1 Cleaning

WARNING! Risk of Injury or Death

To avoid electrocution:

- Do not perform service or maintenance on any electrical parts on this product unless properly trained.
- Do not use water or lubrication on any electrical components.

IMPORTANT!

To avoid damaging the product:

- DO NOT use abrasive cleaners to clean the canvas cover.
- DO NOT tumble dry.

- Canvas cover is machine washable at a max temperature of 60°C.
- With the exception of electronic components, other parts on the Backrest Support can be washed using a damp sponge or cloth.

4.2 Lubrication

If necessary, pivot joints can be sprayed with a silicone based lubricant.

5 After Use

5.1 Disposal

When the product can no longer be used, the product, accessories and the packaging materials should be sorted correctly.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

5.2 Reuse

WARNING! Risk of Injury

To avoid injury when transferring the product to a different user:

- Always have a dealer inspect the product for damage prior to transfer.
- If damage is found, do not use the product. Instead contact dealer or Invacare for service.

6 Technical data

6.1 Product Parameters

Mechanical Parameters

	Bedco 750	Bedco 900	Bedco 1050	Bedco 1200
Total width	800 mm	950 mm	1000 mm	1200 mm
Backrest width	700 mm	850 mm	1000 mm	1200 mm
Backrest Angle	70°			
Weight	9.5 kg	10 kg	10.5 kg	13 kg
Maximum load ¹	70 kg			77 kg
Maximum user weight	135 kg			150 kg
Washing Temperature	60 °C			

¹ On the middle of the backrest support.

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© Invacare Corporation. All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare AB
Invacare Rea AB