# Aquatec® ORCA / ORCA F / ORCA XL

en	Bath lift User Manual . . . . .	3
de	Badewannenlifter Gebrauchsanweisung . . . . .	27
fr	Élévateur de bain Manuel d'utilisation . . . . .	53
es	Elevador de bañera Manual del usuario . . . . .	79
it	Sollevatore da vasca Manuale d'uso . . . . .	105
nl	Badlift Gebruiksaanwijzing . . . . .	131
pt	Elevador de banheira Manual de utilização . . . . .	157
da	Badekarslift Brugsanvisning . . . . .	183
no	Badekarheis Bruksanvisning . . . . .	207
sv	Badkarslyft Bruksanvisning . . . . .	233
cs	Vanový zvedák Návod k obsluze . . . . .	257

This manual **MUST** be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

**AQUATEC**

**Yes, you can.®**

©2017 Invacare® Corporation

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

---

# Contents

---

This manual MUST be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

<b>1 General</b>	<b>4</b>
1.1 Information about the user manual	4
1.2 Symbols in this user manual	4
1.3 Warranty	4
1.4 Compliance	4
1.5 Intended use	5
1.6 Service life	5
<b>2 Safety</b>	<b>6</b>
2.1 Safety information	6
2.2 Safety information on electromagnetic compatibility	6
2.3 Labels and symbols on the product	7
<b>3 Setup</b>	<b>9</b>
3.1 Safety information	9
3.2 Scope of delivery	9
3.3 Setting up the bath lift	10
3.4 Charging the battery	13
<b>4 Usage</b>	<b>15</b>
4.1 Safety information	15
4.2 Controlling the bath lift	15
4.3 Bathing	16
<b>5 Transport</b>	<b>18</b>
5.1 Safety information	18
5.2 Removing the bath lift	18
<b>6 Maintenance</b>	<b>20</b>
6.1 Maintenance and servicing	20
6.2 Cleaning and disinfection	20

<b>7 After Use</b>	<b>22</b>
7.1 Storage	22
7.2 Re-use	22
7.3 Disposal	22
<b>8 Troubleshooting</b>	<b>23</b>
8.1 Identifying and repairing faults	23
<b>9 Technical data</b>	<b>25</b>
9.1 Dimensions and weight	25
9.2 Electronic data	25
9.3 Materials	26
9.4 Environmental Parameters	26

# 1 General

## 1.1 Information about the user manual

This user manual contains important information about the handling of the product. In order to ensure safety when using the product, read the user manual carefully and follow the safety instructions.

The position specifications (A, B, C, etc.) in the handling instructions always refer to the preceding figure.

## 1.2 Symbols in this user manual

In this user manual, warnings are marked with symbols. The title next to the warning symbol indicates the level of danger.

### WARNING

Indicates a dangerous situation which could lead to death or serious injury if not avoided.

### CAUTION

Indicates a dangerous situation which could lead to minor injuries if not avoided.

### IMPORTANT

Indicates a dangerous situation which could lead to damage if not avoided.

Useful tips, recommendations and information to ensure efficient and smooth operation.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

## 1.3 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

Covers and suction cups are excluded from this.

## 1.4 Compliance

Quality is of the utmost importance for our company; all processes are based on the ISO 9001 and ISO 13485 standards.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

The product meets the requirements of the standards EN ISO 14971 (Medical devices - Application of risk management to medical devices), EN ISO 10535 (Hoists for the transfer of disabled persons).

It has a lift capacity greater than 1.5 times the nominal load.

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

- We comply with the current environmental legislation (e.g. WEEE and RoHS directives).
- We only use REACH compliant materials and components.

For further information please contact Invacare® in your country (for addresses, see back page of this manual).

## 1.5 Intended use

Bath lifts facilitate respectively allow the entry into and exit from the bathtub. They are intended solely as a bathing aid for people with reduced mobility. Any other use is prohibited.

### Indications

- Significant reduction in function / missing functionality of the upper and/or lower extremities (such as amputations, paralysis, joint disorders, neuromuscular disorders with impairment of the extremities), when a independent entry into and exit from the bathtub respectively sitting down and up is no longer possible; Trunk control and basic functional capability of the extremities must still be sufficient.
- Chronic consequences of illness with functional limitations of the upper and/or lower extremities, that no longer permit an independent bathing, however when a largely independent bathing is possible by the use of the bath lift and other bathing aids for compensating the disability are not sufficient.

### Contraindications

There are no contraindications associated with proper use.

## 1.6 Service life

The expected service life is five years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this manual.

## 2 Safety

### 2.1 Safety information

#### **WARNING!**

- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment. Invacare product manuals are available on the internet or your local dealer (Addresses are displayed on the back cover of this manual).
- If you are unable to understand the warnings, cautions or instructions, please contact a healthcare professional, dealer or technical personnel before attempting to use this equipment – otherwise, injury or damage may occur.

#### **WARNING!**

#### **Risk of injury**

- Do not use if defective.
- In the event of a malfunction, contact your dealer immediately.
- Do not make any unauthorized changes or alterations to the product.
- Only use spare parts intended for this product.

#### **WARNING!**

#### **Risk of injury**

- Never stand on the bath lift to get into or out of the bathtub, do not use it for climbing or descending, as a lifting platform or for any similar purpose.

#### **IMPORTANT!**

- The information contained in this document is subject to change without notice.
- Check all parts for shipping damage and test before using.

### 2.2 Safety information on electromagnetic compatibility

This product was successfully tested to international standards for its electromagnetic compatibility. However, electromagnetic fields such as those generated by radio and television transmitters, radio equipment and mobile telephones, can influence the operation of products with electrical drives. The electronics used in this product can also cause weak electromagnetic interference but this lies below the statutory limits. You should therefore observe the following warnings:

**CAUTION!**

**Electromagnetic interference**

Wireless communications equipment such as WLAN routers, mobile phones, cordless telephones and their base stations or walkie-talkies can affect this equipment.

- Make sure that such devices are kept at least in a distance of 2 m away from the bath lift.

**CAUTION!**

**Electromagnetic interference**

Electromagnetic interference could cause the device to stop suddenly or (though this is extremely unlikely) to start moving of its own accord.

- Switch off the external device and/or your Invacare® product if you notice such behavior.
- If other electrically operated medical products cause a interference or are themselves disrupted, switch off your Invacare® product and contact your dealer.

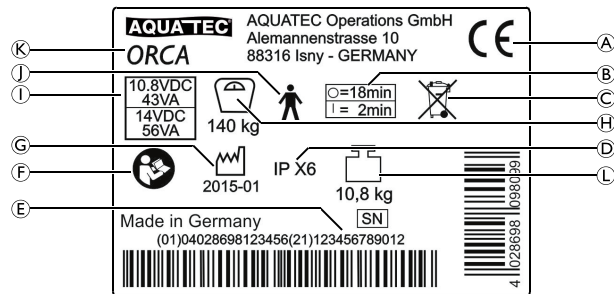
Guidance and manufacturer's declaration relating to electromagnetic emission and immunity are available on request from Invacare® in your country.

**2.3 Labels and symbols on the product**

**Identification label**

The identification label contains important information:

(A)	Conformity mark
(B)	Interval of use (continuous operation of the engine is not allowed): max. 2 minutes continuous use followed by a 18 minute break
(C)	Note (disposal)
(D)	Type of protection
(E)	Serial device number
(F)	Note (observe accompanying documents)
(G)	Production timeframe
(H)	Load capacity
(I)	Nominal voltage/ rated power, see 9.2 Electronic data, page 25
(J)	Type B applied part
(K)	Device designation
(L)	Product weight

The identification label is applied to the back of the backrest.

## 3 Setup

### 3.1 Safety information

#### IMPORTANT!

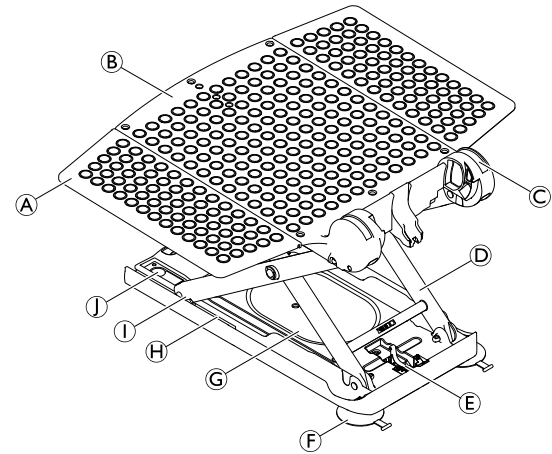
- Check the parts for transport damage before commissioning and contact the dealer if required.
- Ensure during assembly that the parts are positioned correctly with respect to one another.
- It is recommended to wash the covers before first use, see chapter 6.2 Cleaning and disinfection, page 20.

### 3.2 Scope of delivery

This user manual refers to the models ORCA, ORCA F and ORCA XL. These Models differ in weight and the maximum user weight, see section 9.1 Dimensions and weight, page 25. A seat- and backrest cover is optionally available for all three models. The backrest can not be lowered on model ORCA F.

The following components are included within the scope of delivery:


#### Seat

(A)	Side flap
(B)	Seat plate
(C)	Hinge for backrest
(D)	Lifting scissors
(E)	Spindle foot retainer
(F)	Rear suction cups (two tabs with thread)
(G)	Base plate
(H)	Guide for the sliding shoe


①	Sliding shoe
①	Front suction cups (one tab with head)

### Backrest

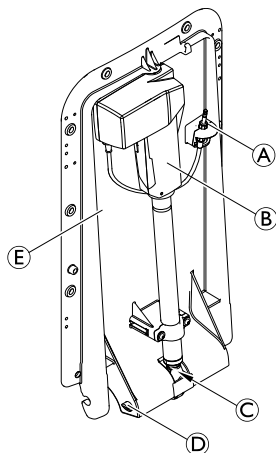Ⓐ	Jack plug
Ⓑ	Drive
Ⓒ	Spindle foot
Ⓓ	Locking pin
Ⓔ	Plate, backrest

### Hand control and charger

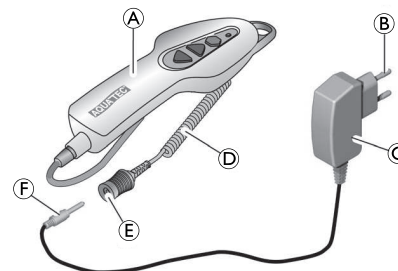Ⓐ	Hand control with battery
Ⓑ	Mains plug
Ⓒ	Charger
Ⓓ	Hand control spiral cable
Ⓔ	Jack socket of hand control
Ⓕ	Jack plug of charger

### 3.3 Setting up the bath lift

#### **WARNING!**

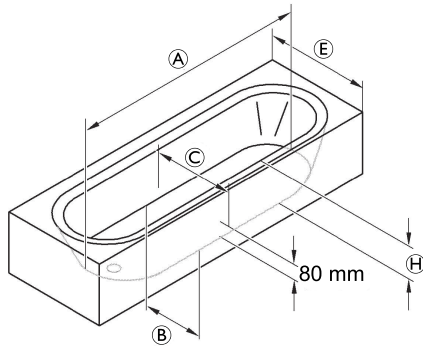
#### **Risk of slipping**

The bath lift can slip if the suction cups do not have a good hold.

- Clean the bathtub and the suction cups with a damp cloth before setting up the bath lift.
- Do not change the position of the bath lift in the bathtub once it is set up.

**WARNING!****Risk of injury and damage!**

- Only set up the bath lift in bathtubs that comply with the specifications of the following table.
- If a lateral overflow is present in the bath tub, use the special side flaps (available as an accessory).


Ⓐ	Tub length, bottom	≥ 800 mm
Ⓑ	Tub width, bottom (inside)	min. 280 mm
Ⓒ	Tub width, at 80 mm height (inside)	≥ 430 mm


Ⓔ	Tub width (outside)	580 - 690 mm (with special side flaps) 700 - 740 mm (with standard side flaps) 750 - 1000 mm (with special side flaps)
Ⓕ	Tub height (inside)	≤ 420 mm (with height adapter, max. + 60 mm)

**IMPORTANT!**

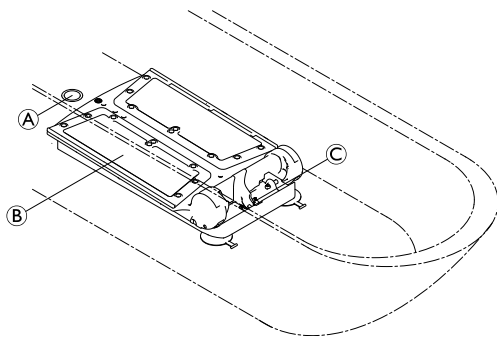
- Ensure that the suction cups are mounted.
- Ensure that the battery in the hand control is charged.
- Ensure that the spindle foot is fully inserted.
- When lifting bath lift components, remember to correctly assess the weights, see section 9.1 Dimensions and weight, page 25.
- If a handrail is mounted at the inside of the bathtub, use the side flap diverters (available as an accessory).
- Fold the side flaps outwards before raising the backrest.

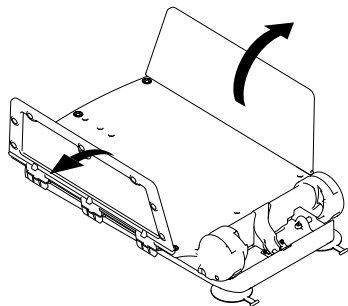1. Place the seat ② on the bottom of the empty bathtub. Ensure that the spindle foot retainer ③ is pointing towards the end of the bathtub opposite the drain ①.

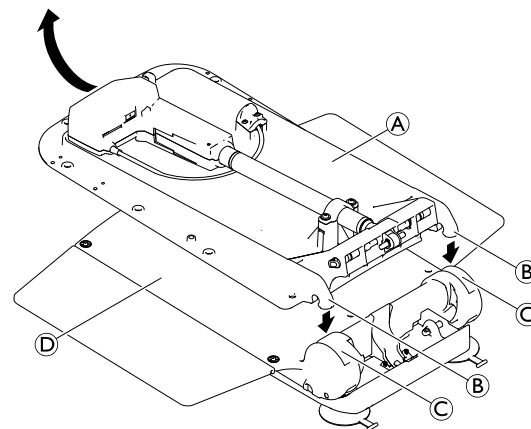2. Fold the side flaps of the seat outwards.

3. Place the backrest ① (without the hand control) onto the seat so that the hooks ② of the backrest fit into the hinges ③ of the seat plate ④.


**IMPORTANT!**

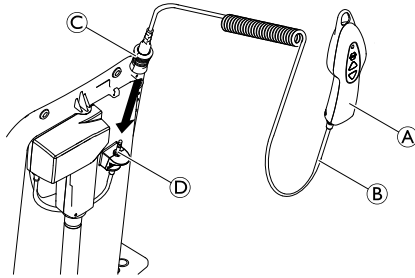**Risk of damage**

Damage to the product caused by the spindle foot slipping out.

- Ensure that the spindle is fully retracted and the spindle foot is level and not twisted, before folding up the backrest.
- Once the backrest is folded up, ensure that the spindle foot is in the spindle foot retainer.

4. Hold the backrest at the center of the head end and carefully lift it into the end position.

5. Plug the jack socket with the unlocking sleeve (C) on the spiral cable (B) of the hand control (A) to the jack plug (D) until the locking mechanism engages noticeably.

**!** **IMPORTANT!**  
**Risk of damage**

- Make sure that the backrest doesn't touch the bath tub edge.
- Place the bath lift further forwards in the bath tub, if necessary.

### 3.4 Charging the battery

The bath lift cannot be used when the battery is charging.

**WARNING!**

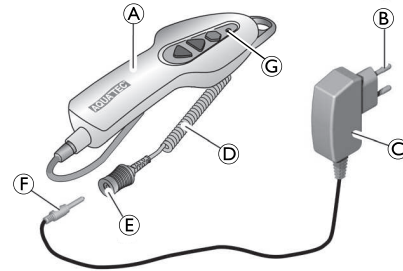
**Risk of electric shock**

- Only recharge the battery in dry rooms at room temperature (15 °C to 25 °C) and never in the bathroom.

**IMPORTANT!**

- Only use the charger supplied to charge the battery in the hand control.
- Only connect the charger to the mains when the hand control is connected to the charger.
- Make sure that the voltage specifications shown on the charger correspond to those of the mains supply.

1. Insert the jack plug (F) of the charger (C) into the jack socket (E) on the spiral cable (D) of the hand control (A) until it engages.
2. Insert the mains plug (B) of the charger into a power socket.

The red indicator lamp © on the hand control extinguishes when the battery is completely charged. It is not possible to „overcharge“ the battery.

3. Disconnect the charger from the power socket.
4. Pull the jack plug of the charger out of the jack socket of the hand control.

## 4 Usage

### 4.1 Safety information

#### CAUTION!

Overloading the bath lift can lead to damage to the device or prevent lifting.

- Do not overload the bath lift and observe the specifications on the identification label and in section 9.1 Dimensions and weight, page 25 with regard to its load capacity.

#### CAUTION!

##### Danger of pinching fingers

- Do not insert fingers into the guide between the seat plate and the backrest.
- Do not reach under the seat plate or between the lifting scissors.

#### IMPORTANT!

Moisture may penetrate into a damaged hand control, which could lead to a short circuit and fire.

- Always perform a visual inspection of the product for external damage before each use.
- If the housing of the hand control is damaged or shows cracks, do not use the hand control.
- Recharge the battery completely before each use, 3.4 Charging the battery, page 13.

1. Refer to operating conditions in section 9.4 Environmental Parameters, page 26.

### 4.2 Controlling the bath lift

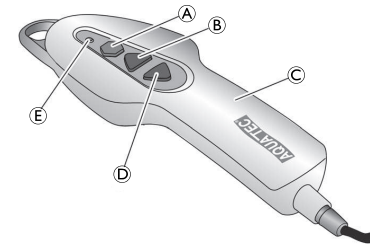
The bath lift can be moved in the following directions:

The bath lift is controlled via the hand control, either by the user or an assistant.

With a fully charged battery, the ORCA and ORCA F can perform 5 bathing operations with a load of 140 kg. The ORCA XL can perform 4 bathing operations with a load of 170 kg.

#### IMPORTANT!

##### Risk of damage

- The hand control may only be used when the backrest is properly installed and in the upright position.

**!** **IMPORTANT!**

If the battery in the hand control © is not sufficiently charged, the red indicator lamp ⑤ lights up red when the Down button ④ is pressed. In this case, the lowering function of the bath lift is disabled. You can however still raise the lift.

- Recharge the battery immediately after raising it.

### Raising the bath lift

**!** **IMPORTANT!**

If the backrest is lowered, it will move into the upright position first (not on model ORCA F).

1. Press the Up button ② on the hand control and keep it pressed.  
The backrest moves into the upright position or the seat moves upwards.
2. Release the Up button on the hand control.  
The bath lift stops in the current position.

### Lowering the bath lift

**!** **IMPORTANT!**

If the seat is already lowered, the backrest lowers towards the rear (not on model ORCA F).

1. Press the Down button ④ on the hand control and keep it pressed.  
The seat moves downwards or the backrest swivels to the rear.
2. Release the Down button on the hand control.  
The bath lift stops in the current position.

### Emergency off switch

**!** **IMPORTANT!**

Only press the Emergency Off button ① if the device does not stop immediately when the Up or Down button is released (e.g. if a button jams). The bath lift stops moving immediately and the red light integrated in the hand control lights up.

If the red indicator light goes out after the emergency off button is released, the fault has been rectified (e.g. button is released again). The device is ready to use again.

If the red indicator light stays on after the emergency off button is released, the fault has not yet been rectified. The device is not ready for use and must be repaired, e.g. in the event of damage to the electronics.

Contact your specialist dealer, if necessary.

## 4.3 Bathing

**WARNING!**

**Risk of slipping**

The bath lift can slip if the suction cups do not have a good hold.

- Before using the bath lift ensure that it is securely positioned and cannot tip over.
- Ensure that the bath lift is set up in the bathtub, → 3.3 Setting up the bath lift, page 10.

**WARNING!**

- If your condition means that you cannot take a bath on your own, only use the device with supervision.
- If operating the device without supervision, make sure to have an alarm system or telephone within reach in case of unforeseen events (e.g. malfunctions).

**CAUTION!**

- Bath oils or salts can impair the performance of the lifting scissors.
- If using such bath additives, do not exceed the recommended dose.

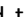
**IMPORTANT!**

- Run the unloaded lift once to the top and then back to the bottom (function test).
- Before using, check the temperature of the bath water.

1. Press and hold the Up button  until the seat is level with the edge of the bathtub.
2. Make sure that the side flaps are level with the seat and rest flat on the edge of the bathtub.
3. Fill the bath with water.
4. Sit on the side flap nearest to you as you would on a chair.
5. Slide backwards until you are sitting in the middle of the seat.
6. Turn yourself into the bathing position, by lifting one leg and then the other over the edge of the bathtub.

7. Once you are sitting correctly, press the Down button  and hold it down to lower the seat into the bathtub.

Once the seat has reached the lowest position, it is possible to recline the backrest by continuing to hold the Down button . This produces a more comfortable bathing position and allows you to immerse yourself deeper in the bath water (not on model ORCA F).

8. Keep the Down button pressed until the backrest has reached the desired position.

To get out of the bathtub, following these steps in reverse order.

9. To raise the backrest or the bath lift, press the Up button  and keep it pressed until it has reached the desired position.

**IMPORTANT!**

- Disconnect the hand control from the device when the bath lift is not in use, see section 5.2 Removing the bath lift, page 18, step 3.

## 5 Transport

### 5.1 Safety information

#### CAUTION!

The backrest is not locked on the seat when it is folded together and can come apart.

- Do not remove the suction cups when transporting the bath lift.
- Remove the hand control from the backrest for transport.
- Refer to the storage and shipping conditions in section 9.4 Environmental Parameters, page 26.

1. Remove the hand control.
2. Disassemble the bath lift into two sections for transport (seat and backrest), → 5.2 Removing the bath lift, page 18.

### 5.2 Removing the bath lift

1. Let the water out of the bathtub.
2. Lower the bath lift using the Down button until the backrest stops in the lowered end position, → 4.2 Controlling the bath lift, page 15.

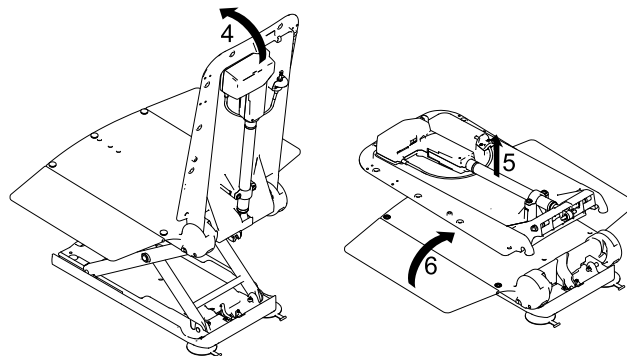
3. Unlock the jack socket on the spiral cable of the hand control by pushing up the unlocking sleeve and pull it off the jack plug.

#### IMPORTANT!


##### Risk of damage

- When folding the backrest down, ensure that spiral cable of the hand control does not become trapped.
- Do not fold the backrest down if there is water in the bathtub.

4. Hold the backrest at the head end with one hand and fold it forwards to the end position.
5. Pull the backrest up off the seat and out of the bathtub.

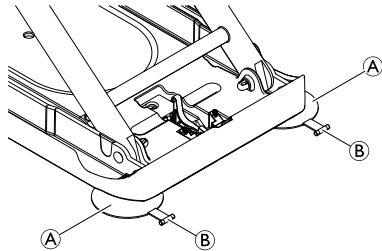6. Fold the side flaps of the seat inwards.

**IMPORTANT!**

- Release both rear suction cups simultaneously.

7. Release the rear suction cup (A) by pulling up on its tab (B).
8. Lift the rear of the seat further. This releases the front suction cups.

**IMPORTANT!**

- If the height adapters have been installed (optional), you also have to pull the tabs of the front suction cups to release them.

9. Lift the seat out of the bathtub.
10. Carefully store all components of the bath lift.

## 6 Maintenance

### 6.1 Maintenance and servicing

The UK Health and Safety Executive's Lifting Operations and Lifting Equipment Regulations 1998 (LOLER), require any equipment that is used in the workplace (e.g. by a paid carer) to lift a load be subject to safety inspection on a six monthly basis. Please refer to the HSE web site for guidance [www.hse.gov.uk](http://www.hse.gov.uk). Link to HSE information document on LOLER in Health and Social Care: <http://www.hse.gov.uk/pubns/hsis4.pdf>

If the cleaning and safety notices are observed, the product will be maintenance free.

#### **!** IMPORTANT!

The hand control may only be opened by the manufacturer of the bath lift.

1. Check the product regularly for damage and to ensure that it is securely assembled.
2. In the event of damage and/or necessary repairs, contact the dealer immediately.
3. The bath lift must be checked by an authorized dealer before every re-use. It is recommended to check the bath lift after 24 months at the latest (in accordance with EN ISO 10535).
4. If the bath lift is not used for a prolonged period, charge the battery completely and then raise and lower the bath lift 3 times without load. Repeat this process every 3 months. This helps to maintain the service life of the battery.

### 6.2 Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

1. Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

#### **!** IMPORTANT!

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.

- For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CG139](http://www.nice.org.uk/CG139) and your local infection control policy.
- Clean the product regularly by hand.
- Do not use abrasive cleaning agents.

#### Cleaning the seat and backrest

#### **CAUTION!**

##### **Danger of pinching fingers**

- Be especially careful when cleaning in the area of the lifting scissors, as they may move and cause injury.

#### **!** IMPORTANT!

##### **Risk of damage**

- Never use a high-pressure cleaner to clean the backrest.

1. Wash the product using a cloth or a brush.
2. Rinse the product with warm water.
3. Dry the product with a cloth.
4. Lightly grease the guide for the sliding shoe (Fig. Sec. 3.1, ) using Vaseline.

### Cleaning the covers (optional)

It is recommended to wash the covers before first use. For hygienic reasons it is recommended to clean the covers regularly and exchange the covers yearly.

1. Unbutton the covers from the seat and backrest.
2. Wash the covers in a washing machine at max. 60 °C using a mild detergent.

#### **IMPORTANT!**

##### **Risk of damage**

- Washing at high temperatures can cause shrinkage.
- Do not dry covers in the tumble dryer.

### Cleaning the charger and hand control

#### **WARNING!**

##### **Risk of electric shock**

- Always disconnect the plug from the power socket before cleaning the charger.
- Never insert objects made of conductive material (e.g. knitting needles, metal pins) into the jack socket of the hand control.
- Never use a wet or damp cloth to clean the charger.

1. Clean the charger with a dry cloth.
2. Wipe the hand control with a damp cloth and dry it with a dry cloth.

## 7 After Use

### 7.1 Storage

#### **IMPORTANT!** **Risk of damage**

- Do not store the product near heat sources.
- Do not expose the product to direct sunlight.
- Refer to the storage and shipping conditions in section 9.4 Environmental Parameters, page 26.

### 7.2 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly (→ 6.2 Cleaning and disinfection, page 20) and check the safety functions, and reset if necessary.

### 7.3 Disposal

The disposal and recycling of used products and packaging must be conducted in accordance with the applicable statutory regulations.

#### **WARNING!**

- Never throw the hand control into a fire or store it near naked flames.

Duty to inform in accordance with German Battery Directive, BattG:

This product contains a rechargeable lithium-ion battery. Discharged batteries or those which can no longer be

recharged must not be disposed of in domestic waste. Old batteries may contain hazardous substances which could be a danger to the environment and public health.

Please return such batteries to your retailer or to your municipal collection point. Returning them is free of charge and is required by law.

Please only throw discharged batteries into the containers provided and mask the poles of lithium batteries.

## 8 Troubleshooting

### 8.1 Identifying and repairing faults

The following table provides information on malfunctions and their possible causes. If you cannot correct the malfunction using the remedies named, please contact your dealer directly.

Fault	Possible cause	Remedy
No motor noise audible/lift does not move.	Jack socket of the hand control is not locked correctly.	Lock the jack socket, → 3.3 Setting up the bath lift, page 10.
	Battery is flat.	Recharge the battery completely, → 3.4 Charging the battery, page 13.
	Contacts are dirty.	Clean the plug on the backrest, → contact your dealer, if necessary.
	Cable is crushed/damaged.	Replace the hand control, → contact your dealer, if necessary.
The lift can be raised but not lowered.	Low-voltage protection is activated.	Recharge the battery completely, → 3.4 Charging the battery, page 13.
The lift stops while being raised.	Excessive load.	Observe maximum load, → 9.1 Dimensions and weight, page 25.
The bath lift is not adhering securely to the bathtub.	Suction cups are old or damaged.	Replace the suction cups, → contact your dealer, if necessary.
Battery is not charging and the red indicator light flashes during charging	Faulty electronics.	Have your authorized dealer check the hand control.
	Faulty battery.	Replace the hand control.

<b>Fault</b>	<b>Possible cause</b>	<b>Remedy</b>
	Faulty charger.	Have your authorized dealer check the charger.
	Battery is too hot.	Do not charge the hand control near heat sources, e.g. heating elements, in direct sunlight. Charging continues automatically once the temperature drops to within permissible limits.

## 9 Technical data

### 9.1 Dimensions and weight

Total length at lowest position	920 mm
Total length at highest position	650 mm
Total height at highest position	1075 mm
Seat length	590 mm
Seat width with side flaps folded out	715 mm
Seat width with side flaps folded in	390 mm
Seat height at lowest position	60 mm
Seat height at highest position	420 mm
Backrest width	360 mm
Minimum angle of backrest	8°
Base plate width	295 mm
Max. load (ORCA / ORCA F) Max. load (ORCA XL)	140 kg 170 kg
Total weight without hand control (ORCA) (ORCA F) (ORCA XL)	10,8 kg 11,4 kg 11,6 kg
Seat weight (ORCA) (ORCA F) (ORCA XL)	6,6 kg 7,2 kg 7,4 kg
Backrest weight (ORCA / ORCA F / ORCA XL)	4,2 kg

Hand control weight	0,4 kg
Charger weight	0,1 kg

### 9.2 Electronic data

Activation force for the buttons of the hand control	< 5 N
Hand control operating voltage <sup>1)</sup>	10.8 V DC / 14 V DC
Rated input power <sup>1)</sup>	43 VA / 56 VA
Rated current	4 A
Battery type	Lithium-ion
Charger input	100 - 240 V AC, 50 - 60 Hz
Charger output	24 V DC, 135 mA, 3 VA
Charge time from low-voltage warning	approx. 4.5 h
Type of protection, drive	IP X6 W <sup>2)</sup>
Type of protection, hand control	IP 67 <sup>3)</sup>
Type of protection, entire device	IP X6 <sup>4)</sup>
Sound power level	52 dB(A)

<sup>1)</sup> 10,8 V DC / 43 VA valid when using hand controls with voltage 10.8 V, indicated on the back of the hand control.

14 V DC / 56 VA valid when using hand controls with voltage 14 V, indicated on the back of the hand control.

- 2) Protected against the penetration of water in the event of strong jets of water, tested under stipulated weather conditions.
- 3) Dust-proof and protected against the penetration of water during temporary immersion.
- 4) Protected against the penetration of water in the event of strong jets of water.

Relative humidity	30 % - 75 %, non-condensing
Atmospheric pressure	795 - 1060 hPa

### 9.3 Materials

Backrest	PPGF
Seatplate	PPGF
Covers	PVC

 All materials used are corrosion resistant and latex free.

### 9.4 Environmental Parameters

<b>Operating conditions</b>	
Ambient temperature	10 - 40 °C
Relative humidity	15 % - 93 %, non-condensing
Atmospheric pressure	795 - 1060 hPa
<b>Storage and shipping conditions</b>	
Ambient temperature	0 - 40 °C

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Invacare AB
Invacare Australia PTY. Ltd.
Aquatec Operations GmbH